Pantone 326C
Pantone 1945C
Pantone 425C
Black

18.5 inches Width

ENZYMAX® LIQUID | LIQUIDE

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For professional use only. Shake well before using. Use with water at temperature between 90° and 140° F (32° to 60° C), demineralized water is ideal.

Prosoak: Add 1 fl. oz. (30mL) of concentrate per 1 gallon (3.8L) of water. Soak for 3-5 minutes. Rinse thoroughly. **Linen Prosoak:** Add 2 fl. oz. (60mL) per 1 gallon (3.8L) of water. Soak clothing for 10 minutes. Proceed with normal washing. **Ultrasonic Cleaning:** Add 1 fl. oz. (30mL) of concentrate per 1 gallon (3.8L) of water. After cleaning, rinse and dry thoroughly. **Evacuators:** Add 2 fl. oz. (60mL) per 1 gallon (3.8L) of water. Insert evacuator hose, turn system on full open for one minute. Suggested maintenance twice weekly.

Store in a cool, dry place. Change daily or when visibly soiled.

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International : 1-352-323-3500

► **DANGER**

May cause allergy or asthma symptoms or breathing difficulties if inhaled.

Avoid breathing dust/fume/gas/mist/vapours/spray. Wear respiratory protection. **IF INHALED:** If breathing is difficult, remove person to fresh air and keep comfortable for breathing. **IF EXPERIENCING RESPIRATORY SYMPTOMS:** Call a POISON CENTER/doctor. Dispose of contents/container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation.

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Manufactured for | Fabriqué pour: Hu-Friedy Mfg. Co., LLC