

The Hu-Friedy Waterline Filter is EPA registered for use as an antimicrobial agent in dental unit waterlines. It delivers clean water to patients and staff for up to one year and requires no daily, weekly, or monthly protocols.

The lifetime of this product is 240 liters or 365 calendar days (if water usage records are not monitored) or if iodine levels fall to less than 0.5 ppm. To maximize the life of the filter, shut off water to the dental unit when not in use.

Use the chart below to monitor the volume of water used through the unit. The chart includes 240 boxes to represent the 240-liter lifespan of the filter. Check off a box in the chart for each liter of water that flows through the filter. Once the chart is filled, you should replace your Hu-Friedy Waterline Filter.

[illegible]

Install Date:

Installed By:

Lot Code:

To reorder, call 1-800-HU-FRIEDY or order through your dental distributor.

Product Name: Hu-Friedy Waterline Filter

Part Code: IMS-1440

Visit us online at www.Hu-Friedy.com

Call 1-800-Hu-Friedy or contact your authorized Hu-Friedy representative for more information.

©2026 Hu-Friedy Mfg. Co., LLC. All rights reserved. [980] IMS-3219/0818

How the best perform

