





# HU-FRIEDY WATERLINE FILTER

## INSTALLATION WITH A-DEC 500 AND 300 BOTTLES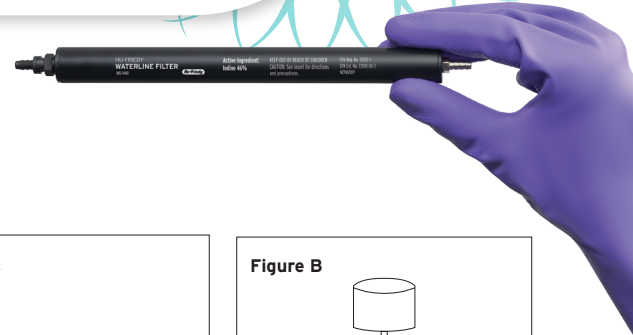

### DIRECTIONS

1. Remove A-dec bottle from the manifold.
2. Locate and remove two hex screws from collar with 1/8" allen wrench (Figure A).
3. Separate collar and lift the riser tube assembly from the bottle (Figure B).
4. Hold the Hu-Friedy Waterline Filter alongside the riser tube and mark the riser tube to allow for at least a 1/4" gap between the end of the Hu-Friedy Waterline Filter and the bottom of the bottle (Figure C).
5. Cut off the lower section of the riser tube at the mark using a tubing cutter, sharp knife or scissors (Figure C).
6. Push a tubing sleeve onto the riser tube, then push the hose barb attached to the Hu-Friedy Waterline Filter into the tubing, making sure that it is fully engaged (Figure D).
7. Slide the tubing sleeve down over the hose barb, and make sure that the barb fitting is firmly threaded into the Hu-Friedy Waterline Filter body.
8. Insert riser tube assembly with Hu-Friedy Waterline Filter into the bottle.
9. Aligning the notches inside the collar with the bottle and riser tube assembly, carefully reconnect the collar to the bottle (Figure E). Do not overtighten.
10. Return the A-dec bottle containing the Hu-Friedy Waterline Filter to the manifold

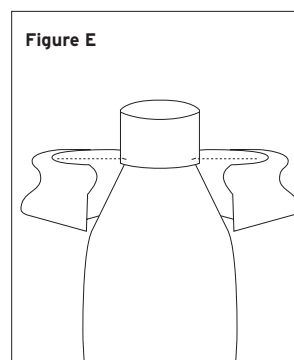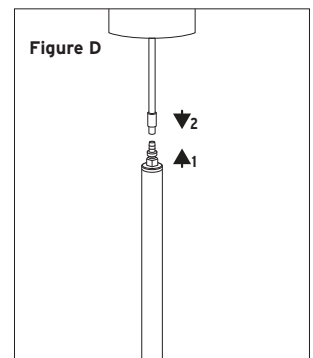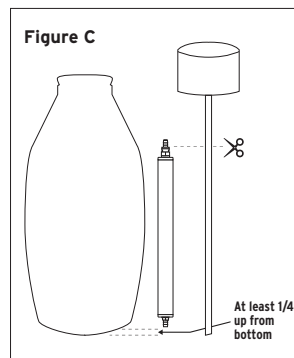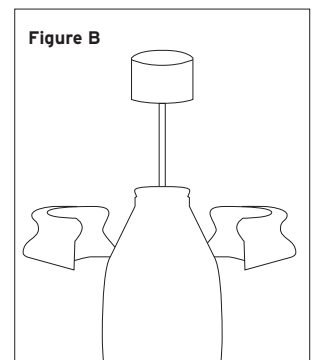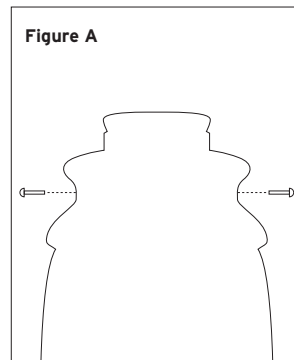For more information, including an installation video, visit [www.Hu-Friedy.com/Dental-Waterline-Cleaner](http://www.Hu-Friedy.com/Dental-Waterline-Cleaner)

Call 1-800-Hu-Friedy or contact your authorized Hu-Friedy representative for more information.  
©2019 Hu-Friedy Mfg. Co., LLC. All rights reserved. [980] HFL-178/0219

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©2019 Hu-Friedy Mfg. Co., LLC. Tous droits réservés. [980] HFL-178/0219