## PRECAUTIONARY STATEMENTS/MISES EN GARDE

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**CAUTION** Causes moderate eye irritation. Avoid contact with eyes, skin and clothing. Wear protective eyewear (goggles) and gloves when handling. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco or using the toilet. Remove and wash contaminated clothing before reuse.

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|                                               |                |
|-----------------------------------------------|----------------|
| Iodine/Iode*                                  | 46.0%/46,0 %   |
| Other Ingredients/Autres ingrédients. . . . . | 54.0%/54,0 %   |
| TOTAL . . . . .                               | 100.0%/100,0 % |

\*Bound to polystyrene divinyl benzene quaternary ammonium anion exchange resin.

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IMS-1440CA

## DIRECTIONS FOR USE/MODE D'EMPLOI

Prior to installing Hu-Friedy Waterline Filter, thoroughly clean dental unit waterlines following the manufacturer's instructions or with an appropriate cleaning agent. Insert Hu-Friedy Waterline Filter into the inlet and outlet fittings of dental unit waterlines.

Once the product is in place, the supply water will pass through the iodinated resin bed, which will continuously release low levels of antimicrobial iodine. After installation, check water flow and verify that the unit is operational. Create a check-off sheet to monitor volume of water used through unit. Shut off water to the dental unit when not in use.

The lifetime of this product is 240 liters or 365 calendar days (if water usage records are not monitored) or if iodine levels fall to less than 0.5ppm. After this period or level is reached, immediately replace the spent Hu-Friedy Waterline Filter with a new or replacement unit. To install the new unit, follow the instructions above for installing a new product.

If Hu-Friedy Waterline Filter is used in conjunction with a bottle water system: The bottle in water bottle systems must be emptied each night and the empty bottle replaced on the manifold. Before daily use, wash the inside of the bottle and rinse thoroughly. Wipe down the outside of Hu-Friedy Waterline Filter and make sure the intake end is not obstructed. Fill with fresh water each morning before its first use. Replace Hu-Friedy Waterline Filter after 240 liters of water usage, if iodine level falls to less than 0.5ppm, or if records of usage are not kept, after 365 calendar days of installation.

If package seal is broken or product appears dry, wrap in newspaper and dispose of it in the trash.

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**Troubleshooting:** If the water flow after product installation is slow or inhibited, check to ensure that the product was installed properly or if the product has been plugged by debris. For bottle installation ensure that the air supply is turned on at the maximum level (45 psi). In addition, ensure that the product is at the bottom of the water bottle. If there is an air leak from fittings, contact your dental supplier for replacement parts or service. Discoloration of tubing from yellow to blue to green to black is normal and is not a concern. Initial discoloration of water is caused by release initial flush of excess iodine. Operate water until flow appears clear.

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## STORAGE & DISPOSAL/CONSERVATION ET ÉLIMINATION

**Storage:** Store at room temperature in original packaging, away from children, pets and direct sunlight. Avoid excessive heat. Do not allow cartridge to freeze. Do not store in open or unlabeled containers.

**Disposal:** Dispose of according to local laws and regulations

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## INSTALLATION INSTRUCTIONS/ INSTRUCTIONS D'INSTALLATION

### PRE-INSTALLATION NOTE:/REMARQUE AVANT L'INSTALLATION:

Thoroughly clean dental unit waterlines with an appropriate antimicrobial cleaning agent following manufacturer's instructions before installing the Hu-Friedy Waterline Filter for the first time. Hu-Friedy Waterline Filter's barb fittings are sized for use with 1/4" O.D. x 1/8" I.D. dental supply tubing.

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**Please note that the filter has no designated inlet or outlet. Either orientation is acceptable.**

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2. Hold the Hu-Friedy Waterline Filter alongside the riser tube hanging from the manifold and mark the riser tube as shown in **Figure A**. Be sure to allow for at least a 1/4" gap between the end of the Hu-Friedy Waterline Filter and the bottom of the bottle.

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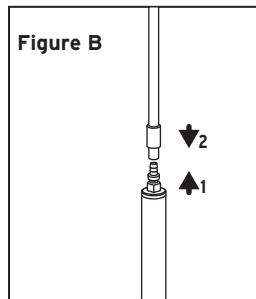
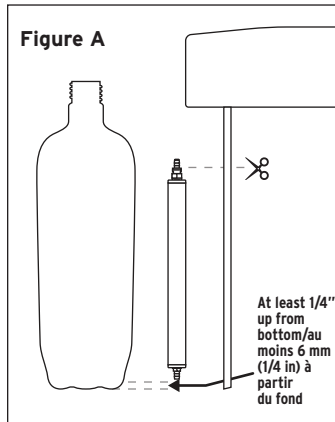
3. Cut off the lower section of the riser tube at the top mark using a tubing cutter, sharp knife, or scissors.

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5. Slide the tubing sleeve down over the hose barb, and make sure that the barb fitting is firmly threaded into the Hu-Friedy Waterline Filter body. Faire glisser le manchon de tube vers le bas sur le raccord cannelé, et

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7. Pressurize the bottle as normal, then begin discharging all dental appliance lines including handpieces, air/water syringe and quick-disconnect (if present) for a few minutes each. The system is now ready for use.

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1. Depressurize the system that will utilize the Hu-Friedy Waterline Filter.

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2. Hold the Hu-Friedy Waterline Filter alongside the tubing where the installation is desired and mark the section of tubing to be removed, as shown in **Figure C**.

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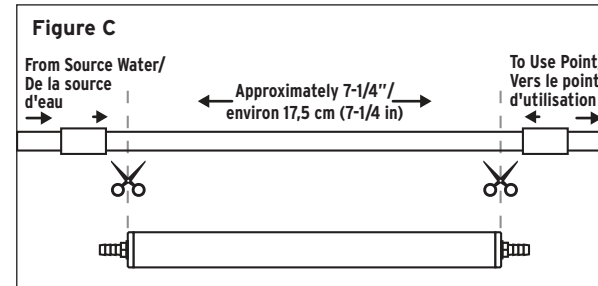
3. Cut the tubing at the marks using a tubing cutter, sharp knife, or scissors.

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4. Push tubing sleeves onto both sections of tubing, then push the hose barbs on each end of the Hu-Friedy Waterline Filter into the tubing, making sure that both hose barbs are fully engaged.

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5. Slide the tubing sleeves down over the hose barbs, and make sure that both barb fittings are firmly threaded into the Hu-Friedy Waterline Filter body.

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6. Reopen the valve at the water source. Allow water to flow through the new filter by opening any device after the filter. Flow water through each connected device (e.g. handpiece, air/water syringe, scaler, etc.) for several minutes. The system is now ready for use.

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### CHANGING THE FILTER: CHANGER LE FILTRE:

Change the Hu-Friedy Waterline Filter after 240 liters or one year (if water usage is not monitored), or if iodine levels fall to less than 0.5 ppm.

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1. Turn off and depressurize devices attached to filter.

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- a. Slide back the tubing sleeve(s) and pull the tubing off the barb(s) and remove filter. Install new filter by pushing tubing over the barbs on the new filter, then sliding the sleeves back into place over the barb(s).

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- b. Unthread the barb fitting(s) from the expired filter, leaving the barb(s) in place inside the tubing. Remove the expired filter. Unthread the barb(s) from the new filter. Ensure the seal washer(s) shown in **Figure D** stay in place and then thread the new filter onto the existing barb(s).

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
### CLEANING NOTE: REMARQUE CONCERNANT LE NETTOYAGE:

Before cleaning waterlines with any type of antimicrobial agent, always remove the Hu-Friedy Waterline Filter first. Replace the section of tubing temporarily, clean and rinse the lines, remove the temporary tubing, and then replace or reinstall the filter.

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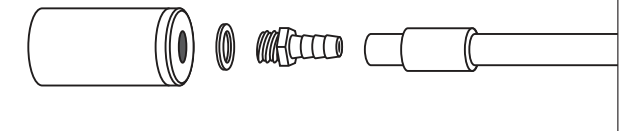**Figure D**

### NOTE:/REMARQUE :

The Hu-Friedy Waterline Filter features one barbed fitting that swivels to facilitate easier filter replacement in an inline installation. Before resuming operation after replacing a filter, always ensure that both barbed fittings are threaded securely into the filter body to prevent leaks.

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