Washer-Disinfector Cleaning Monitor

For Use in Routine Monitoring of the Wash Phase in Washer/Disinfectors

REF IMS-1200W

QTY 50

INDICATIONS FOR USE:

The single use Washer-Disinfector Cleaning Monitor is designed for monitoring cleaning efficiency during the wash process.

INSTRUCTIONS FOR USE:

1. Insert a Washer-Disinfector Cleaning Monitor strip (IMS-1200W) into Cleaning Monitor Holder (IMS-1200H) completely. The front side of the monitor with red test soil will be visible in the holder window, as shown below.

2. Position the Cleaning Monitor Holder face up inside a basket. Place basket on a shelf in the washer. Vary positions and shelves of Washer-Disinfector Cleaning Monitor placement cycle to cycle.
3. Run a complete washer cycle. We recommend monitoring every shelf on every washer each day for a machine release and using a minimum of one Washer-Disinfector Cleaning Monitor strip per load for routine monitoring throughout the day.
4. At the end of the washing cycle, remove the Washer-Disinfector Cleaning Monitor from the holder and read the result. Please see visual interpretation of results below. Caution: Holder and monitor strip may still be hot! Remove with care.

INTERPRETATION OF MONITOR RESULTS:

Untreated Monitor, or critical/poor result

Insufficient result

Optimum result

5. If desired, you may also record the location of monitor placement in the washer-disinfector on the strip itself with a fine tip waterproof marker. Place a hash mark or circle representing the position of the Wash-Checks on the location rectangle that simulates an imaginary washer basket. This allows for placement tracking.
6. If desired, staple the Washer-Disinfector Cleaning Monitor strip onto the Washer-Disinfector Cleaning Monitor Record Sheet and complete the appropriate information. Retain as a permanent record.

STORAGE:

- Store Cleaning Monitors inside the protective foil bag provided at 40-86°F (4-30°C). Do not remove the monitors from the foil bag until ready to use.
- Keep away from disinfectants and fumes.

EXPIRY DATE:

The expiry date is printed on the product packaging.

Please see website for Washer-Disinfector Cleaning Monitor Record Sheet and Troubleshooting Guide.

Hu-Friedy
Manufactured for:
Hu-Friedy Mfg. Co., LLC
2501 Sanders Rd.
Northbrook, IL 60062 | USA
1-800-Hu-Friedy | HuFriedyGroup.com

All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC, its affiliates or related companies, unless otherwise noted.
©2026 Hu-Friedy Mfg. Co., LLC. All rights reserved.

N1558 / REV 2 - 2512

HuFriedyGroup
The Best In Practice

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