


INSERT WEAR GUIDE

Worn tips can diminish your scaling efficiency.
Check your inserts every week to maintain
optimal performance.

 **HuFriedyGroup**
The Best In Practice

 **25% Efficiency Loss (Reorder)**
 **50% Efficiency Loss (Discard)**

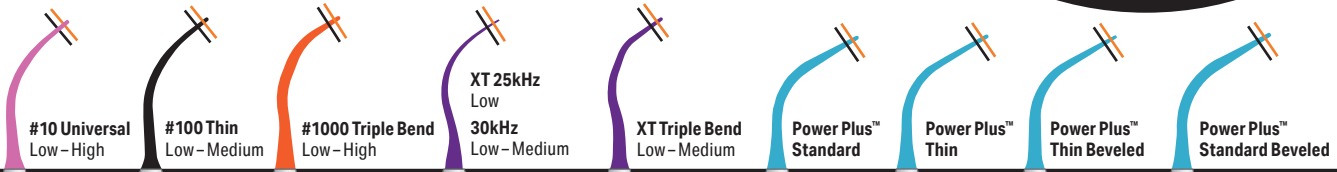

HOW TO USE YOUR WEAR GUIDE

1. Locate the diagram of the insert to be measured (use color as guide).
2. Position point **A** to point **B** as shown and/or align the curvature of the tip (**circled**) to match the guide.
3. Rotate insert until tip is flat against the card.
4. Evaluate tip condition, comparing length to red/blue bars.

Reorder tips when 25% efficiency loss (orange bar)

Discard tip when 50% efficiency loss (black bar)




To reorder inserts call 1-800-Hu-Friedy or your authorized HuFriedyGroup Distributor.

Learn more at **HuFriedyGroup.com/Inserts**

HuFriedyGroup.com

All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC, or its affiliates or related companies, unless otherwise noted.

©2026 Hu-Friedy Mfg. Co., LLC. All rights reserved. HF-466/2608

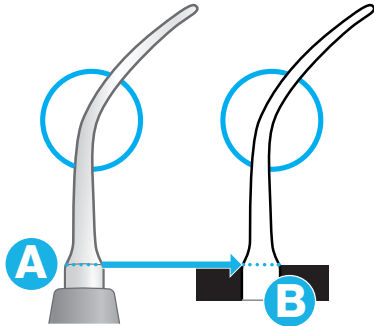

 **HuFriedyGroup**
The Best In Practice