

PIEZO INSERT WEAR GUIDE

How the best perform

Hu-Friedy

Left Perio
US4L

3 Perio
USP3, UEP3

P0 Universal
USP0

P Universal
USP, UEP

S10
USS10, UES10

100 Universal
Modified
UE100M

100 Universal
US100, UE100

Universal Perio
USU

After Five Right
USAFR

After Five Straight
USAFS

After Five Left
US4L

Right Perio
US4R




Try our probe sized tips which are designed to enhance access into periodontal pockets and improve adaptation in interproximal areas, allowing you to more thoroughly debride difficult to reach areas to achieve better clinical results.

AFTER FIVE RIGHT (USAFR) | AFTER FIVE STRAIGHT (USAFS) | AFTER FIVE LEFT (USAFL)

PIEZO TIP WEAR GUIDE INSTRUCTIONS

1. Locate the diagram of the tip to be measured.
2. Position point **A** to point **B** as shown and/or align the curvature of the tip to match the guide.
3. Rotate until tip is flat against the card.
4. Evaluate tip condition, comparing length to red/blue bars.

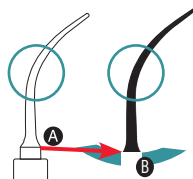

Tip extends beyond blue line
Optimum Efficiency
(Scale Away)



Tip touches blue line
25% Efficiency Loss
(Reorder)



Tip touches red line
50% Efficiency Loss
(Discard)

To reorder tips call
1-800-Hu-Friedy or your
authorized Hu-Friedy dealer

Hu-Friedy