



SIDEKICK® SHARPENER

Sharpening innovation at your fingertips

FACT

Scalers and curettes can dull over time as a result of routine, every day use. Keeping instruments sharp can improve deposit removal and decrease the potential for hand fatigue*.

CHALLENGE

Achieving a fine cutting edge without compromising the original shape of the instrument blade can be challenging and time-consuming.


SOLUTION

Hu-Friedy's Sidekick is a cordless, lightweight sharpening device designed to provide convenient and consistent sharpening results and accurate angles every time.



SHARPENING MADE EASY!


- Easy-to-read letters indicate Graceys (G) or Sickles and Universals (S/U)
- Position the instrument in the specific channel, with the terminal shank resting on the incline of the channel
- Position the back of the instrument along the backstop of the guideplate
- Turn the unit on, and glide the instrument within the channel from side to side
- Repeat 2 - 3 times or until blade is sharp

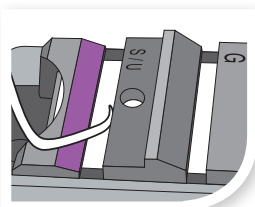


POINTS OF PERFORMANCE

- Easy-to-use guide channels and vertical backstops allow simple positioning of scaler and curettes to provide consistent sharpening results every time.
- Provides consistent sharpening results aiding in instrument longevity.
- Cordless, lightweight, portable sharpening...anywhere, anytime.
- Reciprocating stone does the work so you don't have to
- Maintains sharp blade and accurate angles


ACHIEVE ACCURATE ANGLES EVERY TIME

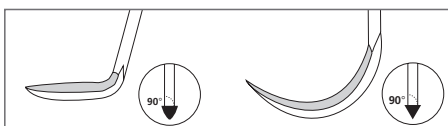

SICKLE SCALERS

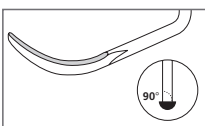

UNIVERSAL CURETTES

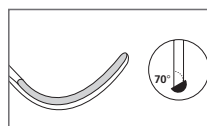

GRACEY CURETTES

*Fundamentals of Periodontal Instrumentation & Advanced Root Instrumentation, J. Nield-Gehrig, 6th Edition. Lippincott, Williams & Wilkins. 2008.

How the best perform



COARSE GRIT SHARPENING STONE

Create sharp edges faster with the new Coarse Stone for the Sidekick Sharpener!

- **Coarser stone grit** allows you to effortlessly keep your instrument edges feeling brand new
- **Dual sharpening sides** that provide longer stone life
- **Easy Sterilization Solution:** Stone can be autoclaved along with Sidekick guideplate and screw



ADDITIONAL ITEMS

REPLACEMENT PARTS

Stone Clip	SDKCLIP
Guideplate Screw	SDKGPSCREW
Guideplate	SDKPLATE
Ceramic Stone	SDKSTONEC
Sharpen-EZ™ Stone Oil	SSO
Plastic Test Sticks	PTS
Magnifying Lens	LENS
Large Parts Box	IMS-1273

SIDEKICK SHARPENING KIT: SDKKIT

Kit includes: Sidekick Unit, User Manual, Ceramic Sharpening Stone, Plastic Test Stick, Magnifying Lens and 2 AA Batteries.

Visit us online at [Hu-Friedy.com](https://www.hu-friedy.com)

Call 1-800-Hu-Friedy or contact your authorized Hu-Friedy representative for more information.
©2020 Hu-Friedy Mfg. Co., LLC. All rights reserved. Hu-Friedy, the Hu-Friedy oval logo, EverEdge and Sharpen-EZ are trademarks of Hu-Friedy Mfg. Co., LLC. [271] HF-592/0718

