Pantone 306C
Pantone 1788C
Black

6.5"

3"

DANGER

Causes skin irritation. Causes serious eye damage. May cause allergy or asthma symptoms or breathing difficulties if inhaled.

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IMS-1332
32 – 1/2 OZ. (14 GRAM) PACKETS
32 – 1/2 OZ (14 GRAMMES) SACHETS

BAR CODE FPO
+D67148102513

HuFriedyGroup
The Best In Practice

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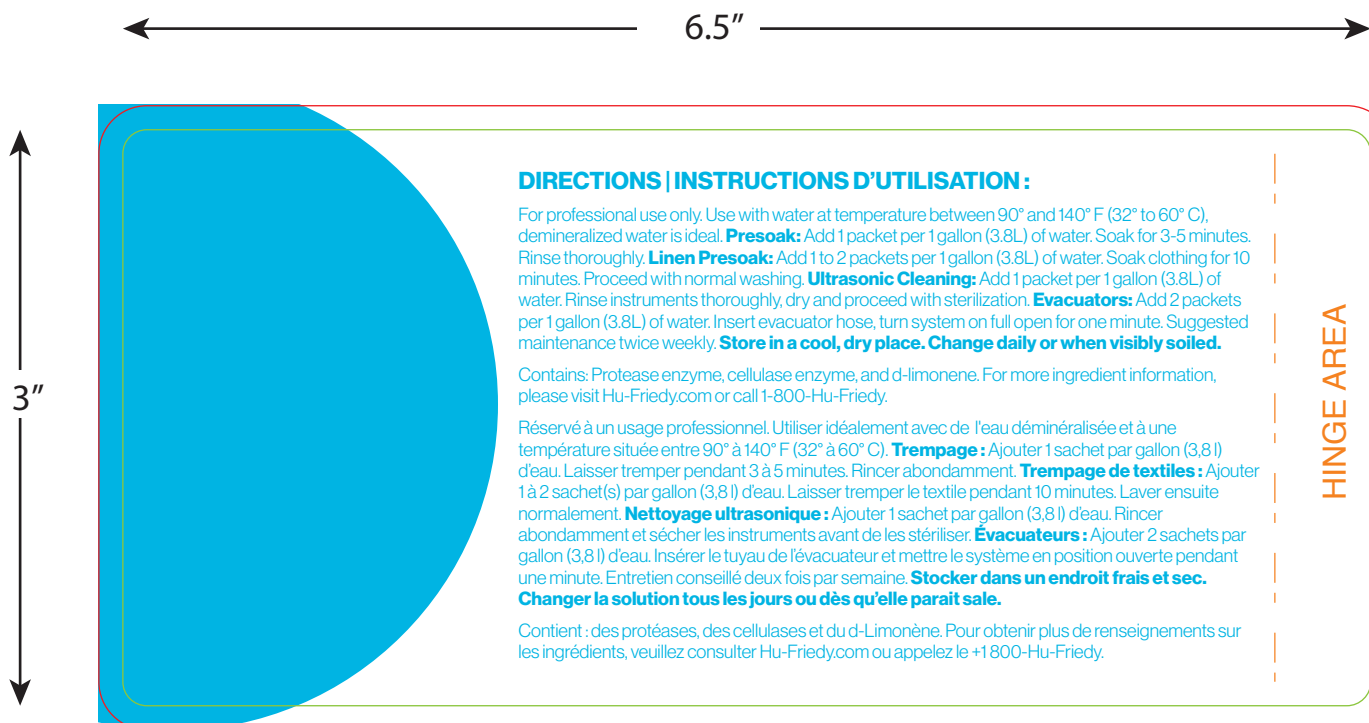
For professional use only. Use with water at temperature between 90° and 140° F (32° to 60° C), demineralized water is ideal. **Presoak:** Add 1 packet per 1 gallon (3.8L) of water. Soak for 3-5 minutes. Rinse thoroughly. **Linen Presoak:** Add 1 to 2 packets per 1 gallon (3.8L) of water. Soak clothing for 10 minutes. Proceed with normal washing. **Ultrasonic Cleaning:** Add 1 packet per 1 gallon (3.8L) of water. Rinse instruments thoroughly, dry and proceed with sterilization. **Evacuators:** Add 2 packets per 1 gallon (3.8L) of water. Insert evacuator hose, turn system on full open for one minute. Suggested maintenance twice weekly. **Store in a cool, dry place. Change daily or when visibly soiled.**

Contains: Protease enzyme, cellulase enzyme, and d-limonene. For more ingredient information, please visit Hu-Friedy.com or call 1-800-Hu-Friedy.

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6.5"

HINGE AREA

Avoid breathing dust/fume/gas/mist/vapours/spray. Wash hands, forearms and face thoroughly after handling. Wear protective gloves/protective clothing/eye protection/face protection. In case of inadequate ventilation wear respiratory protection. **If on skin:** Wash with plenty of water. Take off contaminated clothing and wash it before reuse. If skin irritation occurs: Get medical advice/attention. **If inhaled:** Remove person to fresh air and keep comfortable for breathing. If experiencing respiratory symptoms: Call a POISON CENTER/doctor. **If in eyes:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER/doctor. Dispose of contents/container in accordance with local, regional, national and/or international regulations.

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All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC, its affiliates or related companies, unless otherwise noted. | Tous les noms de société et de produit sont des marques de commerce de Hu-Friedy Mfg. Co., LLC, ses filiales ou ses sociétés apparentées, sauf mention contraire.
© 2026 Hu-Friedy Mfg. Co., LLC. All rights reserved. | Tous droits réservés.
Hu-Friedy Mfg. Co., LLC