Bur Cushion

IMS-1372S/T • IMS-1372SH/TH

EN

Directions for use:

- Remove stainless steel lid from silicone base by sliding the lid away from the base.
- Place the bur shanks into the base at the "x's".
- Insert burs deep enough so the tips do not touch the lid.
- Slide lid back onto the base along the grooves.
- Place the Bur Cushion inside the IMS™ cassette and process instruments.
(The Bur Cushion can be ultrasonically cleaned and sterilized without use of IMS cassettes. Follow the cleaning and care instructions from the bur manufacturer.)

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www.hufriedygroup.com/Reprocessing/
1-800-Hu-Friedy; info@hu-friedy.com

Notes:

- It is normal if burs are at slight angles when held within the Bur Cushion.
- Burs are gripped in the base. The lid is provided to protect the burs and other instruments within the IMS™ cassette.
- Burs are cutting instruments and should always be handled with care. Touch only the shank of the bur. Use a hemostat or dressing pliers to assist in handling burs.
- If the silicone base of the Bur Cushion discolors, wipe with isopropyl alcohol, then rinse thoroughly.
- If burs are carbon steel, a milk bath is recommended if using steam sterilization.

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MDSS CH GmbH
Laurenzenvorstadt 61
5000 Aarau
Switzerland

Made in USA

All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC, its affiliates or related companies, unless otherwise noted.
©2025 Hu-Friedy Mfg. Co., LLC. All rights reserved.

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Washer-disinfector for thermal
disinfection possible

PAPER
INSERT
D2564
REV 11 - 2508

HuFriedyGroup
The Best In Practice