How to Properly Use VistaClean™ Irrigant Solution Concentrate

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VistaClean™ Irrigant Solution Concentrate is an aqueous cleaner derived from citrus botanicals and is designed for use in dental waterlines. It may be used in waterlines continuously or intermittently as desired. It's produced from 100% natural botanicals by a proprietary process and contains powerful cleaning characteristics. As an aqueous cleaner it has the ability to effectively emulsify organic and inorganic contaminants, soils and oxidation products from waterlines. It also provides lubrication as an irrigant while protecting lines and system components from deposits and scale. When used according to directions, it is completely non-toxic, non-corrosive, simple to use and has no negative effect on bonding.

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For Bottle System Users

Initial Cleaning:

All bottles and waterlines should be cleaned with an EPA registered antimicrobial cleaner like VistaTab™ Dental Waterline Cleaner Tablets prior to use and at least once per month. See the VistaTab Tablet directions for complete details.

Daily Use:

1. Fill bottle to normal capacity with clean water.
2. Add the correct number of VistaClean Irrigant Solution Concentrate drops based on the size of the bottle as shown below:

Bottle Capacity	Min Drops	Max Drops
500 ml	1	2
750 ml	1	3
1.0 Liter	1	3
1.5 Liters	1	4
2.0 Liters	2	5

3. Return the bottle to manifold on the dental unit.
The water system is now ready for use with patients.

Note: Always start with a fresh batch each time the bottle is removed and filled. Use the above formula. If water remains in the bottle at the end of the day, the bottle may be left attached to system manifold and used the following day. The bottle containing water and VistaClean Solution drops may be left attached to the dental unit manifold when not in use to keep system sealed. If the water foams or patients notice an astringent taste, reduce the number of drops used when making each batch. VistaClean Solution should not be used as a sole waterline treatment, rather, in combination with an EPA registered antimicrobial cleaner for bacteria control. VistaTab Dental Waterline Cleaner Tablets IMS-1451 are recommended.

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General Note: Avoid contact with eyes. NEVER USE FULL STRENGTH.

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+D6713040181%

HuFriedyGroup
The Best In Practice

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All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC its affiliates or related companies, unless otherwise noted. Tous les noms de société et de produit sont des marques de commerce de Hu-Friedy Mfg. Co., LLC, ses filiales ou sociétés apparentées, sauf mention contraire.
©2021 Hu-Friedy Mfg. Co., LLC. All rights reserved. Tous droits réservés.