

HU-FRIEDY LINGUAL DEBONDING PLIERS


SPRING ACTION DESIGN
REQUIRES MINIMAL
FORCE TO SMOOTHLY
DEBOND LINGUAL
BRACKETS

These pliers aid in lingual bracket removal. The remarkable design allows for quick debonding while ensuring patient comfort is not compromised.

The lingual debonding pliers are used with complementary lingual cutters and pliers Hu-Friedy offers:

PART CODE	DESCRIPTION
678-700	Lingual Distal End Cutter
678-701	Lingual Pin & Lig Cutter
678-702	Lingual Weingart Pliers
678-702-90	Lingual Weingart Pliers (90° bend)
678-703	Lingual Bracket Removing Pliers
678-704	Lingual Hammerhead NiTi Pliers
678-704SER	Lingual Hammerhead NiTi Pliers (with serrations)
678-705	Lingual Mathieu Pliers
678-706	Lingual Ligature Director
678-707	Lingual Ligature Director (U-shaped)
678-708	Lingual Torqueing Key with Picker
678-709	Lingual Debonding Pliers

Hu-Friedy pliers are crafted with Immunity Steel® alloy, a high quality surgical grade stainless steel which combines an optimal blend of carbon and chromium ensuring strength and corrosion resistance.

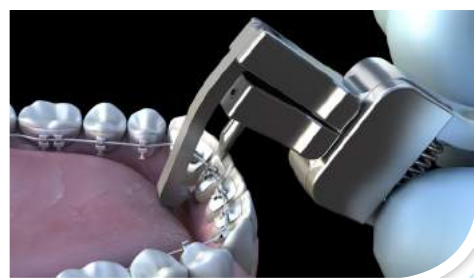

Debonding pliers are placed lingually such that one arm of the pliers reaches below the bracket.



The arm then hooks under the tie wing of the bracket while the other arm wedges under the bracket's adhesive interface. When squeezed, the wedged arm slides down the lingual surface of the tooth, successfully removing the bracket.

View the product video on YouTube by searching "Hu-Friedy Lingual Debonding Pliers 678-709."

Visit us online at Hu-Friedy.com

Call 1-800-Hu-Friedy or contact your authorized Hu-Friedy representative for more information.
©2026 Hu-Friedy Mfg. Co., LLC. All rights reserved. HFO-1017/0217

How the best perform

