

Daily Use Instructions for Your Hu-Friedy® Waterline Filter

NOTES ON USE:

- The Hu-Friedy® Waterline Filter must be left in place during times when the dental unit is not used (weekends, vacations, etc.). Contact HuFriedyGroup for extended closure protocol recommendations.
- Compliance with heat sterilization of handpieces between each patient must be continued following Hu-Friedy Waterline Filter installation.
- Do not shock or use any other Dental Unit Waterline treatment when the Hu-Friedy Waterline Filter is in place. The only solution that should run through an installed Hu-Friedy Waterline Filter is water. If your office protocol includes a periodic shocking requirement, please contact Customer Support at 1-800-Hu-Friedy (1-800-483-7433) or care@Hu-Friedy.com.
- Refer to the Hu-Friedy Waterline Filter complete Instructions for Use available at Hu-Friedy.com for precautionary statements, disposal and troubleshooting information.

When Installed in an Independent Water Bottle

At the start of each workday:

Perform a two-minute flush

Flush all lines that draw from the Hu-Friedy® Waterline Filter water source (including unused lines if they cannot be blocked or disconnected to avoid dead legs that harbor biofilm).

- Ensures that Hu-Friedy Waterline Filter treated water is moving through the lines, maintaining iodine levels necessary for bacteriological control (reduced CFU counts/odor/taste).



Perform hygienic procedures

1. Remove the bottle from the manifold, wash the inside of the bottle (per bottle manufacturer's instructions for use) and rinse thoroughly.
2. Wipe down the outside of the filter with a clean paper towel for sanitary purposes and make sure the intake end is not obstructed.
3. Fill the bottle with fresh water (tap or distilled) and return to manifold.

Throughout each workday:

Perform a 20-30 second flush between patients. Flush every device connected to the dental water system that enters the patient's mouth.

- Required to meet CDC Guidelines for Infection Control in Dental Healthcare Settings.
- Ensures that Hu-Friedy Waterline Filter treated water is moving through the lines, maintaining iodine levels necessary for bacteriological control (reduced CFU counts/odor/taste).

At the end of each workday:

Empty the bottle, and place the empty bottle back on the manifold.

When Installed in a Municipally Plumbed Dental Unit

At the start of each workday:

Perform a two-minute flush

Flush all lines that draw from the Hu-Friedy® Waterline Filter water source (including unused lines if they cannot be blocked or disconnected to avoid dead legs that harbor biofilm).

- Ensures that Hu-Friedy Waterline Filter treated water is moving through the lines, maintaining iodine levels necessary for bacteriological control (reduced CFU counts/odor/taste).

Throughout each workday:

Perform a 20-30 second flush between patients

Flush every device connected to the dental water system that enters the patient's mouth.

- Required to meet CDC Guidelines for Infection Control in Dental Healthcare Settings.
- Ensures that Hu-Friedy Waterline Filter treated water is moving through the lines, maintaining iodine levels necessary for bacteriological control (reduced CFU counts/odor/taste).

At The End of Each Workday:

No additional protocol required



All claims made based on use with potable water.

©2020 Hu-Friedy Mfg. Co., LLC. All rights reserved. All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC, its affiliates or related companies, unless otherwise noted. HLF-451/0620