Anesthetic Aspirating Syringe, 1.8ml & 2.2ml

INSTRUCTIONS

Loading Syringe

1. Retract piston rod fully by pulling back on thumb ring.
2. Keeping piston rod fully retracted, place anesthetic cartridge, plunger end first, into barrel. Plunger faces harpoon.
3. While holding cartridge in place, push piston rod forward with thumb ring to engage rubber plunger with harpoon. Turn thumb ring one quarter turn.
4. Attach needle to aspirating syringe according to needle manufacturer's instructions.
5. Remove needle cap and expel a few drops of anesthetic to ensure proper attachment. Syringe is now ready for use.
6. Aspiration can be achieved by pulling back on thumb ring after needle is positioned.

Removing Anesthetic Cartridge

7. When completed with injection, remove and dispose of needle properly.
8. Remove cartridge by fully retracting piston rod, as in loading, to release aspirating harpoon from rubber plunger.
9. Turn syringe over to release cartridge from barrel.

Additional Anesthesia

10. If additional cartridges are needed, repeat loading steps above, being careful not to bend needle.

This syringe model is designated for the use with cartridges with solid plug. This syringe model is not compatible with hole plug cartridges.

CLEANING AND STERILIZATION PROCEDURES

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Hu-Friedy Mfg. Co., LLC
EC REP Hu-Friedy Europe LLC & Co. KG