



Clear Collection™ Instrument Usage Guide

Clockwise and Counter Clockwise Wedges

The Clockwise and The Counter Clockwise Wedge instruments are used to produce indents into the facial and the lingual plastic for the rotational movement of a specific tooth. From an occlusal view, evaluate the tooth you wish to turn or rotate: Clockwise or Counter Clockwise (i.e. anti-clockwise). Select the appropriate wedge instrument to produce the desired rotation. Position it at the line angle near the embrasure, and indent the labial and the lingual to create a rotational couple for that tooth.

Clockwise Wedge 678-808

Counter Clockwise Wedge 678-807

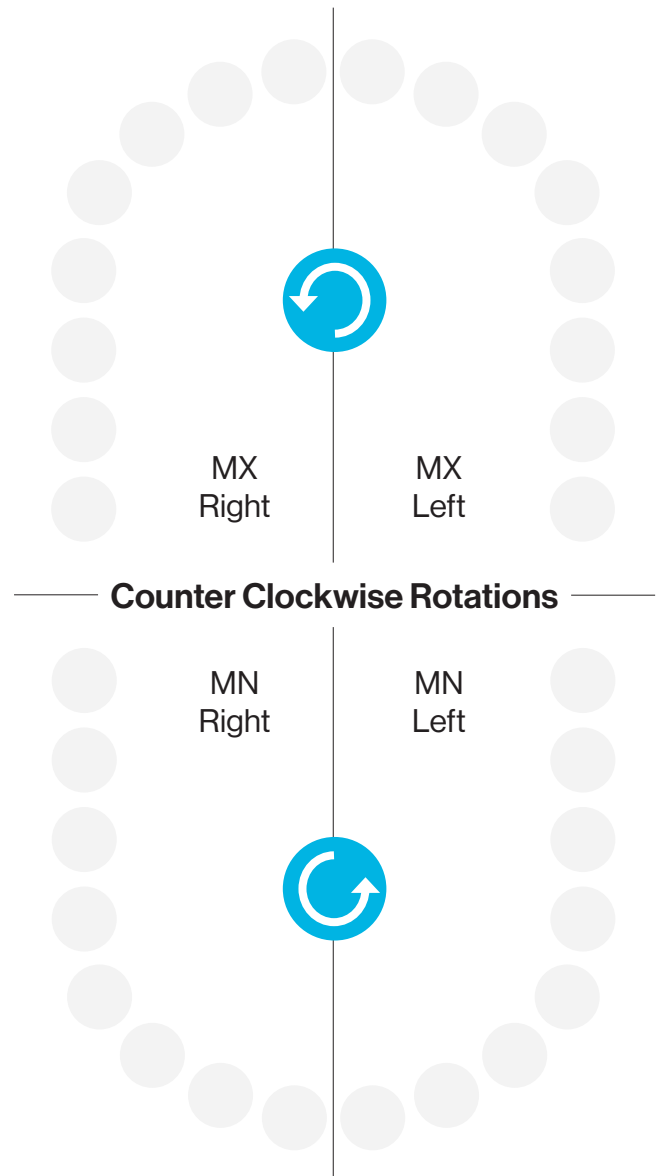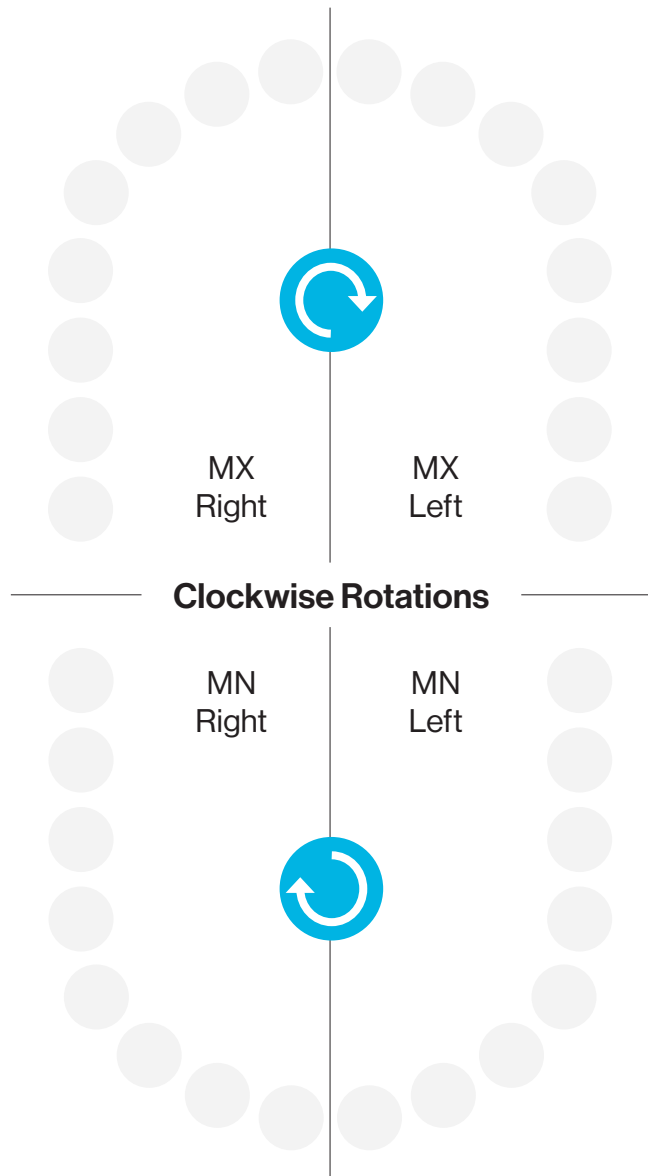

For more information visit Hu-Friedy.com/ClearCollection

All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC its affiliates or related companies, unless otherwise noted.

©2026 Hu-Friedy Mfg. Co., LLC. All rights reserved. HFL-573/0321