

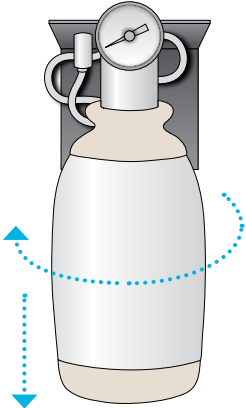
Installing a DentaPure™ DP40B Cartridge onto an A-dec® Unit



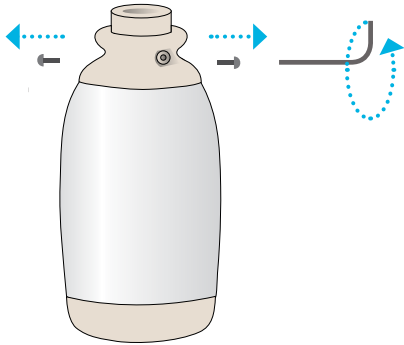
0.7 Liter or 2 Liter

- 1**

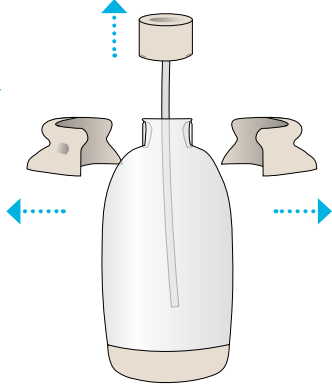
Remove the A-dec® bottle from the manifold


- 2**

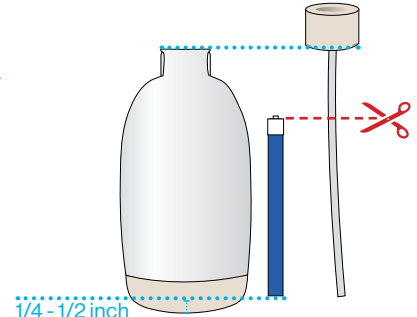
Remove two hex screws with 1/8th inch allen wrench


- 3**

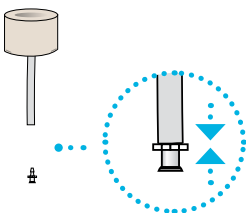
Separate collar and lift pick-up tube assembly from the bottle


- 4**

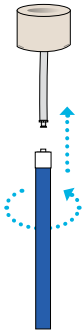
Align the bottle alongside the pick-up tube to ensure that your DP40B cartridge can attach to the pick-up tube with enough space to fit inside the bottle with 1/4 - 1/2 inch of empty space from the bottom


- 5**

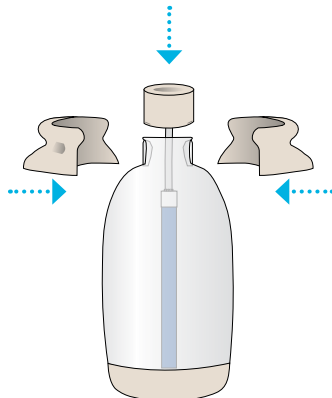
Attach the luer fitting onto the pick-up line


- 6**

Twist the luer end of the cartridge onto the cut end of the pickup tube


- 7**

Place the pick up tube assembly with the cartridge back into the bottle. Reattach the collar to the bottle, replace the hex screws, and replace the A-dec® bottle to the manifold



Learn More at HuFriedyGroup.com/DentaPure



Hu-Friedy Mfg. Co., LLC, 1666 E. Touhy Ave., Des Plaines, IL 60018 | HuFriedyGroup.com
All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC its affiliates or related companies, unless otherwise noted. A-dec® is a registered trademark of A-dec, Inc.
©2026 Hu-Friedy Mfg. Co., LLC. All rights reserved. DLIT01533 Rev A 1022

Crosstex is now a proud member of

HuFriedyGroup
The Best In Practice