

Say Goodbye to Tedious Tablet Protocols

Easy, Hassle-Free Dental Unit
Waterline Compliance

- ✓ **EPA**
standards
- ✓ **CDC**
guidelines
- ✓ **ADA**
recommendations
- ✓ **OSAP**
recommendations

Still using or considering tablets?

With the DentaPure® cartridge, ensuring dental unit waterline safety has never been easier. If your office is still using tablets for DUWL compliance, you're aware of the disadvantages:

- Cost: At first glance tablets appear to be a less expensive option. However, when diligently following tablet instructions for use (IFU) the average tablet treatment cost is approximately \$821/year for just one operator*.
- Tablets only work with strict compliance to daily protocols and IFUs; this includes regular DUWL shocking
- Staff time and expense to manage daily, monthly and quarterly procedures
- Tablets must be added at every water change and given the proper time to dissolve
- Potential risk of staff exposure to daily chemicals
- Undissolved tablet solids may adversely affect expensive handpieces over time
- Service technician calls may increase due to clogged lines

All claims based on use with potable water.

* Includes cost for initial shock, daily tablets, shocking per IFU quarterly waterline testing and labor.

† Or 240L of water, if usage records are kept.



Independent Water Bottle Cartridge DP365B

For independent water bottles. Easily self-installed onto existing pickup tube with included fitting.

1 Simple Installation. 1 Year of Compliant Water†


The multi-award winning DentaPure cartridge is not restricted by the EPA Rule for BMP for Dental Amalgam Waste and protects your dental unit water for an **entire year!**†

How does the DentaPure Cartridge work?

The DentaPure® cartridge contains non-allergenic iodinated resin beads



As water passes through, the resin releases 2 - 6 ppm of atomic isotopes of elemental iodine (I_2) during a typical dental treatment

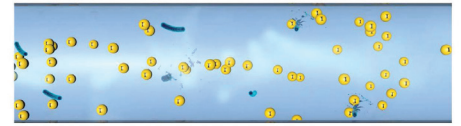

The (I_2) isotopes control the bacteria, keeping dental unit water safe for 365 days[†]



Installation is simple!

The DentaPure DP365B cartridge installs onto your dental unit's existing water bottle pickup tube via the included fitting.

A municipal cartridge (DM365M) for incoming waterlines plumbed directly to city water is also available.

NO tablets / **NO** tablet downsides
NO routine shocking
NO harsh chemicals
NO silver
NO distilled water required
NO special disposal requirements
NO allergenic iodine proteins^{††}



Liquid Ultra® solution DS503L
 1 box of 10 sets. Each set contains one bottle of solutions 1 & 2.

Not yet ready to make the switch from tablets to a cartridge system?

Tablet IFUs require periodic shocking. Liquid Ultra® solution can help!

Liquid Ultra solution is the only EPA registered DUWL treatment granted ALL of the following claims:

- Kills biofilm bacteria
- Removes existing biofilm from Dental Unit Waterlines
- Prevents and suppresses formation of biofilm in Dental Unit Waterlines

Contact your dealer today.

All claims based on use with potable water.

[†] Or 240L of water, if usage records are kept.

^{††} The elemental iodine in the DentaPure cartridge contains no allergenic proteins. People are not allergic to iodine; they are allergic to the protein sometimes attached to it.¹ Iodinated water is completely safe for patients to ingest!

¹ Schabelman E, Witting M. The relationship of radiocontrast, iodine, and seafood allergies: a medical myth exposed, J Emerg Med. 2010 Nov;39(5):701-7. doi: 10.1016/j.jemermed.2009.10.014. Epub 2010 Jan 4

All product names are trademarks of Crosstex International Inc., a Cantel Medical Company, its affiliates or related companies, unless otherwise noted.



Receive 1 CEU

Free waterline treatment course:

Children Hospitalized from Contaminated Dental Unit Water: Could This Happen in Your Office?

Register today at
CrosstexLearning.com