

Cleaning/Shocking Dental Unit Waterlines

When a DentaPure™ Bottle Cartridge is Installed (DP365B or DP40B)

Dental unit waterlines **must not** be cleaned/shocked with a DentaPure™ Cartridge in place. If this happens, use **iodine test strips** (Crosstex item# DPTEST) to determine that the level of iodine is above 0.5 ppm, or replace the cartridge.

1. Remove

- Remove the DentaPure™ Cartridge from the luer fitting by turning it clockwise. The luer fitting should remain attached to the pick-up tube.
- If installing a new DentaPure™ Cartridge, discard the depleted DentaPure™ Cartridge in your regular office trash.
- If continuing use of the removed cartridge, place it in a zip-top bag labeled with the operatory number and initial cartridge installation date (estimated date is acceptable). **DO NOT DISCARD.**



2. Install

- Install the non-functional DentaPure™ Cartridge demo unit onto the luer fitting on the pick-up tube.
- Take note of the "Non-Functional Demo Unit Only. Not for Use in Dental Facilities" etching on the demo unit and the absence of a filter inside the end cap.



3. Clean/shock your DUWLs

- Follow the Instructions for Use (IFU) that came with your cleaning/shocking product.

4. Once the cleaning/shocking process is complete:

- Remove the non-functional demo unit from the luer fitting on the pick-up tube.
- If installing a new DentaPure™ Cartridge, follow the included Instructions for Use.
- If reusing the DentaPure™ Cartridge that was removed prior to cleaning/shocking, remove it from the zip-top bag and re-install on the pick-up tube.
 - Ensure that you are re-installing the functional cartridge by looking for the presence of the filter in the end cap.**
- Operate dental unit instruments until all air is purged from the water system. Resume system use.



If you plan to use the non-functional demo unit in the future:

- Thoroughly rinse the inside and outside with warm water.
- Shake out excess water and allow to dry completely.
- Place the dry non-functional cartridge in a zip-top bag with a copy of the Instructions for Use and label the bag: *Non-functional DentaPure™ Cartridge Demo Unit for cleaning/shocking.*

For more information, contact Customer Service at crosstex@crosstex.com or 888-276-7783

For Technical Support contact dentapuretechnical@crosstex.com

Crosstex International, 6789 W. Henrietta Rd., Rush, NY 14543 | www.crosstex.com
All product names are trademarks of Crosstex International, Inc., its affiliates or related companies, unless otherwise noted. Marks not registered in all jurisdictions.

© 2021 Crosstex International, Inc. DLIT01525 Rev A 0321

