Hu-Friedy

WATERLINE FILTER SHOCK ACCESSORY KIT

Always remove the Hu-Friedy Waterline Filter before cleaning waterlines with any type of antimicrobial agent. To introduce an antimicrobial waterline cleaner using a dental bottle, the riser tubing inside the bottle needs to be reinstalled temporarily. To make this process easier, this accessory kit includes a special fitting and tubing to replace the riser tubing previously removed to accommodate the Hu-Friedy Waterline Filter.

INSTRUCTIONS FOR USE

1. Turn off and depressurize any devices attached to the Hu-Friedy Waterline Filter.
2. Remove the dental bottle from the manifold.
3. Unthread the barb fitting from the top of the filter, leaving the barb in place inside the tubing attached to the manifold. Remove the waterline filter.
4. Thread the female end of the special barb fitting included with this kit onto the male-thread end of the barb left inside the tubing attached to the manifold.
5. Cut the tubing attached to the special barb fitting to a length that allows at least a 1/4" gap between the end of the tubing and the bottom of the bottle.
6. Clean dental waterlines with an approved antimicrobial cleaner according to manufacturer's instructions.
7. After cleaning and rinsing lines, remove the bottle, unthread the special fitting from the male barb fitting left in place inside the tubing attached to the manifold. Keep this special fitting and tubing for future cleanings.
8. Replace or reinstall the Hu-Friedy Waterline Filter according to included instructions.
9. If you plan to re-use this accessory in the future:
 - Thoroughly rinse the inside and outside of the shock accessory with warm water.
 - Shake out excess water and allow the accessory to dry completely.
 - Place dry accessory in the original bag, or a zip-top bag with a copy of the Instructions for Use.

Manufactured for:

Hu-Friedy Mfg. Co., LLC

3232 N. Rockwell Street | Chicago, IL 60618 | USA

Telephone: 1-800-Hu-Friedy or 1-773-975-6100

Website: Hu-Friedy.com

©2020 Hu-Friedy Mfg. Co., LLC. All rights reserved. All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC its affiliates or related companies, unless otherwise noted. N4498/0520

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