

BACK TO PRACTICE

Hu-Friedy Waterline Solutions

Closure and Reopening Protocols



Hu-Friedy Waterline Filter (IMS-1440, IMS-1440CA)

For the Instructions for Use (IFU) and additional product information on the Hu-Friedy Waterline Filter IMS-1440, please [click here](#).

For the IFU and additional product information on the Hu-Friedy Waterline Filter IMS-1440CA, please [click here](#).



PRIOR TO OFFICE CLOSURE

If installed inside an independent dental water bottle:

1. Turn off and depressurize devices attached to the filter.
2. Remove the dental bottle and empty.
3. Wash the inside of the bottle (per bottle manufacturer IFU) and rinse thoroughly.
4. Wipe down the outside of the Hu-Friedy Waterline Filter with a clean, dry paper towel.
5. Replace bottle on the manifold.

If installed inline (such as ahead of a dental device such as a free-standing scaler):

6. Leave the filter installed inline.
7. Follow dental device manufacturer's recommendations for periods of inactivity.

NOTE: If the filter is set to expire during the shutdown period, order a new filter so it can be installed before the office re-opens.

PRIOR TO RE-OPENING OFFICE

Consult delivery system manufacturer recommendations for periods of inactivity.

If installed inside an independent dental water bottle:

1. Remove Hu-Friedy Waterline Filter.

NOTE: The Hu-Friedy Waterline Filter should ALWAYS be removed if cleaning/shocking the lines. If cleaner/shock treatment is running through the filter, it is recommended to replace with a new filter.

2. Replace the section of tubing temporarily (Consult a service technician, if needed).
3. Thoroughly clean dental unit waterlines following the manufacturer's instructions or with an appropriate cleaning agent.
4. Rinse the lines.
5. Remove the temporary tubing.
6. Replace or reinstall the filter.
7. Fill with fresh water each morning before its first use. Replace Hu-Friedy Waterline Filter after 240 liters of water usage, if iodine level falls to less than 0.5ppm, or if records of usage are not kept, after 365 calendar days of installation.

If installed inline (such as ahead of a dental device such as a free-standing scaler):

1. Follow dental device manufacturer's recommendations for periods of inactivity.

For more information call 1-800-Hu-Friedy, contact your authorized Hu-Friedy representative, or visit [Hu-Friedy.com](https://www.hu-friedy.com)

BACK TO PRACTICE

Hu-Friedy Waterline Solutions

Closure and Reopening Protocols



VistaTab™ Dental Waterline Cleaner Tablets (IMS-1451)

For the Instructions for Use (IFU) and additional product information on the VistaTab™ Dental Waterline Cleaner tablets IMS-1451, please [click here](#).



PRIOR TO RE-OPENING OFFICE

Shock the dental unit waterlines

1. Remove handpieces and purge all waterlines with air to remove water.
2. Remove bottle and clean it thoroughly with hot water, a cleanser recommended by your bottle manufacturer and bottle brush.
3. Add two (2) 1.7-gram VistaTab Dental Waterline Cleaner tablets for every 750 ml of potable water. Allow the tablets to dissolve.
4. Flow about half of the treated water through the dental waterlines and let the treated water sit within the lines for 5 minutes.
5. Purge the remaining water through the lines.
6. Repeat steps 3, 4 and 5.
7. Flush at least 500 ml of potable water through the lines to remove any residual chemical.

NOTE: Do not shock through an installed Hu-Friedy Waterline Filter.

Refer to the VistaTab Dental Waterline Cleaner Tablet Instructions for Use if use of these tablets as a weekly DUWL routine treatment is desired.

For more information call 1-800-Hu-Friedy, contact your authorized Hu-Friedy representative, or visit [Hu-Friedy.com](https://www.hu-friedy.com)

©2026 Hu-Friedy Mfg. Co., LLC. All rights reserved. All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC its affiliates or related companies, unless otherwise noted. HFL-409/0620