


### Filter Life & Replacement Thresholds

- 1 Prefilter (CVR3251)**  
Approximately one year (**change at least annually**).
- 2 Deionization Filter (CVR3252)**  
**4 months to 3 years**, depending on volume of usage and quality of local water supply. Change BOTH deionization filters whenever the system-mounted TDS meter "OUT" displays a number equal to or greater than 4 parts per million (ppm).
- 3 Deionization Filter (CVR3253)**  
**4 months to 3 years**. Replace at the same time as CVR3252.
- 4 Hyperfiltration Element (CVR3254)**  
**3-5 years**, depending on the volume of usage and quality of local water supply. Replace the hyperfiltration filter whenever the system-mounted TDS meter "IN" exceeds 10% of the TDS of the municipal water supply TO the system, as measured with the included, handheld meter.
- 5 Polishing Filter (CVR3255)**  
Approximately one year (**change at least annually**).

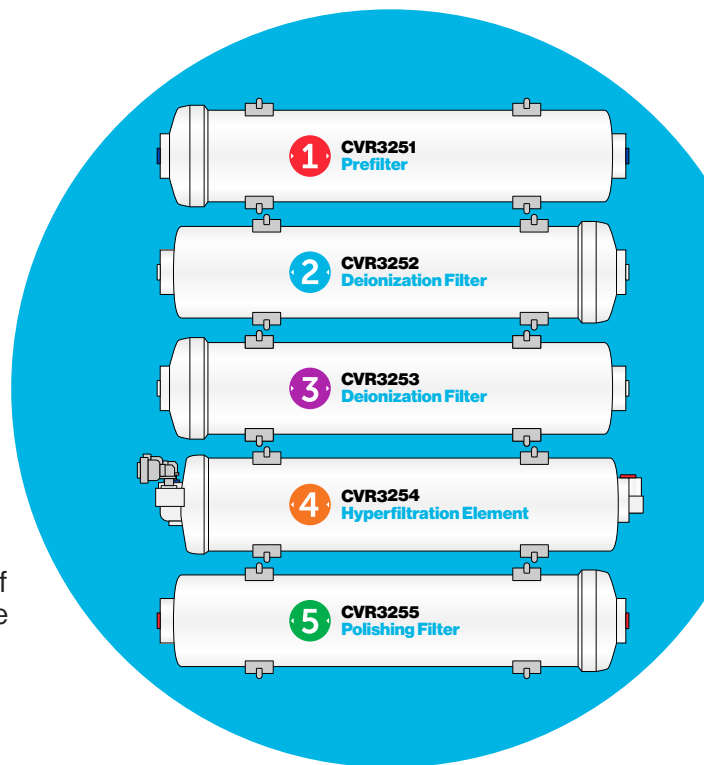

### Daily

- Turn off the system at the end of each work day by closing the valves on the system board, water storage tank, and autoclave wand.

### Monthly

- Check water quality with the included TDS meters, and record the data in the chart in the Owner's Manual.

### Annually

- Replace the Prefilter (CVR3251) and Polishing Filter (CVR3255). Replace other filters only as needed, as indicated by TDS test.
- Check the air pressure in the water storage tank. Air pressure should be between 6.0 psi (41 kPa) and 7.5 psi (52 kPa). See instructions in the Owner's Manual or online.
- Sanitize the water storage tank and faucet using non-scented chlorine bleach and included syringe. See instructions in the Owner's Manual or online.

### Need Help?

Visit our searchable support database online at [vrg.support](http://vrg.support). Or call us at **(800)531-2221**.

Please post this sheet where you will see it regularly, such as the inside of a cabinet door near your VistaPure™ System.

**CROSSTEX**



Manufactured for: Crosstex International, Inc.  
6789 W. Henrietta Road, Rush, NY 14543 USA  
[HuFriedyGroup.com](http://HuFriedyGroup.com)

Made in USA

VM2536 / REV C - 11/2023

All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC, its affiliates or related companies, unless otherwise noted.

©2023 Hu-Friedy Mfg. Co., LLC. All rights reserved.