

# INSTRUCTIONS FOR USE

Sterilization



## ConForm® Mail-in Sterilizer Monitoring Service

### Instructions for use:

1. Remove the test strip from the pocket marked "test strip".  
DO NOT remove test strip from the glassine envelope, as this will invalidate the test. DO NOT remove or process the control strip.
2. Place the test strip in the sterilizer per the manufacturer's instructions (i.e., in the lowest tray near the door, or in the center of the chamber where the load is most densely packaged.) Run a normal sterilization cycle.
3. Upon completion of the sterilization cycle, return the test strip to the pocket marked "test strip." Seal the flap closed.
4. Complete all required information requested on the envelope.  
We cannot process envelopes that do not have identifying information (i.e. customer # or phone #), or are left blank. Optional information will be recorded if it is included.
5. Mail the envelope.
6. Test results will be faxed or e-mailed to your office quarterly. Test failures are reported immediately by fax or email, with a follow-up phone call to verify receipt of the report.



Ref.#'s CST060, CST120, CST480,  
CVT120, CVT520

ConForm® is a registered trademark of Crosstex International, Inc.



**CROSSTEX**  
A CANTEL MEDICAL COMPANY

Manufactured by: Crosstex International, Inc.

10 Ranick Road | Hauppauge, NY 11788 | 631.582.6777 | 888.276.7783 | [Crosstex@crosstex.com](mailto:Crosstex@crosstex.com)

Made in USA

CT-IFU  
Rev. 0 - 3/17