

INSTRUCTIONS FOR USE

Sterilization



Self-Seal Nylon Sterilization Pouches

Instructions for use:

1. Choose the appropriate pouch size for the items to be sterilized.
2. Carefully place instruments into the pouch and remove any excess air.
3. Utilize internal and external dry heat chemical indicators in accordance with facility guidelines and local standards. External dry heat indicators are supplied with this product for your convenience. When used apply one indicator to the outside of the pouch, avoiding placement at the seal. Ensure indicator is facing up during sterilization. Indicators color transition from green to dark when exposed to dry heat sterilization.
4. Remove the paper from the adhesive strip, fold evenly along perforation and secure firmly. Ensure the seal is flat and free of wrinkles or creases.
5. Ensure the pouch contents are a minimum of 1" (2.5 cm) away from all sides of the pouch.
6. Follow directions for sterilization according to the manufacturer of the sterilizer.
7. Allow pouches to cool in the sterilizer 5-10 minutes after cycle completion.



Ref.#'s NSP-400, NSP-405,
NSP-410, NSP-420, NSP-430



Do not reuse.



Not made with natural rubber latex.



Crosstex International, Inc.
10 Ranick Road, Hauppauge, NY 11788-4209 USA
www.crosstex.com 800.276.7783 631.582.6777

Made in USA
NSP-IFU / REV A - 05/2020

All product names are trademarks of Crosstex International, Inc., a Cantel Medical Company, its affiliates or related companies, unless otherwise noted. Marks not registered in all jurisdictions.

CROSSTEX
A CANTEL MEDICAL COMPANY