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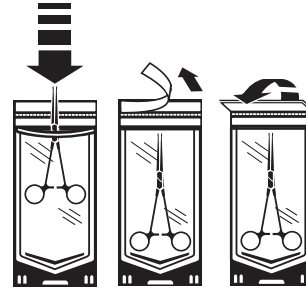
INSTRUCTIONS FOR USE:

Sure-Check™ Self-Sealing Sterilization Pouches

With internal/external multi-variable (parameter) steam indicator

CROSSTEX

Crosstex International, Inc.
6789 W. Henrietta Road, Rush, NY 14543 USA
585.359.0130 | 631.582.6777 | hufriedygroup.com

EU REP MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany

CH REP MDSS CH GmbH
Laurenzenvorstadt 61
5000 Aarau
Switzerland

UK REP MDSS-UK RP Ltd.
Parkway House, Palatine Rd,
Northenden, Wythenshawe,
Manchester M22 4DB
United Kingdom

Made in USA
SC-INS / REV 6 - 12/2025

All product names are trademarks of Hu-Friedy Mfg. Co., LLC,
its affiliates or related companies, unless otherwise noted.
© 2026 Hu-Friedy Mfg. Co., LLC. All rights reserved.

Not made with natural rubber latex.
Meets ISO 11140-1 Type 4 and 11607.

HuFriedyGroup
The Best In Practice

- STORAGE** - Store all sterilization pouches in a cool, dry place.
- LOADING*** - Carefully place items into pouches and remove excess air before sealing. Where possible, use a pouch rack to stand pouches on edge in the sterilizer to facilitate sterilization and drying. When loading multiple pouches, be sure that the plastic side faces the paper side of each adjacent pouch.
- STERILIZATION** - Steam chemical indicator ink printed on the inside of each Sure-Check Sterilization Pouch is certified as a multi-variable (parameter) indicator. Internal multi-variable (parameter) chemical indicators are designed to react to the 3 variables required for sterilization - temperature, specified time, presence of steam - and provide a more reliable indication that the conditions for sterilization have been met. Pink indicators will turn brown when processed by steam. Blue indicators will turn gold/brown when EtO processed. Multi-variable (parameter) indicators meet the 2008 CDC† Guidelines for sterilization. Sure-Check Sterilization Pouches are only available for use in steam sterilizers (autoclaves) and EtO. This procedure, when used in conjunction with routine biological monitoring (spore testing), provides you with the highest level of sterility assurance.
- SHELF LIFE** - When properly sealed, sterility is maintained up to three (3) years provided the integrity of the pouch has not been compromised, i.e. pouch becomes wet, damaged or is opened during storage.

Caution - wet pouches may not preserve sterility of contents.

SINGLE-USE POUCHES: Sure-Check Sterilization Pouches are single use items. Do not re-use.

* REFERENCE - ANSI/AAMI ST79 Comprehensive Guide to Steam Sterilization and Assurance in Health Care Facilities.

† CDC Guideline for Disinfection and Sterilization in Healthcare Facilities, 2008.

Sure-Check Sterilization Pouches are not considered hazardous waste and can be disposed of in regular trash (40 CFR 261.24 tested using the TCLP, EPA Publication SW-846).

In case of any serious incident that has occurred and involves our devices within the European Union, please report it to Hu-Friedy Mfg. Co., LLC, and your national competent authority of the member state.

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