


Safe-Flo™ SE Valve

REF M1010SUSE

INSTRUCTIONS FOR USE

1. Insert the end of the saliva ejector into the larger end of the SE valve.

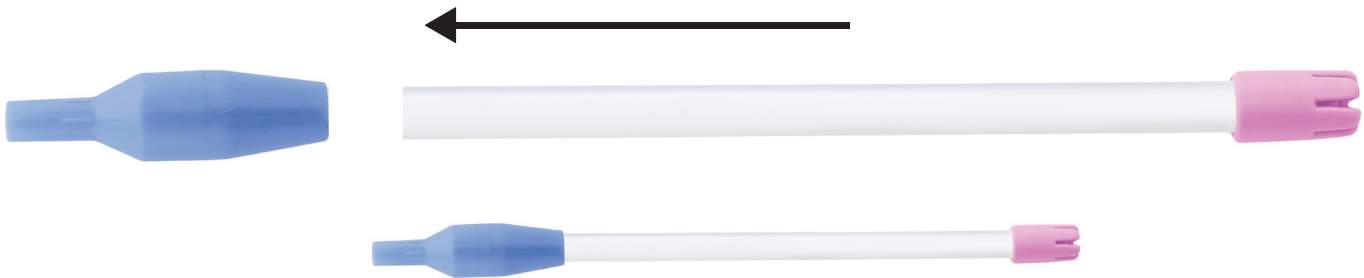

2. Attach the valve/saliva ejector combination onto the open end of the low volume evacuation suction line.
3. Bend the saliva ejector tube into desired position and place into the mouth.
4. After single patient use, dispose of the valve/ejector combination in the standard trash.

CROSSTEX



Crosstex International, Inc.
6789 W. Henrietta Road, Rush, NY 14543 USA
585.359.0130 | 631.582.6777 | crosstex.com



MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany



GH REP STERIS GmbH c/o BDO AG,
Längfeldweg 116A,
CH-2504 Biel-Bienne,
SWITZERLAND



For professional use only.
Not made with natural rubber latex.

Made in USA

M1010SUSE-IFU-EN / REV D - 03/2022

In case of any serious incident that has occurred and involves our devices within the European Union, please report it to Hu-Friedy Mfg. Co., LLC and your national competent authority of the member state.

All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC, its affiliates or related companies, unless otherwise noted.

US Pat. No. 8,256,464, 8,714,200, 8,752,579 and foreign counterparts.

©2026 Hu-Friedy Mfg. Co., LLC. All rights reserved.