

Sparkle™ Air/Water Syringe Tips

REF BCSAWSCL / BCSAWSWH / BCSAWS / BCSAWSB

INSTRUCTIONS FOR USE

No Adapter or converter needed with original equipment

1. On most nut style syringes, loosen the syringe tip locking nut and insert the Sparkle™ Air/Water Syringe Tip the same as you would a metal tip.
2. On the quick-change syringe, press in the outside ring and insert the Sparkle Air/Water Syringe Tip the same as you would a metal tip.
3. Before using the syringe, make sure the Sparkle Air/Water Syringe Tip is firmly seated in the syringe. The Sparkle Air/Water Syringe Tip is properly installed when no lines or grooves are visible beyond the syringe tip locking nut.



Sparkle Air/Water Syringe Tips are packaged and sealed in a clean environment (not sterile).

CROSSTEX



Manufactured for: Crosstex International, Inc.
6789 W. Henrietta Road, Rush, NY 14543 USA
585.359.0130 | 631.582.6777 | HuFriedyGroup.com

EU | REF | MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany

CH | REF | MDSS CH GmbH
Laurenzenvorstadt 61
5000 Aarau
Switzerland

UK | REF | MDSS-UK RP Ltd.
Parkway House, Palatine Rd,
Northenden, Wythenshawe,
Manchester M22 4DB
United Kingdom



Not made with natural rubber latex.

In case of any serious incident that has occurred and involves our devices within the European Union, please report it to Hu-Friedy Mfg. Co., LLC and your national competent authority of the member state.

Made in China

BCSAWSB-IFU-EN / REV 3 - 10/2025

All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC, its affiliates or related companies, unless otherwise noted.

©2025 Hu-Friedy Mfg. Co., LLC. All rights reserved.