HuFriedyGroup Pedo & Permanent Crowns
Stainless Steel Primary & Permanent Molar Crowns

EN

Brand Name: Pedo Crowns / Permanent Crowns.

Intended Purpose: A preformed crown is a prefabricated device made of austenitic alloys intended to be affixed temporarily to a tooth after removal of, or breakage of, the natural crown (that portion of the tooth that normally protrudes above the gums).

Indication: Restoration for a badly decayed tooth.

Target Population: Dental patient who needs restoration for a badly decayed tooth.

Intended User: Dentist.

Instructions for Use

HuFriedyGroup Stainless Steel Crowns are intended to provide full coverage, stronger, provisional treatment for primary molars and fill the need for long term provisional coverage on permanent molars. To help minimize contouring and trimming, these stainless steel primary and permanent molar crowns are precrimped, prebeveled and prefestedoned.


Crown Selection and Preparation

1. Select the proper crown size. Measure the mesio-distal dimension of the tooth in the patient's mouth. Next, measure the crowns in the kit to determine the corresponding size.
2. Once the appropriate size has been identified, test for accurate fit in the mouth. Note the occlusal relationship between the opposing teeth and the relationship to the adjacent dentition.
3. Using an explorer, probe for any open margins between the preparation and the crown. If necessary, reshape the crown to achieve a better marginal fit.
4. If there is excessive blanching of the gingival tissue when the crown is seated, the crown will need to be trimmed. Using an indelible pen or wax pencil, mark the crown following the contour of the gingiva. Use crown scissors to trim 1 mm below the mark. Try to cut in a smooth motion to avoid producing sharp metal burs that might irritate the patient's tissue. If sharp areas are produced, use a heatless stone to remove the burs and polish with a rubber wheel. Use contouring pliers to reshape and crimp the crown at the trimmed margins. Complete another test fit to ensure accuracy, proper contacts and occlusion.
5. Complete another try-in fit to ensure accuracy, proper contact and check the occlusion.
6. When checking the occlusion, carefully notice any differences in the maxillary/mandibular relationship. If the crown is fully seated and the crown prevents proper occlusion, further reduction of the preparation may be necessary. Slightly reduce the tooth further and re-try the crown once again to confirm proper occlusion.

Seating the Crown

1. Isolate the prepared tooth by placing rubber dam or by using cotton rolls.
2. Wash and thoroughly dry the preparation.
3. Mix the cement and fill the selected crown, following cement manufacturer's instructions.
4. Seat the crown. Recommendation: for mandibular crowns start at the lingual surface, pressing across to the buccal surface; for maxillary crowns start at the buccal surface, pressing across to the lingual. This will assist in the residual cement being squeezed out. Do not use excessive force during final seating.
5. During the initial seating of the crown, verify proper alignment by checking the relationship with adjacent teeth and opposing teeth. When proper alignment is confirmed, instruct the patient to bite on gauze or an occlusal bite stick; this will ensure the crown is fully seated into the proper occlusion. If using a device to assist in crown seating during cementation, please use care to obtain proper occlusion. Do not use excessive force in order to avoid seating the crown below the level of proper occlusion.
6. Following individual cement manufacturer's instructions, remove the excess cement at the allocated time. Cement removal can be done with an explorer, scaler, or excavator. Use dental floss to remove cement from the interproximal surfaces and subgingival areas.

HuFriedyGroup Stainless Steel Crowns are made with a stainless steel alloy that contains elemental nickel and chromium. A small percentage of the population is known

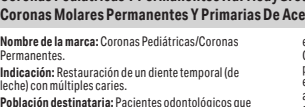to be allergic to these metals. If an allergic reaction occurs, direct the patient to consult a physician.

Cleaning Procedure

1. After washing the product with tap water at room temperature for 30 seconds, remove foreign substances with a soft brush.
2. After diluting the enzyme detergent in tap water at 40°C-45°C at a ratio of 1:100, put it in an ultrasonic cleaner so that the product is completely immersed, and then ultrasonically clean for 10 minutes.
3. Put distilled water in an ultrasonic cleaner, completely immerse the product, and rinse at a temperature of 40°C-45°C for 3 minutes. (Primary)
4. Put distilled water in an ultrasonic cleaner, completely immerse the product, and rinse at a temperature of 40°C-45°C for 3 minutes. (Secondary)
5. After wiping with a clean lint-free cloth, dry for 10 minutes at a dryer temperature of 60°C.

Side Effects

No side effects have been discovered or identified yet.

This product is intended for use only by dental professionals.

This product is single use (disposable). When disposing of the used product, follow the prescribed waste disposal procedures. Reuse is prohibited. Potential risks if the product was reused include:

- Introduction of microbes into the body due to reuse of item which may have been contaminated during its first use, which can result in subsequent health deterioration of patient
- Soft tissue injury from any rough edges if the crown was previously trimmed

The user and/or patient that any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient.

Keep product at room temperature and out of direct sunlight.

To reorder **HuFriedyGroup Pedo and Permanent Crowns**, call 1-800-Hu-Friedy.

Descriptions of symbols on the packaging box is required. Please find the symbols below:

	Medical device
	Manufacturer
	Authorized representative in the European Community
	Date of manufacture
	Country of manufacture
	Model number
	Lot Number
	Distributor
	Do not re-use

Hu-Friedy

Distributed by: Hu-Friedy Mfg. Co., LLC

2501 Sanders Rd. Northbrook, IL 60062 | USA

HuFriedyGroup.com | 1-800-Hu-Friedy

In case of any serious incident that has occurred and involves our devices within the European Union, please report it to Hu-Friedy Mfg. Co., LLC and your national competent authority of the member state.

All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC, its affiliates or related companies, unless otherwise noted.

©2026 Hu-Friedy Mfg. Co., LLC. All rights reserved.

HuFriedyGroup
The Best In Practice

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©2026 Hu-Friedy Mfg. Co., LLC. Tutti i diritti sono riservati.