

Flavored/Un-Flavored Dental Dams

EN


[REF] 19202 / 19200 / 19201 / 19402 / 19400 / 19401 / 19102 / 19100
19101 / 19302 / 19300 / 19301 / 19500

INSTRUCTIONS FOR USE

INSTRUCTIONS FOR USE:

Used to isolate and protect teeth and mouth during a dental procedure. Both latex and non-latex are available.

1. Examine mouth for which teeth need to be free from the dental dam.
2. Mark spaces on the dental dam where those teeth will be.
3. Use dental puncher to punch out those areas.
4. Fit the dental dam onto frame.
5. Fit clamp around tooth to keep the dam from moving out of place.
6. Place dam around teeth and in mouth correctly.

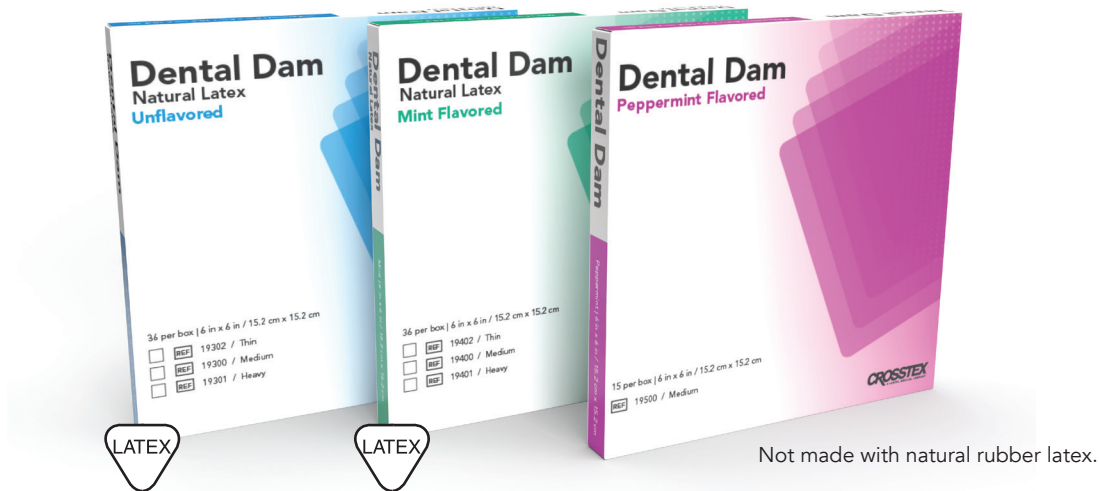

Manufactured for: Crosstex International, Inc.
6789 W. Henrietta Road, Rush, NY 14543 USA
585.359.0130 | 631.582.6777 | crosstex.com



MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany



Made in Malaysia
DDAM-IFU-EN / REV B - 09/2022

All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC, its affiliates or related companies, unless otherwise noted.
©2022 Hu-Friedy Mfg. Co., LLC. All rights reserved.

CROSSTEX