

# Sani Vac™ Evacuation System Cleaner

REF JVAC

## INSTRUCTIONS FOR USE

EN

### INSTRUCTIONS FOR USE:

1. Add 1 oz. (30mL) of SANI Vac to 100 oz. (3L) of warm water. Mix well.
2. Draw a minimum of 33 oz. (1L) of the prepared solution through the evacuation system tube of each line. The prepared solution services up to 3 lines.
3. Repeat procedure after each patient or at the end of each day to prevent a build-up of organic debris and odors and to ensure optimal performance of the high volume suction.



**Storage and Disposal:** Store in a cool, dry place away from direct sunlight.

Dispose in accordance with local regulations.



WARNING

### HAZARD STATEMENTS:

Causes eye irritation. May cause mild skin irritation.

**Precautionary Statement(s):** Wash hands thoroughly after handling.

If skin irritation occurs, get medical attention. **IF IN EYES:** Rinse continuously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists, get medical attention.

**IF SWALLOWED:** May cause nausea and vomiting. May be irritating to mouth and throat. **IF INHALED:** Can cause allergic type reaction in susceptible individuals upon repeated exposure.

Refer to Safety Data Sheet (SDS) for additional information.

For 24 hour chemical emergency assistance call INFOTRAC:

1-800-535-5053 or 1-352-323-3500.



Manufactured for: Crosstex International, Inc.  
6789 W. Henrietta Road, Rush, NY 14543 USA  
585.359.0130 | 631.582.6777 | crosstex.com

Made in Canada

JVAC-IFU-EN / REV B - 09/2022

All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC, its affiliates or related companies, unless otherwise noted.  
©2022 Hu-Friedy Mfg. Co., LLC. All rights reserved.

**CROSSTEX**