INSTRUCTIONS FOR USE

Advanced, Easy to Use and Cost Effective Clinical Water Treatment Technology
For use as a Microbiological Dental Unit Water Purification System

Keep Out Of Reach Of Children

CAUTION

PLEASE SAVE THESE INSTRUCTIONS FOR FUTURE USE. Only selected information is contained on the cartridge packaging, therefore you should keep this sheet for future reference.

Active Ingredient: Iodine*	46%
Other Ingredients	54%

Unit	DP365VC-CA
Iodinated Resin	240.0 cc

*Bonded to strong base anion exchange resin TOTAL 100%

IN THE REMOTE EVENT THE CYLINDER OR CARTRIDGE IS BROKEN AND THE IODINATED RESIN PELLETS ARE EXPOSED, USE EXTREME CAUTION.

FIRST AID:

IF IN THE EYES: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye. Call poison control center or doctor for treatment advice.

IF ON SKIN OR CLOTHING: Take off contaminated clothing. Rinse skin immediately with plenty of water for 15-20 minutes.

IF SWALLOWED: Call poison control center or doctor immediately for treatment advice. Have person sip a glass of water if able to swallow. Do not induce vomiting unless told to do so by the poison control center or doctor. Do not give anything by mouth to an unconscious person. Have product container or label with you when calling a poison control center or doctor, or going for treatment.

PRECAUTIONARY STATEMENTS:

HAZARDS TO HUMANS AND DOMESTIC ANIMALS:

CAUTION: Avoid direct contact with eyes, skin and clothing. Wear protective eyewear (goggles) and gloves when handling. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco or using the toilet. Remove and wash contaminated clothing before reuse. Safety and effectiveness in children under the age of two has not been established.

ENVIRONMENTAL HAZARDS: The exposed resin pellets are toxic to fish and aquatic organisms. Do not discharge effluent containing this product into lakes, streams, ponds, estuaries, oceans or other waters. Check with Canadian provincial and federal regulations that may apply.

NOTE: USED, UNDAMAGED CARTRIDGES AND CYLINDERS ARE NOT CONSIDERED HAZARDOUS AND MAY BE WRAPPED IN NEWSPAPER AND DISPOSED OF IN THE REGULAR TRASH.

DIRECTIONS FOR USE: To obtain maximum bacteriological reduction, dental unit waterlines are to be cleaned with an approved dental unit waterline agent prior to initial installation of DentaPure™ Cartridges.

GENERAL PRECAUTIONS

USES: DentaPure Cartridge is for use on dental unit waterlines attached to the dynamic dental instruments. Used according to labeling, this cartridge reduces bacteria from the water supplied through the instruments to a level that will meet or exceed the current ADA recommendations for water quality having a maximum of 200 cfu/ml.

IMPORTANT Use DentaPure Cartridge on dental unit waterlines only, not for general consumption. While the unit will perform above these limits, it is recommended that the dental unit water have a pH level of less than 8 and not have a chlorine level greater than 1 ppm. Do not use heated water. Heated water will promote the growth of bacteria and the buildup of biological film in the water line leading to the DentaPure unit. Water pressure must not exceed 45 psi.

Once the cartridge is in place, the supply water will pass through the cartridge’s iodinated resin bed, purifying the water delivered by the dental instrument. To maintain a continuous iodine presence within the dental tubing, install a new cartridge immediately after removing the used cartridge. The cartridge must be left in place during times that the dental unit is not used (weekends, vacations, etc).

Adherence to CDC/ADA/CDA guidelines for aseptic procedures including a 2 minute morning flush, a 20-30 second flush between patients, and hand piece sterilization must be continued after the installation of the cartridge.

STORAGE: Store at room temperature in original packaging, away from children, pets and direct sunlight. Avoid excessive heat. Do not allow cartridge to freeze. Do not store in open or unlabeled containers.

TROUBLESHOOTING

Slow or no water flows through cartridge—make sure new cartridge was installed properly.

Air leak from fitting, contact your dental dealer for replacement parts.

Change the cartridge if the level of iodine output falls to less than 0.5 ppm. Use Iodine sensitive test strips to verify performance. See the chart below for suggested change intervals. To change cartridge, follow installation instructions. Remove the used cartridge and install the new DentaPure™ Cartridge. Operate the dental instrument to check flow and verify that the unit is ready for use. Install a new water check-off sheet. Dispose of the old cartridge according to disposal instructions. Shut off water to dental unit when not in use.

DISPOSAL: Securely wrap cartridge in newspaper and discard in trash.

Discoloration of the tubing from blue to green or black is normal and is not a concern.

Initial dark discoloration of water is caused by release of activated carbon. Operate water until flow appears clear.

Suggested Use Table:

DENTAPURE Unit Size	If Water Usage Records Are Kept	If Water Usage Records are NOT Kept	If Iodine Content is Monitored
DP365VC-CA	2,400 Liters	365 Calendar Days	Change when iodine output falls to less than 0.5 ppm

QUESTIONS OR COMMENTS? Email: care@hu-friedy.com
or call 800.483.7433.

 Manufactured for: Crosstex International, Inc.
6789 W. Henrietta Road, Rush, NY 14543 USA
800.483.7433

Made in USA
VM1074-CA / REV A - 02/2023
All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC,
its affiliates or related companies, unless otherwise noted.
©2023 Hu-Friedy Mfg. Co., LLC. All rights reserved.

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Fabriqué pour : Crosstex International, Inc.
Hu-Friedy Mfg. Co., LLC, ses filiales ou sociétés apparentées, sauf mention contraire.
©2023 Hu-Friedy Mfg. Co., LLC. Tous droits réservés.