

VistaTab™ Dental Waterline Cleaner Tablets

Directions for Use




For use with Independent Water Bottle Delivery Systems

VistaTab™ Dental Waterline Cleaner Tablets are antimicrobial tablets used for the cleaning and control of microbial contaminants in dental unit waterlines.

Get back to practice faster! The entire tablet use protocol can be completed in 15 minutes or less per chair on any day that the office is open.

Using VistaTab™ Dental Waterline Cleaner Tablets is easy too. Just follow the steps below.

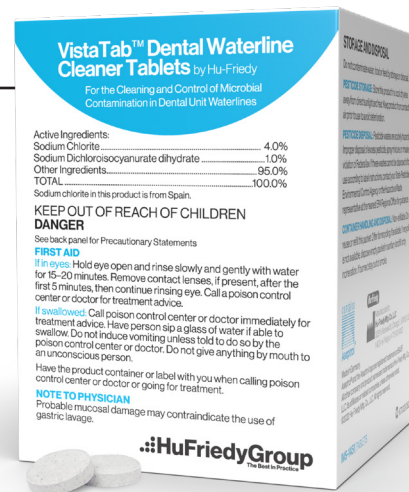

Initial Treatment Protocol

- **STEP 1:**
Fill a 750mL water bottle with potable water and add two VistaTab™ Dental Waterline Cleaner Tablets. Allow the tablets to dissolve. Water will turn blue.
- **STEP 2:**
Flow about half of the treated water through all waterlines and let sit for 5 minutes. Purge the remaining water through all lines connected to water on the dental unit.
- STEP 3:**
Repeat steps 1 and 2.
- **STEP 4:**
Rinse and refill the water bottle with potable water and flush at least 500mL of water through all lines connected to water on the dental unit to remove any residual chemical.
- STEP 5:**
The unit is now ready for patient treatment.

Return to your regular waterline maintenance and testing protocol. If you plan to use VistaTab™ Dental Waterline Maintenance Tablets for routine dental waterline maintenance or shocking following dental waterline testing failure, refer to the Routine Treatment Protocol instructions.

Routine Treatment Protocol

- **STEP 1:**
Every week, or as needed, fill a 750mL water bottle with potable water and add one VistaTab™ Dental Waterline Cleaner Tablet. Allow the tablet to dissolve. The water will turn blue.
 - **STEP 2:**
Flow approximately half of the treated water through all waterlines and let sit for 5 minutes. Purge the remaining water through all lines connected to water on the dental unit.
 - **STEP 3:**
Rinse and refill the water bottle with potable water and flush at least 500mL of water through all lines connected to water on the dental unit to remove any residual chemical.
 - STEP 4:**
The unit is now ready for patient treatment
- Return to your regular waterline maintenance and testing protocol. VistaTab™ Dental Waterline Maintenance Tablet Routine Treatment can be performed weekly, or as needed per your office protocol.

Refer to VistaTab™ Dental Waterline Cleaner Tablets Instructions for Use for Precautionary Statements, Directions for Use and Troubleshooting available at [HuFriedyGroup.com](https://www.hufriedygroup.com)

For more information, contact HuFriedyGroup Customer Care at 1-800-Hu-Friedy or email watersupport@hu-friedy.com

VistaTab™ Dental Waterline Cleaner Tablets EPA Reg. No. 70060-25-75372.

Hu-Friedy Mfg. Co., LLC, 1666 E. Touhy Ave., Des Plaines, IL 60018 | [HuFriedyGroup.com](https://www.hufriedygroup.com)

All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC, their affiliates or related companies, unless otherwise noted. Marks not registered in all jurisdictions.

©2026 Hu-Friedy Mfg. Co., LLC. All rights reserved. HFL-866 / 1023

