



ProLuxTM

Luxating Elevators


 **HuFriedyGroup**
The Best In Practice

Luxate and Elevate

.....

ProLux™ Luxating Elevators are manufactured with thin, sharp working ends which allow for atraumatic extractions—they help to preserve the socket while gently luxating the tooth. This atraumatic approach can lead to increased patient comfort and enhanced healing, providing optimal clinical results.

Working end designs allow for **quick definition of the purchase point**

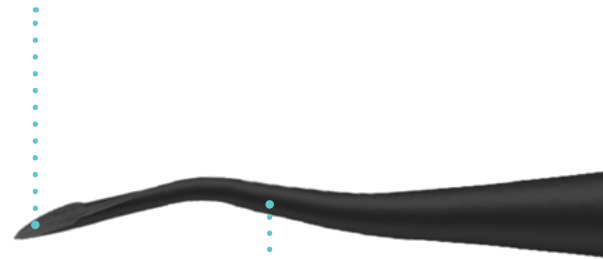

Tapered, single-piece construction directs tip apically



Black Line Instrument Collection

- Performance engineered coating for a harder, smoother surface for **optimal edge retention and enhanced lubricity**
- Distinct black finish for **enriched contrast and visual acuity at the surgical site and underlying tissue**
- **Minimal light reflection** afforded by a sleek black instrument design
- Handcrafted with Immunity Steel® for **optimal strength and corrosion resistance**



HuFriedyGroup

Ergonomically designed handle
provides **enhanced comfort
and control**

HuFriedyGroup

PROLUX™ LUXATING ELEVATORS



ProLux™ Spade, Yellow

Black Line: **PROLUX1X**


Satin Steel: **PROLUX1**

The fine point of the ProLux™ Luxating Elevator Spade facilitates sulcular retraction and aids in positive engagement into the PDL space. The thin profile and sharpened edges incise the ligament quickly and expand the alveolus laterally.

In cases where there is a lack of coronal structure, or the root has fractured subgingivally, the sharpened tip of the spade can still be easily advanced along the side of the root. This ProLux™ Luxating Elevator Spade is well suited to create a purchase point for the application of additional ProLux™ Luxating Elevators or other instrumentation, such as extraction forceps.

Recommended Area of Use:

- Premolar to Premolar in Upper and Lower Arch

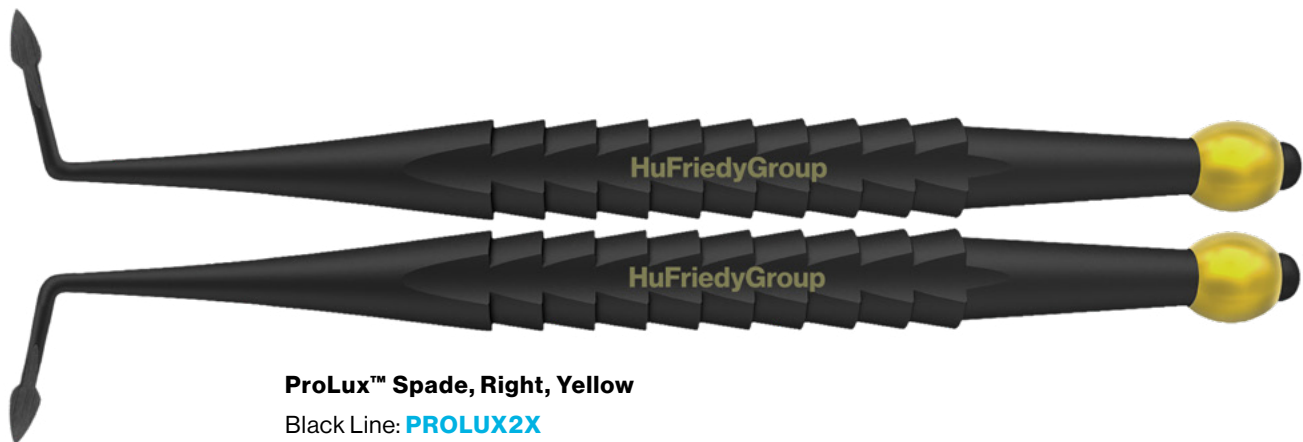

ProLux™ Spade, Right, Yellow

Black Line: **PROLUX2X**

Satin Steel: **PROLUX2**

ProLux™ Spade, Left, Yellow

Black Line: **PROLUX3X**

Satin Steel: **PROLUX3**

These ProLux™ Luxating Elevator Spades utilize the familiar 'East/West' angulation for application in the mandibular molar area.


Their thin profile and sharpened edges incise the ligament quickly and expand the alveolus laterally.

In cases where there is a lack of coronal structure or the root has fractured subgingivally, the tip of the spade can still be easily advanced along the side of the root.

These ProLux™ Luxating Elevator Spades are well suited to create a purchase point for the application of additional ProLux™ Luxating Elevators or other instrumentation, such as extraction forceps.

Recommended Area of Use:

- Mandibular Molars
- Root Tips

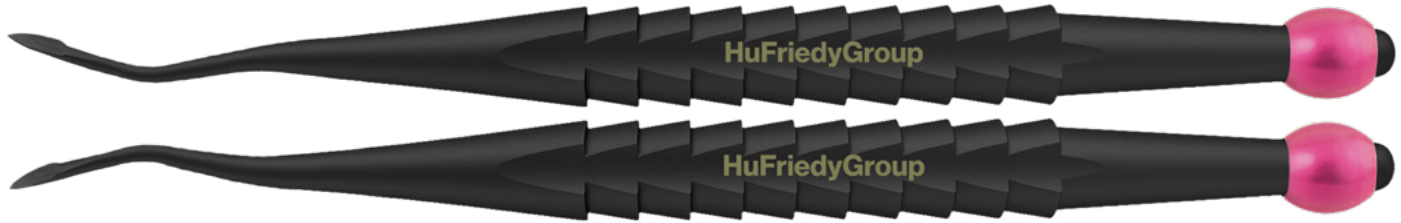

ProLux™ Spade, Right, Purple

Black Line: **PROLUX4X**

Satin Steel: **PROLUX4**

ProLux™ Spade, Left, Purple

Black Line: **PROLUX5X**

Satin Steel: **PROLUX5**

These ProLux™ Luxating Elevators feature the same fine point of a sharpened spade but orientated for distal and lingual or mesial application. The thin profile and sharpened edges incise the ligament quickly and expand the alveolus laterally.

In cases where there is a lack of coronal structure, or the root has fractured subgingivally, the tip of the ProLux™ Luxating Elevator can still be easily advanced along the side of the root. This ProLux™ Luxating Elevator Spade is well suited to create a purchase point for the application of additional ProLux™ Luxating Elevators or other instrumentation, such as extraction forceps.

PROLUX4X and PROLUX4 Recommended Area of Use:

- Distal and Lingual Maxillary Premolars and Molars
- Root Tips

PROLUX5X and PROLUX5 Recommended Area of Use:

- Mesial and Bucco-Mesial Maxillary Premolars and Molars
- Root Tips



ProLux™ Small, Straight, 2.5mm Wide, Tip Green

PROLUX6

The narrow Small Straight ProLux™ Luxating Elevator (2.5mm) can be advanced into the tightest of periodontal spaces to completely sever the fibers.

Its completely straight design enables a direct transfer of apical force to the honed leading edge to quickly sever the PDL and expand the socket laterally.

Recommended Area of Use:

- Premolar to Premolar in Upper and Lower Arches
- Roots




ProLux™ Small, Curved, 2.5mm Wide Tip, Blue
PROLUX7

The narrow Small Curved ProLux™ Luxating Elevator (2.5mm) can be advanced into the tightest of periodontal spaces to completely sever the fibers.

The slight back curvature of this ProLux™ Luxating Elevator allows for better access to root tips and into the posterior than the Small Straight ProLux™ Luxating Elevator.

Recommended Area of Use:

- Mesial of Molars to Mesial of Molars in Upper and Lower Arches
- Roots

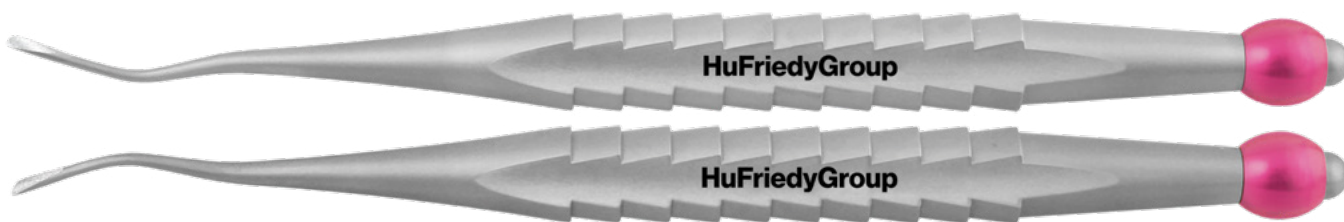

ProLux™ Small Distal Angle, Right, 2.5mm, Purple
PROLUX8

ProLux™ Small Distal Angle, Left, 2.5mm, Purple
PROLUX9

These narrow Small ProLux™ Luxating Elevators (2.5mm) can be advanced into the tightest of periodontal spaces to completely sever the fibers and laterally expand the socket.

The angulation of these ProLux™ Luxating Elevators makes them particularly effective for application on the distal (PROLUX8) or mesial (PROLUX9) of a single-rooted tooth, as well as for the removal of sectioned molar roots.

PROLUX8 Recommended Areas of Use:

- Distal and Lingual Maxillary Premolars and Molars
- Roots

PROLUX9 Recommended Areas of Use:

- Mesial and Bucco-Mesial Maxillary Premolars and Molars
- Roots

PROLUX™ LUXATING ELEVATOR KITS



PROLUXSPADES
(Satin Steel)



PROLUXSPADESX
(Black Line)

	Satin Steel	Black Line
PROLUX™ LUXATING ELEVATORS SPADE KIT A collection of 5 ProLux™ Luxating Elevator Spades and a cassette.	PROLUXSPADES	PROLUXSPADESX
ProLux Spade, Yellow	PROLUX1	PROLUX1X
ProLux Distal, Angle Spade, Purple	PROLUX4	PROLUX4X
ProLux Mesial, Angle Spade, Purple	PROLUX5	PROLUX5X
ProLux Spade, Right, Yellow	PROLUX2	PROLUX2X
ProLux Spade, Left, Yellow	PROLUX3	PROLUX3X
5 Instrument Infinity Series™ Exam Cassette, Black	IMN6057	IMN6057



PROLUXKIT4

PROLUX™ LUXATING ELEVATORS KIT OF 4 A collection of 4 ProLux™ Luxating Elevators with a sharpening stone and a cassette.	PROLUXKIT4
ProLux Small, Straight, 2.5mm Wide Tip, Green	PROLUX6
ProLux Small, Curved, 2.5mm Wide Tip, Blue	PROLUX7
ProLux Small, Distal Angle, 2.5mm, Purple	PROLUX8
ProLux Small, Mesial Angle, 2.5mm, Yellow	PROLUX9
Ceramic, Cyndrical Sharpening Stone	SSKC
5 Instrument Infinity Series™ Exam Cassette, Black	IMN6057

**The
Best In
Practice**

Learn More at **HuFriedyGroup.com**

Hu-Friedy Mfg. Co., LLC, 1666 E. Touhy Ave., Des Plaines, IL 60018 | HuFriedyGroup.com

All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC its affiliates or related companies, unless otherwise noted. Marks not registered in all jurisdictions.

©2024 Hu-Friedy Mfg. Co., LLC. All rights reserved. HFL-923/2411