

Appropriate settings for the Piezo Tips.
 Réglages appropriés pour les Inserts Piezo.

Tip	PWR PIEZO			PWR PAIR		
	endo	perio/scaler	restorative	endo	perio/scaler	restorative
SCALING						
UPS1S		1 - 2 ^[1]			1 - 2 ^[1]	
UPS1 - UPS2 - UPS3 - UPS4 - UPS5		1 - 6 ^[1]			1 - 5 ^[1]	
PERIO						
UPP2 - UPP3 UPP10 - UPP11 - UPP12 - UPP13 - UPP14		1 - 3 ^[1]			1 - 3 ^[1]	
UPP15 - UPP16R - UPP16L		1 - 3 ^[1]			1 - 3 ^[1]	
IMPLANT CLEANING						
UPICTH + UPIC1		1 - 2 ^[1]			1 - 2 ^[1]	
ENDO REVISION						
UPER1		1 - 6			1 - 5	
UPER2 - UPER3 - UPER4 - UPER5		1 - 3			1 - 3	
ENDO RETRO						
UPRD3 - UPRD4 - UPRD5		1 - 3			1 - 3	

[1] Can be used with or without "soft mode" function. / Il peut être utilisé avec ou sans la fonction "soft mode".

PWR Tips Setting Table

Tableau de Reglage des Inserts PWR



Manufactured for / Fabriqué pour:
Hu-Friedy Mfg. Co., LLC
2501 Sanders Rd.,
Northbrook, IL 60062 | USA
1-800-Hu-Friedy | HuFriedyGroup.com

All company and product names are trademarks of
Hu-Friedy Mfg. Co., LLC, its affiliates or related
companies,
unless otherwise noted.
Marks not registered in all jurisdictions.
© 2026 Hu-Friedy Mfg. Co., LLC. All rights reserved

02150831 - PWR TIPS SETTING TABLE
REV 00 - 20/06/2024