## English

### INSTRUCTIONS FOR USE

The PWR Air Perio Subgingival Tip (APPERIOTIP) is a plastic insert to be fit on PWR Air Perio Handpiece for subgingival use. It is used in periodontal pockets deeper than 5 mm, exclusively with specific PWR PERIO powder for subgingival use distributed by HuFriedyGroup.

The PWR Air Perio Subgingival Tip (APPERIOTIP) must be exclusively used in conjunction with the PWR Air Perio Handpiece, according to the intended use defined in the PWR Pair Combo Unit "USE & MAINTENANCE MANUAL".

**⚠ WARNING:** The PWR Air Perio Subgingival Tips (APPERIOTIPS) must be used only by specialized and properly trained personnel, such as a Physician, Dentist and/or Dental Hygienist, able-bodied adults of any weight, age, height, gender and nationality.

**NOTE:** Regarding the intended patient group, patient selection criteria, indications for use, contraindications and warnings, refer to the PWR Pair Combo Unit "USE & MAINTENANCE MANUAL".

**IMPORTANT:** The terms "**WARNING**" and "**CAUTION**" and "**NOTE**" identify conditions that require particular caution and must, therefore, be examined carefully. In particular:

**⚠ WARNING:** Identifies conditions and practices that present a risk of death or serious injury for the patient and/or user.

**⚠ CAUTION:** Identifies conditions and practices that may cause minor injuries and/or damage to the devices.

**NOTE:** Warning not associated with a hazardous or imminently dangerous condition.

**⚠ CAUTION:** The information contained in this document is limited to that strictly necessary for the first use of the product and must be consulted as a supplement to the PWR Pair Combo Unit "USE & MAINTENANCE MANUAL". Read the PWR Pair Combo Unit "USE & MAINTENANCE MANUAL" paying particular attention to the "Safety Requirements" chapter, before performing any type of operation on the system and commencing any type of treatment.

**⚠ WARNING:** If a serious accident attributable to the device occurs during correct and intended use, it is recommended to report it to the Competent Authority and to the company indicated on the product label.

**Sterility:** The APPERIOTIP is supplied sterile by prior exposure to ETO.

**⚠ WARNING:** The APPERIOTIP is single use only.

- The APPERIOTIP is intended to be used on an individual patient during a single procedure and then it must be disposed of.
- The APPERIOTIP shall not be reprocessed, re-sterilized and used again. Remove and dispose of the APPERIOTIP following the current regulations in force about the correct disposal of hospital waste.
- Reuse of the APPERIOTIP can cause serious injury to the patient.
- Only use authentic APPERIOTIP.
- Before intervention: Inspect the sterile package and the product to check for integrity. Do not use the APPERIOTIP if package is opened or damaged. APPERIOTIP loses sterility in case of broken or damaged packaging. In case of damaged packaging, properly dispose of the APPERIOTIP. Do not sterilize and use the APPERIOTIP.

### USE

**⚠ CAUTION:** Before operating, set the entire system up and verify each function; read the "USE & MAINTENANCE MANUAL" of the device.

**⚠ CAUTION:** Adjust the water flow in order to have a minimum level.

#### Connection of APPERIOTIP to the PWR Air Perio Handpiece

Insert the APPERIOTIP onto the PWR Air Perio Handpiece, pushing it until it bottoms up (Figure 1).

**⚠ CAUTION:** Handle the APPERIOTIP with care.

**⚠ CAUTION:** The APPERIOTIP can be used only with the PWR Air Perio Handpiece.

#### Disconnection of the APPERIOTIP from the PWR Air Perio Handpiece

Remove the APPERIOTIP by using exclusively the PWR AIR K9 Tip Wrench supplied with the device.

**NOTE:** Introduce the PWR AIR K9 Tip Wrench exactly into the position indicated in Figure 2.

**⚠ WARNING:**

- Before intervention: Make sure the APPERIOTIP is correctly fully inserted on the PWR Air Perio Handpiece; the two pieces must be in contact.
- Do not replace the APPERIOTIP while the handpiece is operating in order to prevent any user injury.
- Excessive pressure applied on the APPERIOTIP may cause harm to the patient.
- Inspect the APPERIOTIP before and during use. If damage is observed replace it with a new one.

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All company and product names are trademarks of
©2024 Hu-Friedy Mfg. Co., LLC. All rights reserved.
Hu-Friedy Mfg. Co., LLC
Hu-Friedy Mfg. Co., LLC, its affiliates or related