



# ProLux<sup>TM</sup>

Luxating Elevators


 **HuFriedyGroup**  
The Best In Practice

## Luxate and Elevate

---

ProLux™ Luxating Elevators are manufactured with thin, sharp working ends which allow for atraumatic extractions—they help to preserve the socket while gently luxating the tooth. This atraumatic approach can lead to increased patient comfort and enhanced healing, providing optimal clinical results.

Working end designs allow for **quick definition of the purchase point**

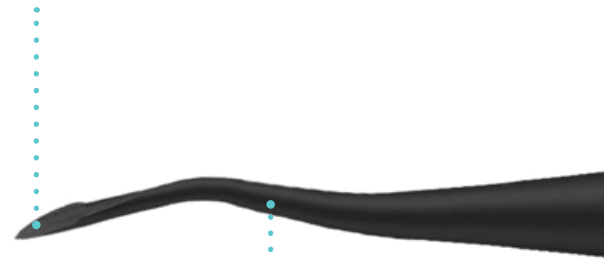

**Tapered, single-piece construction** directs tip apically



## Black Line Instrument Collection

- Performance engineered coating for a harder, smoother surface for **optimal edge retention and enhanced lubricity**
- Distinct black finish for **enriched contrast and visual acuity at the surgical site and underlying tissue**
- **Minimal light reflection** afforded by a sleek black instrument design
- Handcrafted with Immunity Steel® for **optimal strength and corrosion resistance**



Ergonomically designed handle provides **enhanced comfort and control**

HuFriedyGroup

# PROLUX™ LUXATING ELEVATORS



## ProLux™ Spade, Yellow

Black Line: **PROLUX1X**


Satin Steel: **PROLUX1**

The fine point of the ProLux™ Luxating Elevator Spade facilitates sulcular retraction and aids in positive engagement into the PDL space. The thin profile and sharpened edges incise the ligament quickly and expand the alveolus laterally.

In cases where there is a lack of coronal structure, or the root has fractured subgingivally, the sharpened tip of the spade can still be easily advanced along the side of the root. This ProLux™ Luxating Elevator Spade is well suited to create a purchase point for the application of additional ProLux™ Luxating Elevators or other instrumentation, such as extraction forceps.

### Recommended Area of Use:

- Premolar to Premolar in Upper and Lower Arch

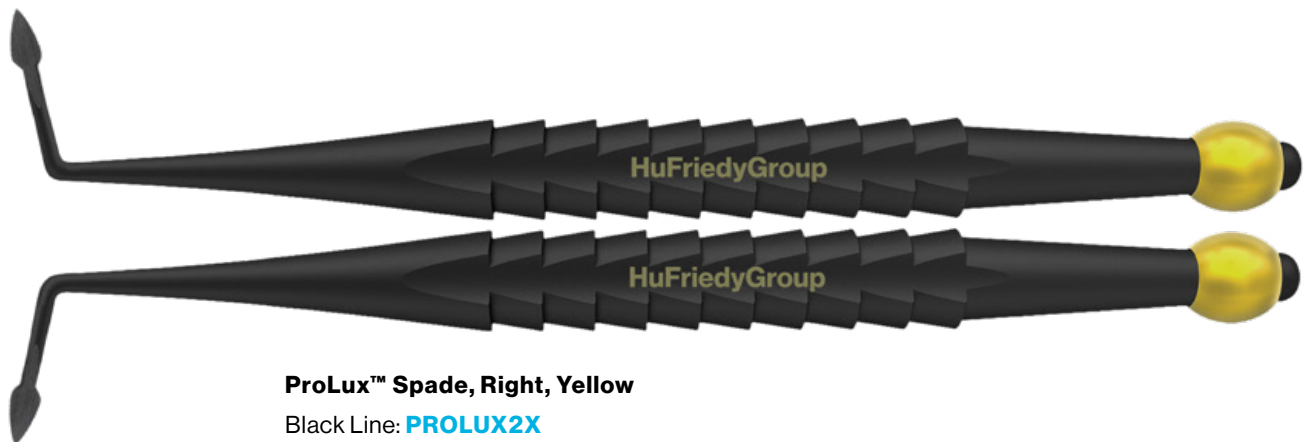

## ProLux™ Spade, Right, Yellow

Black Line: **PROLUX2X**

Satin Steel: **PROLUX2**

## ProLux™ Spade, Left, Yellow

Black Line: **PROLUX3X**

Satin Steel: **PROLUX3**

These ProLux™ Luxating Elevator Spades utilize the familiar 'East/West' angulation for application in the mandibular molar area.


Their thin profile and sharpened edges incise the ligament quickly and expand the alveolus laterally.

In cases where there is a lack of coronal structure or the root has fractured subgingivally, the tip of the spade can still be easily advanced along the side of the root.

These ProLux™ Luxating Elevator Spades are well suited to create a purchase point for the application of additional ProLux™ Luxating Elevators or other instrumentation, such as extraction forceps.

### Recommended Area of Use:

- Mandibular Molars
- Root Tips

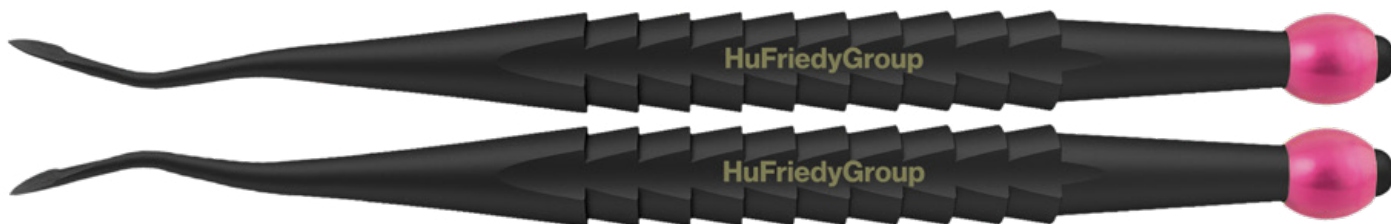

#### **ProLux™ Spade, Right, Purple**

Black Line: **PROLUX4X**

Satin Steel: **PROLUX4**

#### **ProLux™ Spade, Left, Purple**

Black Line: **PROLUX5X**

Satin Steel: **PROLUX5**

These ProLux™ Luxating Elevators feature the same fine point of a sharpened spade but orientated for distal and lingual or mesial application. The thin profile and sharpened edges incise the ligament quickly and expand the alveolus laterally.

In cases where there is a lack of coronal structure, or the root has fractured subgingivally, the tip of the ProLux™ Luxating Elevator can still be easily advanced along the side of the root. This ProLux™ Luxating Elevator Spade is well suited to create a purchase point for the application of additional ProLux™ Luxating Elevators or other instrumentation, such as extraction forceps.

#### **PROLUX4X and PROLUX4 Recommended Area of Use:**

- Distal and Lingual Maxillary Premolars and Molars
- Root Tips

#### **PROLUX5X and PROLUX5 Recommended Area of Use:**

- Mesial and Bucco-Mesial Maxillary Premolars and Molars
- Root Tips



#### **ProLux™ Small, Straight, 2.5mm Wide, Tip Green**

**PROLUX6**

The narrow Small Straight ProLux™ Luxating Elevator (2.5mm) can be advanced into the tightest of periodontal spaces to completely sever the fibers.

Its completely straight design enables a direct transfer of apical force to the honed leading edge to quickly sever the PDL and expand the socket laterally.

#### **Recommended Area of Use:**

- Premolar to Premolar in Upper and Lower Arches
- Roots




**ProLux™ Small, Curved, 2.5mm Wide Tip, Blue**  
**PROLUX7**

The narrow Small Curved ProLux™ Luxating Elevator (2.5mm) can be advanced into the tightest of periodontal spaces to completely sever the fibers.

The slight back curvature of this ProLux™ Luxating Elevator allows for better access to root tips and into the posterior than the Small Straight ProLux™ Luxating Elevator.

**Recommended Area of Use:**

- Mesial of Molars to Mesial of Molars in Upper and Lower Arches
- Roots

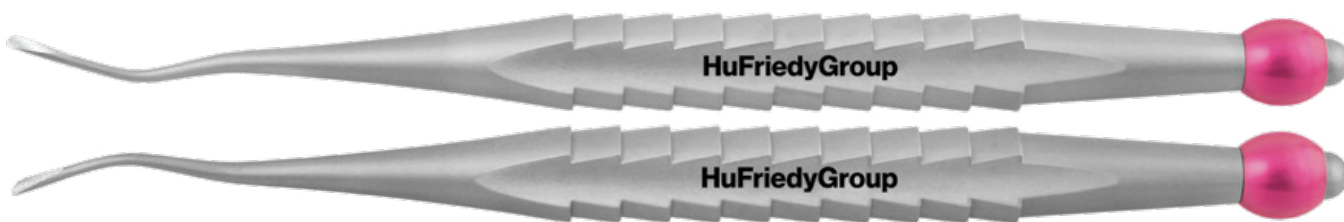

**ProLux™ Small Distal Angle, Right, 2.5mm, Purple**  
**PROLUX8**

**ProLux™ Small Mesial Angle, Left, 2.5mm, Purple**  
**PROLUX9**

These narrow Small ProLux™ Luxating Elevators (2.5mm) can be advanced into the tightest of periodontal spaces to completely sever the fibers and laterally expand the socket.

The angulation of these ProLux™ Luxating Elevators makes them particularly effective for application on the distal (PROLUX8) or mesial (PROLUX9) of a single-rooted tooth, as well as for the removal of sectioned molar roots.

**PROLUX8 Recommended Areas of Use:**

- Distal and Lingual Maxillary Premolars and Molars
- Roots

**PROLUX9 Recommended Areas of Use:**

- Mesial and Bucco-Mesial Maxillary Premolars and Molars
- Roots

# PROLUX™ LUXATING ELEVATOR KITS



**PROLUXSPADES**  
(Satin Steel)



**PROLUXSPADESX**  
(Black Line)

	Satin Steel	Black Line
<b>PROLUX™ LUXATING ELEVATORS SPADE KIT</b> A collection of 5 ProLux™ Luxating Elevator Spades and a cassette.	<b>PROLUXSPADES</b>	<b>PROLUXSPADESX</b>
ProLux Spade, Yellow	<b>PROLUX1</b>	<b>PROLUX1X</b>
ProLux Distal, Angle Spade, Purple	<b>PROLUX4</b>	<b>PROLUX4X</b>
ProLux Mesial, Angle Spade, Purple	<b>PROLUX5</b>	<b>PROLUX5X</b>
ProLux Spade, Right, Yellow	<b>PROLUX2</b>	<b>PROLUX2X</b>
ProLux Spade, Left, Yellow	<b>PROLUX3</b>	<b>PROLUX3X</b>
5 Instrument Infinity Series™ Exam Cassette, Black	<b>IMN6057</b>	<b>IMN6057</b>



**PROLUXKIT4**

<b>PROLUX™ LUXATING ELEVATORS KIT OF 4</b> A collection of 4 ProLux™ Luxating Elevators with a sharpening stone and a cassette.	<b>PROLUXKIT4</b>
ProLux Small, Straight, 2.5mm Wide Tip, Green	<b>PROLUX6</b>
ProLux Small, Curved, 2.5mm Wide Tip, Blue	<b>PROLUX7</b>
ProLux Small, Distal Angle, 2.5mm, Purple	<b>PROLUX8</b>
ProLux Small, Mesial Angle, 2.5mm, Purple	<b>PROLUX9</b>
Ceramic, Cyndrical Sharpening Stone	<b>SSKC</b>
5 Instrument Infinity Series™ Exam Cassette, Black	<b>IMN6057</b>

**The**  
**Best In**  
**Practice**

Learn More at **HuFriedyGroup.com**

All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC its affiliates or related companies, unless otherwise noted. Marks not registered in all jurisdictions.  
HuFriedyGroup.com

©2025 Hu-Friedy Mfg. Co., LLC. All rights reserved. HFL-923/2507