
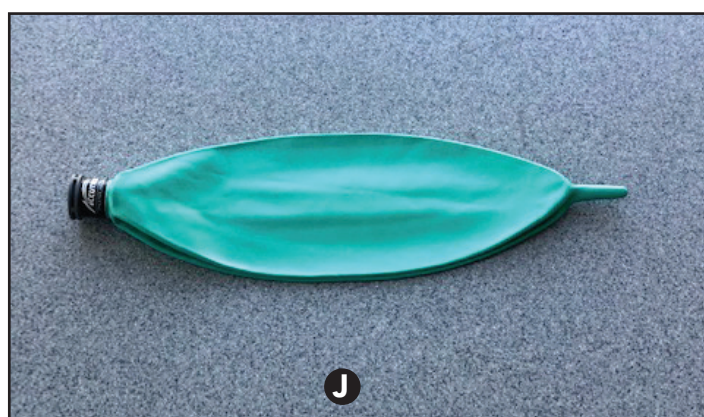
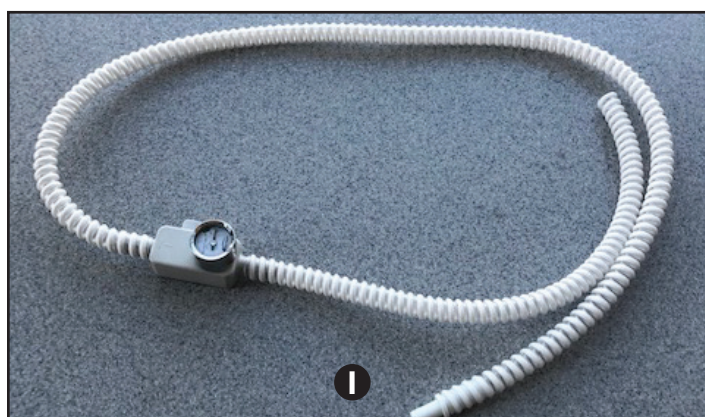
Accutron™ Circuits and Masks

REPROCESSING INSTRUCTIONS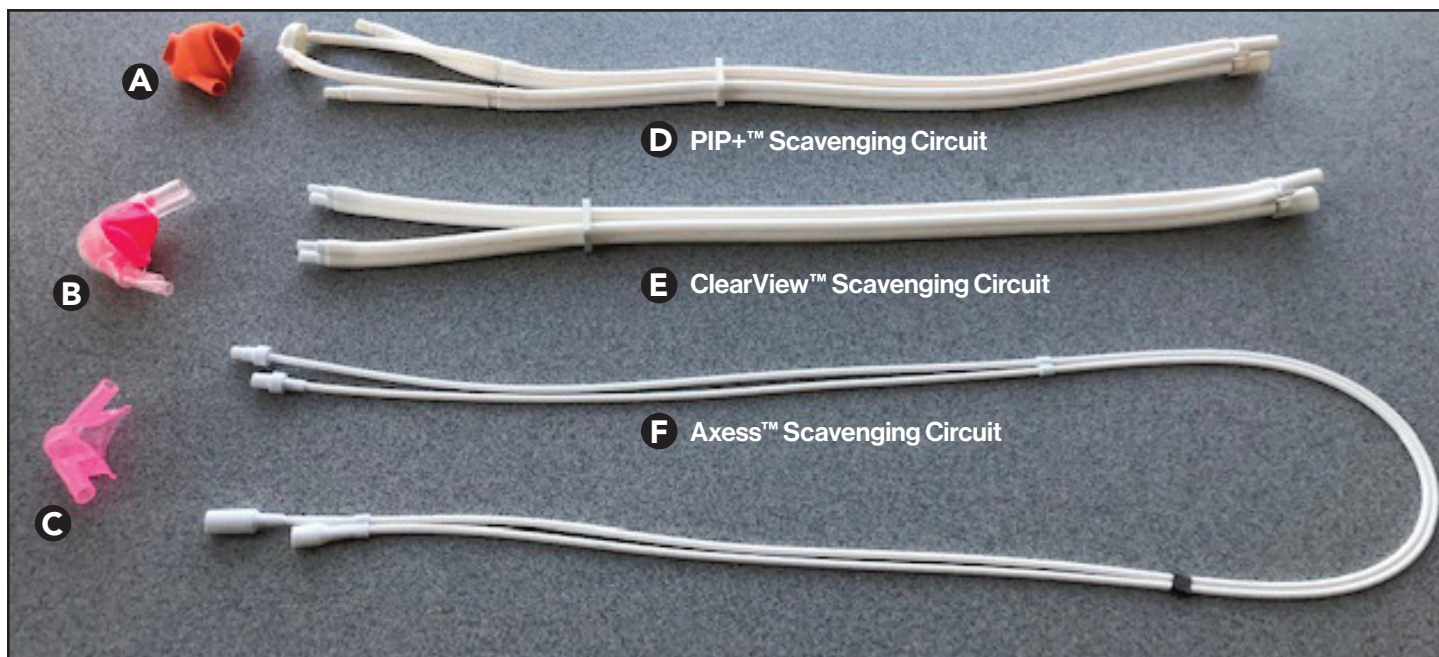

WARNING: Reprocess scavenging circuits and multi-use nasal masks after each use per following instructions. Failure to follow these instructions could lead to patient or user infection and adverse health impact.

Instructions:

A, B, C Single-use only. Discard after each use. Do not reprocess.

D, E, F, G : After removing single-use masks from circuits **D, E, F**, reprocess per instructions below.

Note: Leave hub attached to small tubes on **D**.

H, I, J: Wipe exterior surfaces with disinfectant. Do not immerse gauge of **I**.

OPTION 1: Automated washer/thermal disinfectant. (Connect tubing to fittings to allow flow through tubing during cycle):

Typical US Automated Washer/Disinfectant Cycle	
Step	Description
1	2 minute prewash with cold tap water
2	20 second enzyme spray with hot tap water
3	1 minute enzyme soak
4	15 second hot tap water rinse
5	2 minute detergent wash with hot tap water (64-66°C/146-150°F)
6	15 second hot tap water rinse
7	2 minute thermal rinse (80-93°C/176-200°F)
8	7-30 minute hot air dry (116°C/240°F)

Typical European Automated Washer/Disinfectant Cycle	
Step	Description
1	2 minute pre-cleaning with cold tap water, draining
2	5 minute alkaline cleaning at 55°C, draining
3	3 minute neutralization rinse with cold tap water, draining
4	2 minute rinse with cold tap water, draining
5	5 minute thermal disinfection at 90° C with hot, demineralized water
6	30 minute hot air drying

Note: The washer/disinfectant manufacturer's instructions should be strictly adhered to. Use only cleaning agents recommended for the specific type of automated washer/disinfectant. A washer/disinfectant with approved efficacy (e.g. CE mark, validation according to ISO 15883) should be used.

OPTION 2: Steam sterilization:

1. Rinse circuits and multi-use nasal masks (DEFG) and remove gross contaminants manually with detergent solution and/or enzymatic detergents following its manufacturer's instructions. Allow to dry before next step.
2. When preparing for sterilization, avoid kinks in tubing. Either wrap product in two layers of sterilization wrap or place in a Sure-Check™ Self-Seal Sterilization Pouch with a suggested minimum size of 12 in x 18 in for **D** PIP+™ Scavenging Circuits or **E** ClearView™ Scavenging Circuits and 7.5 in x 14 in for **F** Axess™ Scavenging Circuits to avoid kinks/bends.
3. Steam sterilize per one of the following validated cycles.

Note: All Accutron circuits were validated per EN ISO 14937:2009 for 132°C for 3 minutes in a Pre-Vacuum style sterilizer. Therefore, any Pre-Vacuum steam sterilization cycles that are ≥ 132°C and ≥ 3 minutes are acceptable.

Approved European Cycles:

134°C for ≥ 3 minutes, Pre-Vacuum sterilizers (Note: times vary from 3 -18 minutes depending on country).

Approved cycles for US and other countries allowing temps < 134°C (temps < 134°C prohibited in Europe):

130°C (266°F) not validated or approved.

132°C (269.9°F) for ≥ 3 minutes, Pre-Vacuum sterilizers.

132°C (269.9°F) for ≥ 15 minutes, Gravity Style sterilizers (this cycle approved per US test methods).

134°C (273.2°F) for ≥ 3 minutes, Pre-Vacuum sterilizers.

135°C (275°F) for ≥ 3 minutes, Pre-Vacuum sterilizers.

LIMITATIONS: Circuits **D, E, F** and multi-use masks **G** have been steam sterilized 250 times without significant loss of function. Inspect prior to use per instructions below as damage could occur over time.

STORAGE: Use normal asepsis containers and locations.

MAINTENANCE, INSPECTION, AND TESTING: Prior to use, visually inspect for damage to tubing or fittings that could hinder flow or allow gas to leak. Discard and replace damaged items.

Accutron, Inc.
1625 W. Pinnacle Peak Rd., Phoenix, AZ 85027 USA
800.531.2221 | HuFriedyGroup.com/Accutron

EU REP MT Promed Consulting GmbH
Ernst-Heckel-Straße 7
66386 St. Ingbert
Germany

CH REP MDSS CH GmbH
Laurenzenvorstadt 61
5000 Aarau
Switzerland

UK REP MDSS-UK RP Ltd.
6 Wilmslow Road, Rusholme
Manchester M14 5TP
United Kingdom

Made in USA 26613-INT / REV 15 - 06/2025

All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC, its affiliates or related companies, unless otherwise noted.
©2026 Hu-Friedy Mfg. Co., LLC. All rights reserved.

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© Hu-Friedy Mfg. Co., LLC. Toate drepturile rezervate.