

Maintaining Your PWR Air

Daily Care Tips for Your Device



Ensure staying power and protect your investment by regularly maintaining your PWR Air device. Adhere to the following PWRful tips for **Best In Practice** performance.




After Each Patient Maintenance Checklist

- Use Flush Function for both the PWR Air Polishing and PWR Piezo LED handpieces.
- Disconnect the PWR Air Polishing handpiece from its cord. Refer to below illustrations regarding disconnecting PWR Air Polishing Handpiece.



- Clean and Sterilize PWR Air Polishing handpiece in individual sterilization pouch.

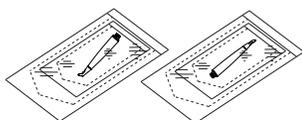


- Clean non-sterilizable parts of the device with a compatible EPA-registered intermediate-level surface disinfectant.



For a full list of instructions, please refer to the **PWR Air Instructions For Use**.

End of Day Maintenance Checklist

- Use the “Refill” function to release pressure and proceed to remove the powder chambers.
REMANDER: Wait until the “Refill” LED stops flashing

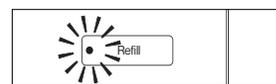

- Dispose of remaining powder.
WARNING: May be compromised by moisture
- Clean the powder chamber with compressed air.
REMANDER: Be sure to blow compressed air between threads and on the o-ring
- Use the Flush function with your PWR Air Polishing Handpiece attached.
- Ensure that your PWR Air device is switched off.
- Clean the PWR Air device, handpiece cord and footswitch using a clean, dry cloth with a low fiber release, dampened with a detergent solution (pH 6-9).

