
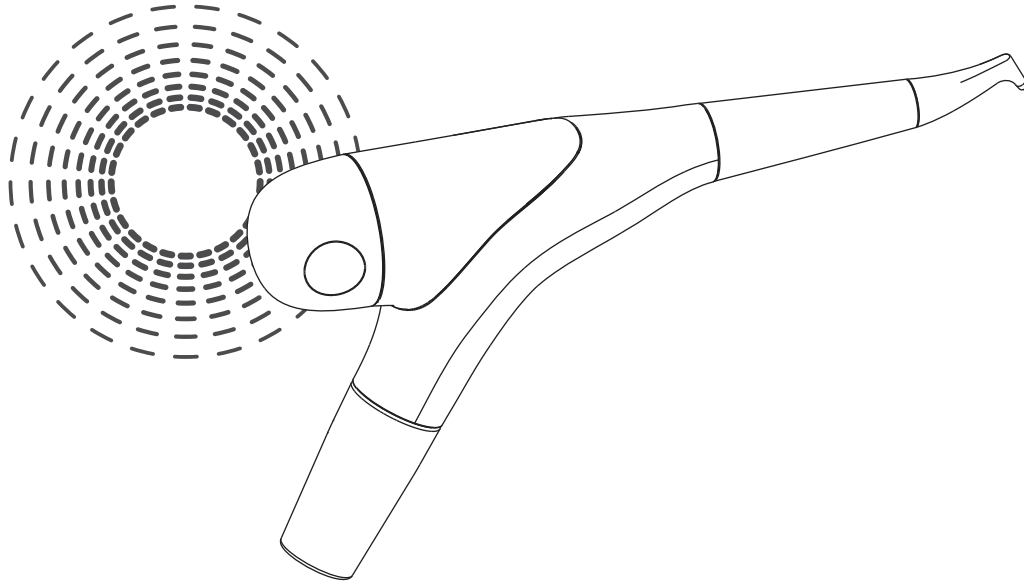
AIR-FLOW[®] handy 3.0

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Rx ONLY Caution! Federal (USA) law restricts this device to sale by or on the order of a dentist.

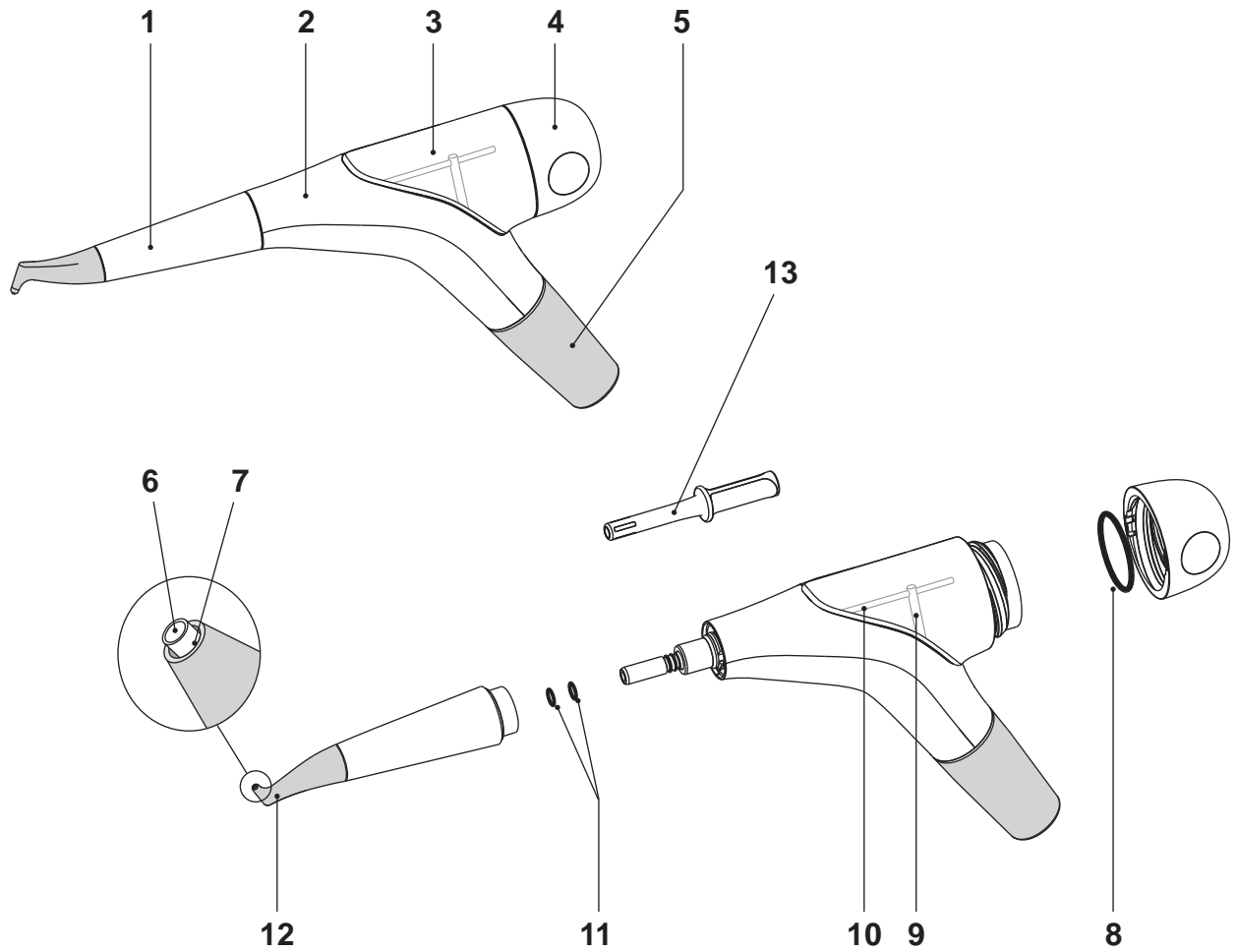Hu-Friedy  **EMS**⁺
ELECTRO MEDICAL SYSTEMS

COMPONENTS

- 1 Handpiece
- 2 Body
- 3 Powder chamber
- 4 Powder chamber cap
- 5 Cord adapter
- 6 Powder outlet tube
- 7 Water outlet port
- 8 Cap ring
- 9 Air tube
- 10 Air/powder tube
- 11 O-rings for handpiece connection
- 12 Nozzle
- 13 EasyClean

Hu-Friedy & EMS supply units with various accessories. The "Packing list" shows exactly what your unit includes.

Technical modifications

Hu-Friedy & EMS reserve the right to modify the technique, accessories, operating instructions or contents of the set due to technical or scientific improvements.

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DEAR CUSTOMERS,

Thank you for buying a new Hu-Friedy & EMS product. It meets the highest quality and safety standards.

Adaptable on the turbine connection of your dental unit, the air polishing AIR-FLOW handy 3.0 functions with the powders provided by Hu-Friedy & EMS.

The confinement of the powder by a water jet makes it possible to steer the water jet with great precision and thus ensures a pleasant treatment for your patient.

This product removes dental plaque, soft deposits and surface stains from pits, grooves, interproximal spaces or smooth surfaces of the teeth.

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ABOUT THIS MANUAL

Note that the English version of this manual is the master from which translations derive. In case of any discrepancy, the binding version is the English text.

These operating instructions are to ensure the correct installation and use of this product.

 Please read these operating instructions carefully as they explain all of the most important details and procedures. Please pay special attention to the safety precautions.

Always keep this instruction close to hand.

To prevent injury to people and damage to property, please heed the corresponding directives. They are marked as follows:

Danger

Risk of injury

Caution

Risk of property damage or environmental harm

Please note

Useful additional information and hints

Wear gloves

Prohibited

Authorized

We would be pleased to answer your questions or listen to your suggestions. We do, of course, provide support in case of technical problems. Please call directly your approved Hu-Friedy & EMS repair center or your dealer.

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Hu-Friedy & EMS

CONTENTS

Treatment	2	Maintenance.....	52
Indications for use	2	Replacing the cap and the O-rings.....	52
Contra-indications.....	2	Annual maintenance	54
Installation and setting up	4	Safety precautions	56
Water supply	4	Storing the product	58
Compressed air supply	4	Product disposal	60
Checking the turbine connection (Typical is shown).....	6	Warranty.....	60
Handpiece connection	8	Accessories.....	60
Drying the sterilized handpiece	8	Hu-Friedy & EMS-Service	62
Connecting to the dental unit	10	Symbols	64
Setting the water flow rate	12	Technical data.....	68
Filling the powder chamber with Easy Fill ..	14	Troubleshooting	70
Closing the cap	16		
Handling and water/air settings.....	18		
Precautions	20		
Treatment	24		
Working Technique	26		
Precautions to be taken at the end of the treatment	30		
Fluoride Application	32		
Patient Information.....	34		
Start the cleaning	36		
Cleaning and sterilizing	38		
Cleaning the handpiece	38		
Unclogging / Cleaning the handpiece using the Easy Clean	40		
Cleaning the Easy Clean.....	42		
Cleaning and disinfecting the body and body cord.....	44		
Daily cleaning of the powder chamber	46		
Daily cleaning of the powder chamber cap	48		
Sterilization of the handpiece and the Easy Clean	50		

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TREATMENT

Indications for use

The AIR-FLOW handy 3.0 Dental Handpiece is a dental handpiece intended for use in the cleaning and polishing of teeth by the projection of a mixture of water, air, and Hu-Friedy & EMS prophylaxis powder onto the tooth surface.

The device removes soft deposits and areas of discoloration and can be used to prepare teeth for dental procedures such as the placement of composite fillings, porcelain inlays, and laminate veneers.

The device can be used to clean implant abutments and to clean teeth prior to treatments such as shade matching, fluoridation, and bleaching.

The device can also be used to degrease crowns and bridges prior to placement and clean fixed bands and brackets on orthodontic appliances.

Contra-indications

 In order to avoid any risk of emphysema, hypersensitivity, bleeding and patient discomfort, the use of supragingival powders (Hu-Friedy & EMS CLASSIC, CLASSIC COMFORT and SOFT) is strictly forbidden during subgingival treatments (contacts with soft tissues, dentine, cementum).

 Patients suffering from chronic bronchitis or asthma must not, under any circumstances, be treated with an air polishing device. The jet of air and powder could cause respiratory difficulties.

 Patients on a low salt diet must not be treated with a powder containing sodium bicarbonate. For patients on a low salt diet use the powders without sodium bicarbonate provided by Hu-Friedy & EMS.

 Predisposed persons may be sensitive to the powder. If allergic reactions are observed, stop using the product and completely remove it.

 The aromatized powder provided by Hu-Friedy & EMS can lead, in specific cases, to allergic reactions to the flavour. For patients known to suffer such reactions, use the powders without flavour provided by Hu-Friedy & EMS.

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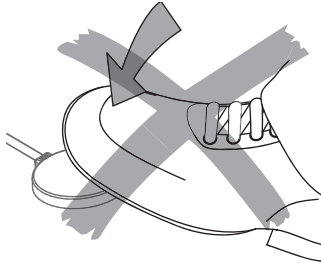
Checking the turbine connection (Typical is shown)


 The unit is equipped with an adapter specially designed for the turbine connection of your dental unit. Only use the unit with this special turbine connection. To connect it to another type of turbine will damage it.

 The turbine of your dental unit must not be under pressure when the AIR-FLOW handy 3.0 is connected. Do not activate the turbine foot pedal. If you have a turbine equipped with light, turn the light off.

 Make sure the o-rings of your turbine connection are in good condition. A turbine connection with o-rings in bad condition can damage the unit.

 The user should check the compatibility of this connection with his chair.

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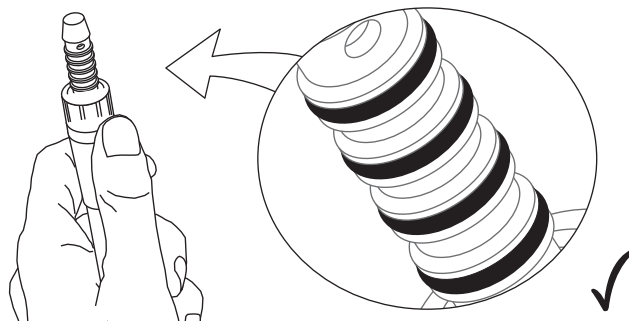
HANDPIECE CONNECTION

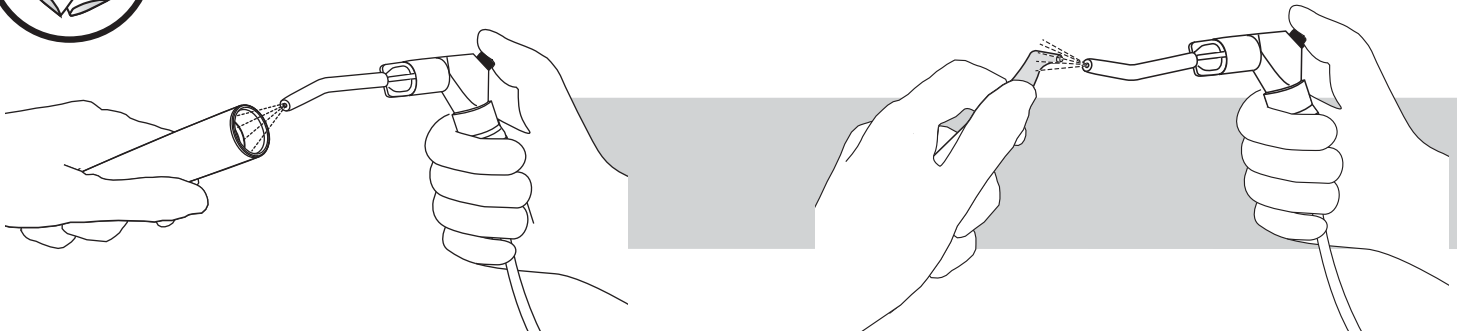Drying the sterilized handpiece


Clean and sterilize the handpiece before first use and after each treatment as described in the chapter "Cleaning and sterilizing" in this manual.

Following sterilization, blow the interior of the handpiece from both sides with compressed air to eliminate possible humidity.

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
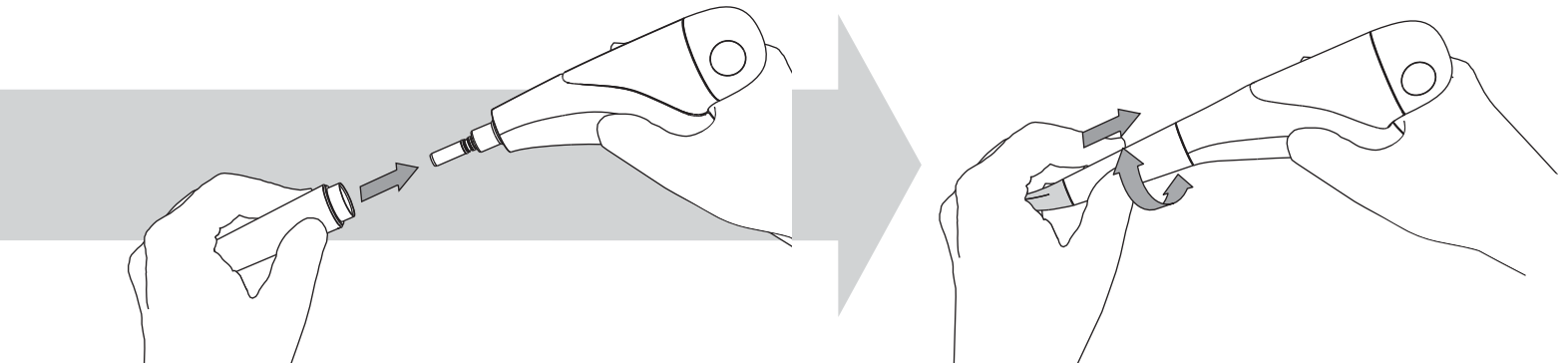
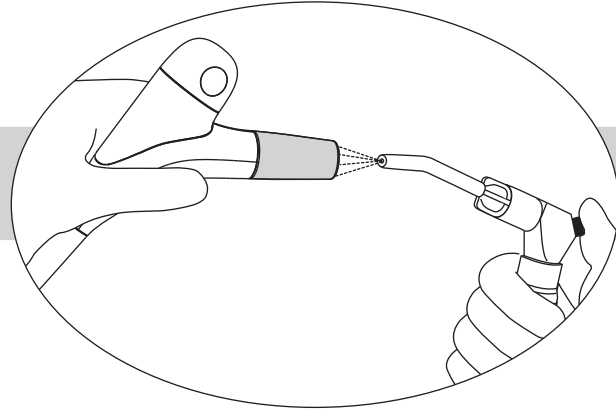
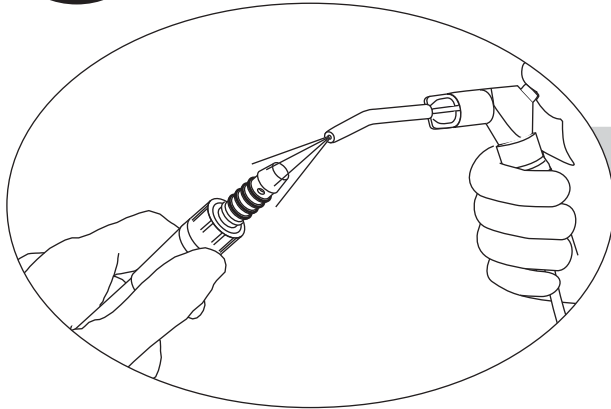
Connecting to the dental unit



The turbine connection and the connector must be absolutely dry. Moisture at the connection can cause blockage in the air/powder channels of the unit.

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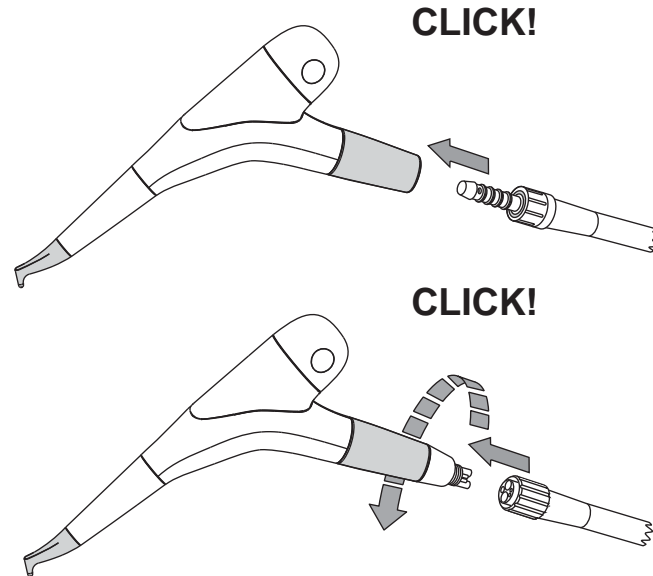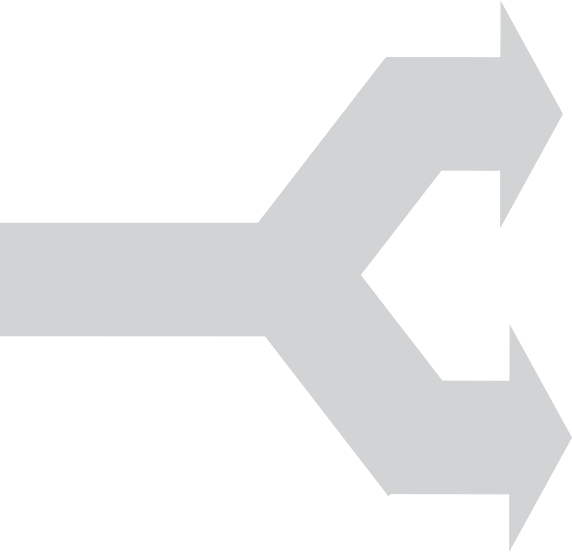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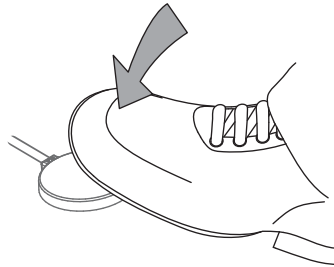Setting the water flow rate


 Never use the handpiece without water flow.

 It is better to set the water flow rate before using the device for the first time while the powder chamber is empty.

Set the water flow rate from dental unit to obtain a small continuous spray.

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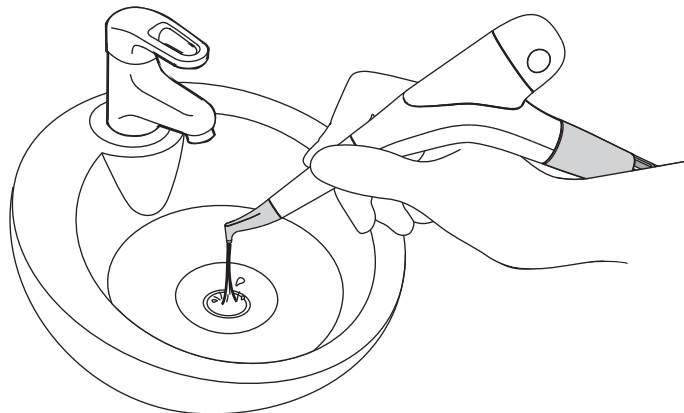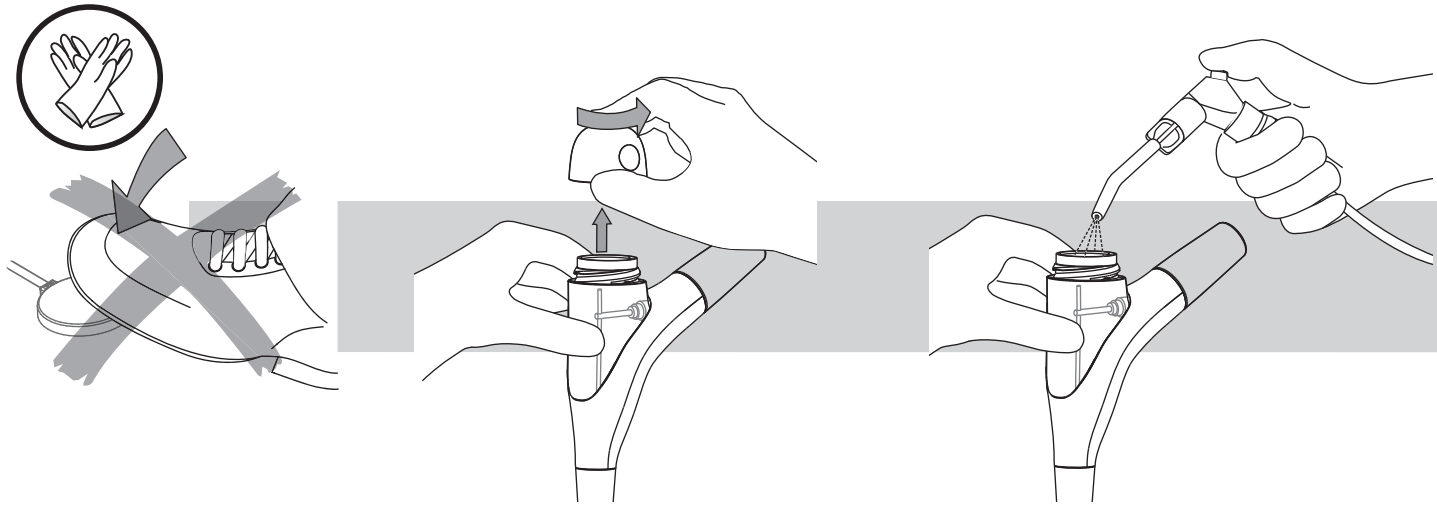
Filling the powder chamber with Easy Fill

Before using this product with the handpiece, please carefully read and follow the recommendations of the instruction manual originally delivered with this product.

Make sure that the powder chamber and the cap of the powder chamber are absolutely dry. Moisture can cause the powder to cake.

Use only powder provided by Hu-Friedy & EMS.

Do not go over the maximum limit as indicated on the picture above. The openings of the tubes should not be covered with powder. The tubes could get clogged.


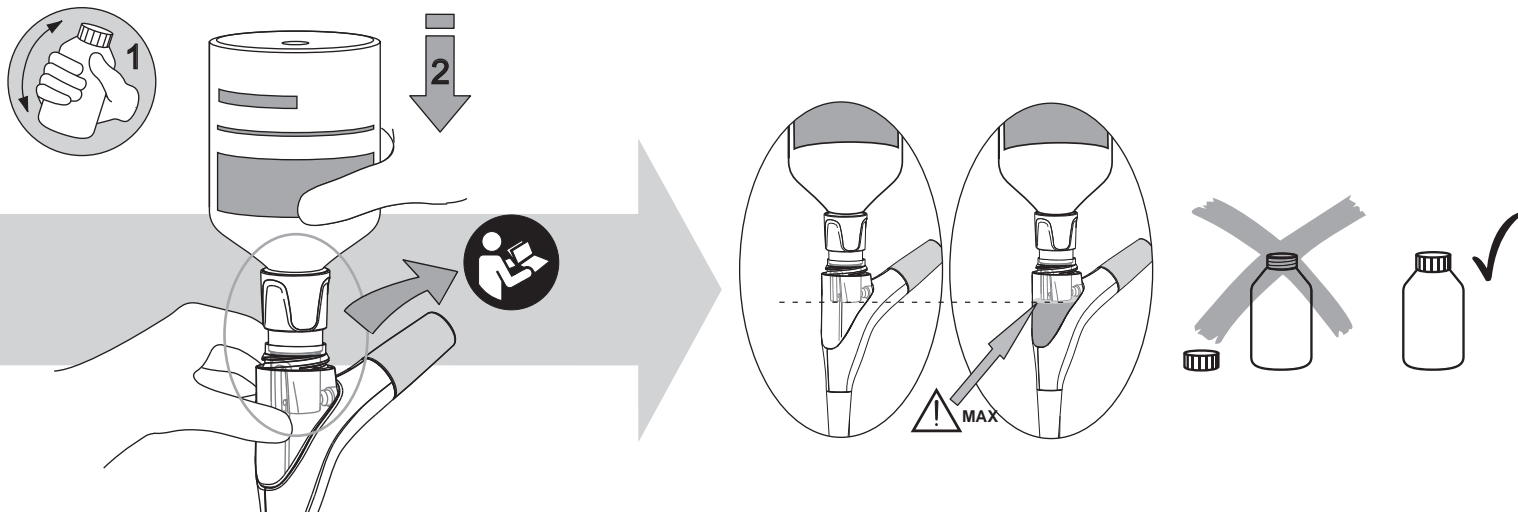
Close the bottle properly to preserve the powder of the humidity.


Press the powder bottle to increase the powder flow.

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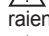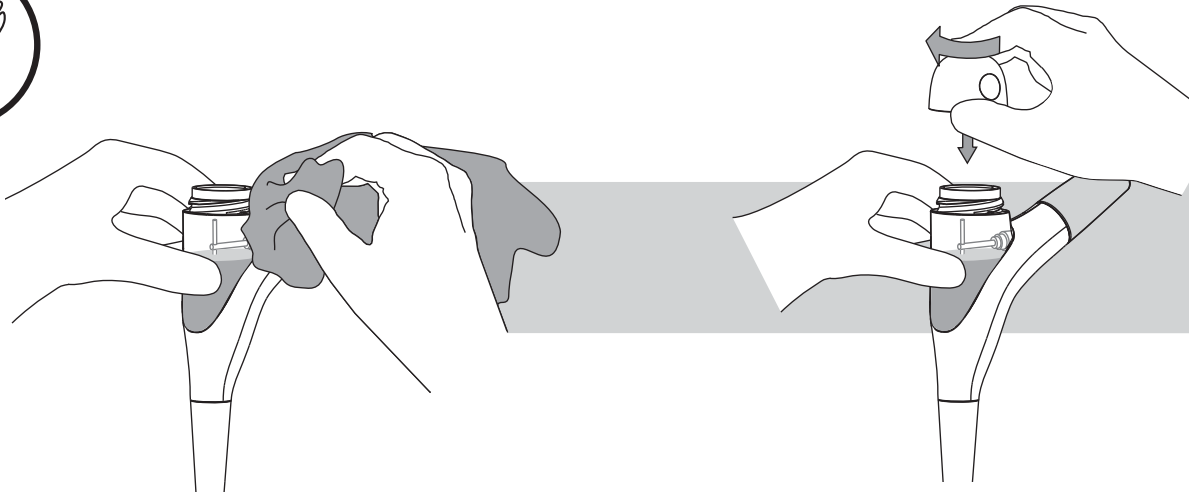Closing the cap

Clean the threads of the powder chamber before screwing on the cap.

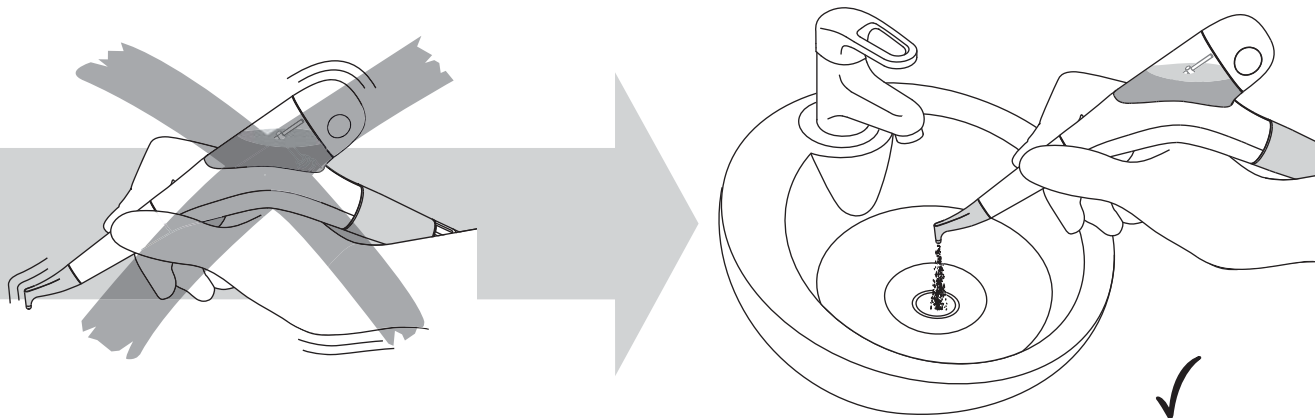
Ensure that the cap is correctly screwed to avoid projection of powder.

Do not shake the device as this could cause the powder to clog the tubes.

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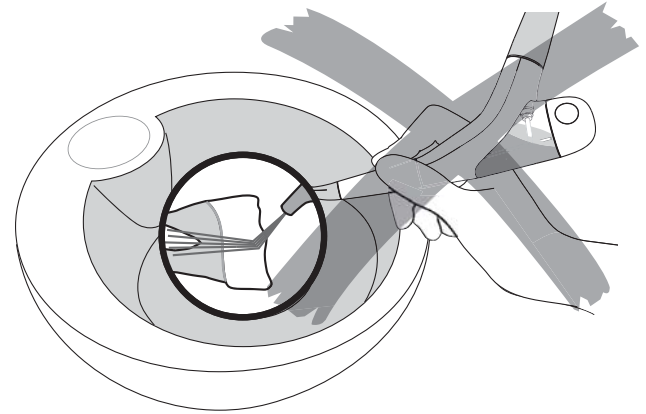
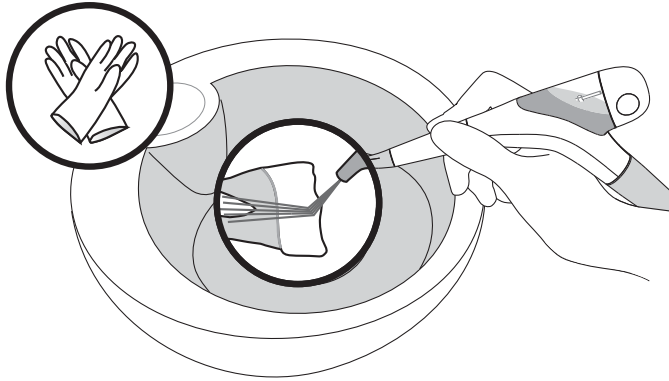
Handling and water/air settings


Familiarize yourself with the use of the unit by cleaning a coin or an extracted tooth.

 You can modulate the result by making adjustments:

- Increasing the air pressure increases the cleaning effect and reduces the polishing effect
- Increasing the water flow rate increases the polishing effect and reduces the cleaning effect.

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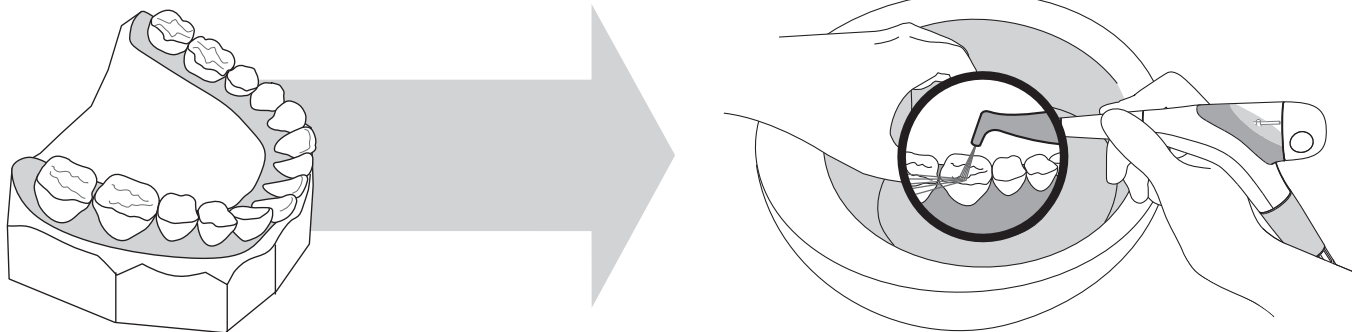PRECAUTIONS

Wear a mask and eye protection

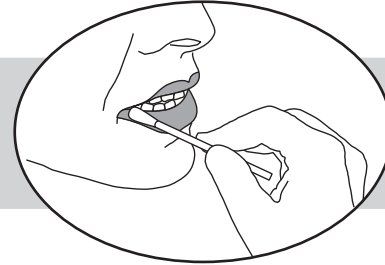
-
-  A powder jet accidentally directed into the eye can cause severe ocular injury. Therefore it is strongly recommended for all persons to wear eye protection during treatment.
 -  To limit the risk of bacterial or viral infections and inhalation of powder, we recommend that the dentist and hygienist wear gloves, protective glasses and a safety mask of protection.
 -  The contact lenses or glasses of the patient may get dirty during the treatment. We recommend to take them off.
 -  The nozzle should never be pointed toward the patient before, during or after operation.

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
Rinsing the patient's mouth before treatment for at least 30 seconds with a cleaning solution will greatly prevent bacteria development during the treatment.

Application of Vaseline® on patient's lips will prevent them from becoming dry and chapped.

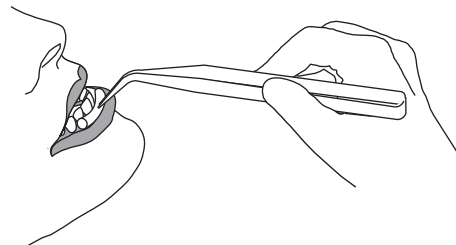The cotton rolls stop the flow of saliva, part the lips and protect the gums.

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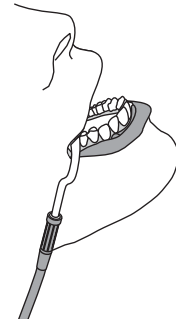
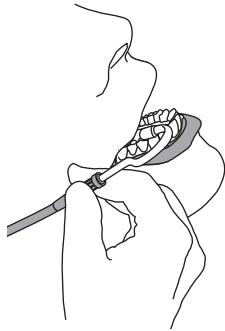
TREATMENT


Position the pump for sublingual aspiration.

 Use the high-speed evacuator of your dental unit to evacuate the air/powder mixture deviated by the treated tooth.

 The handpiece and the high-speed evacuator must be handled by the same operator. The high-speed evacuator is thus directed in an optimal way towards the nozzle.

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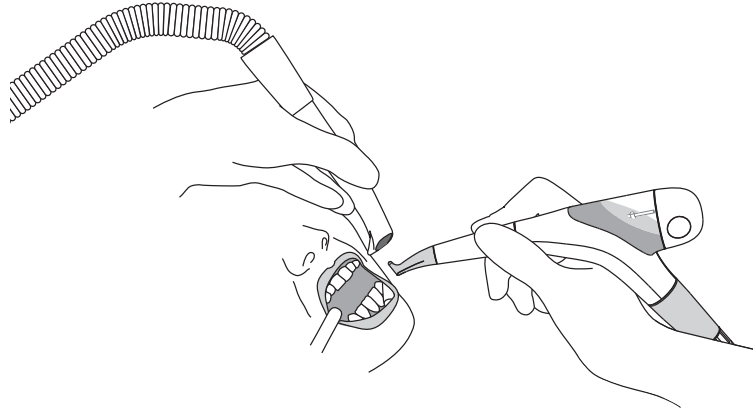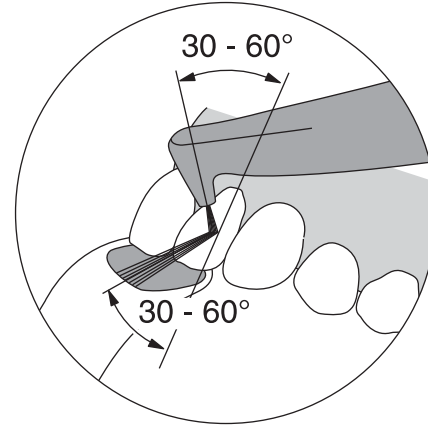
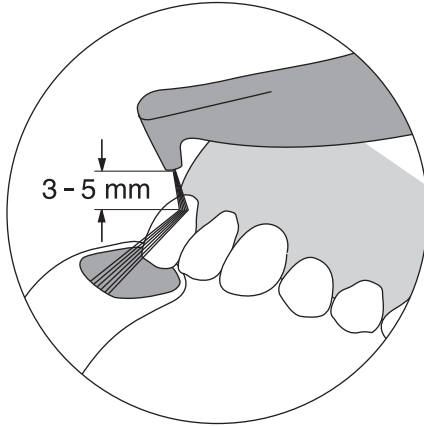
Working Technique

Direct the nozzle strictly towards the surface of the tooth. Respect a distance of 3 to 5 mm.

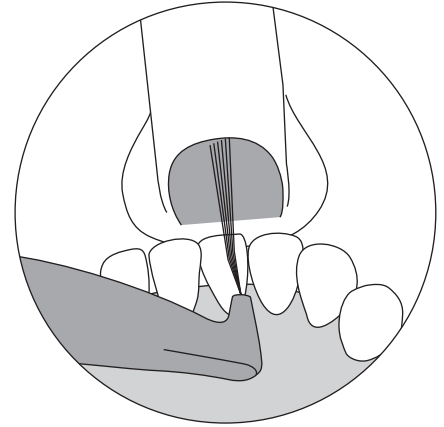
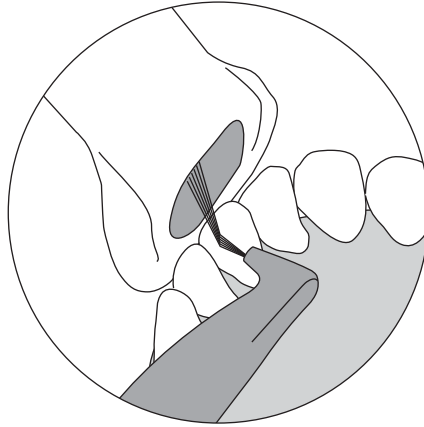
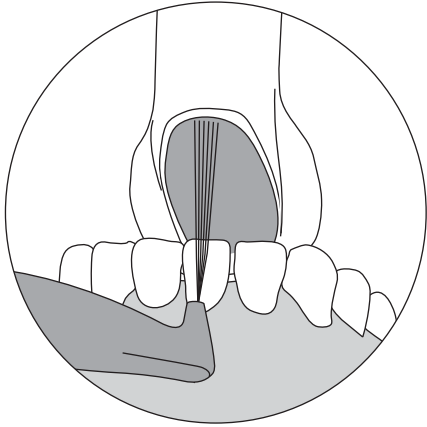
You can vary the angle between nozzle and tooth from 30 to 60 degrees. The flatter the angle, the larger the area cleaned.


During treatment, direct the high-speed pump towards the air/powder jet deviated by the tooth. The angle of reflection is identical to the angle of incidence.

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

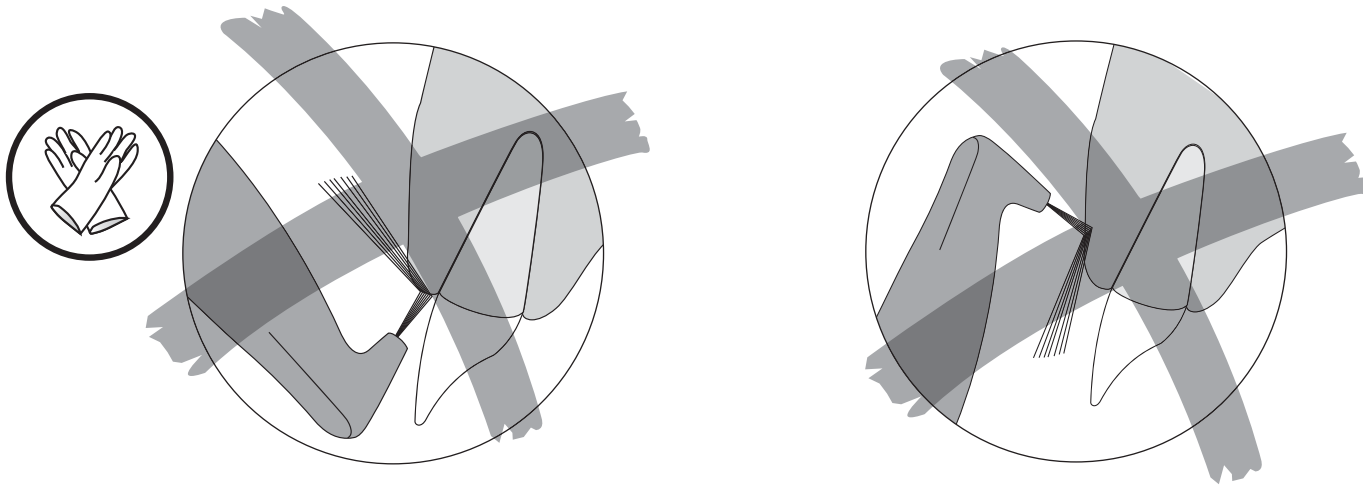
 The air/powder jet is powerful. It can cause injury to the gums or an emphysema caused by the introduction of air into the soft tissue spaces. It is strongly recommended that the operator never direct the nozzle directly at the gum tissue or into the gingival sulcus.

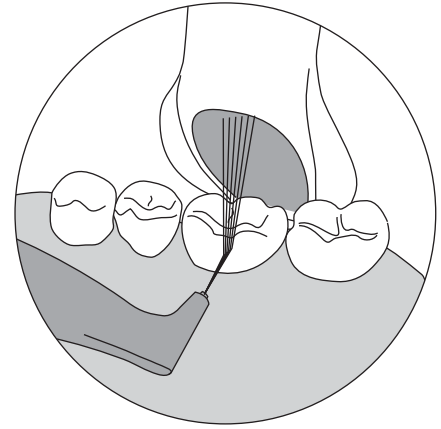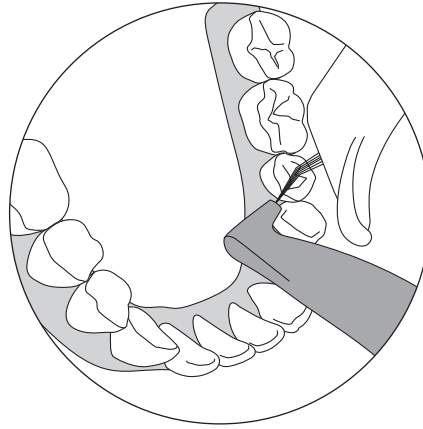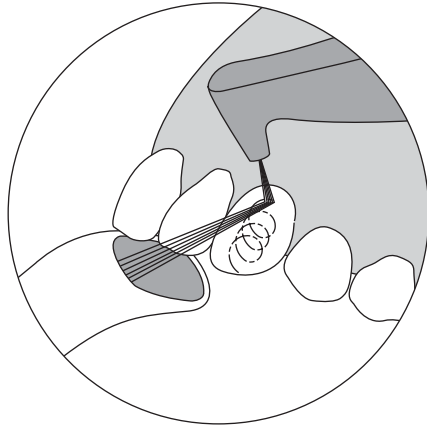 During treatment, make small circular motions.

Finally after the treatment polish all dental surfaces by setting the water flow rate to the maximum.

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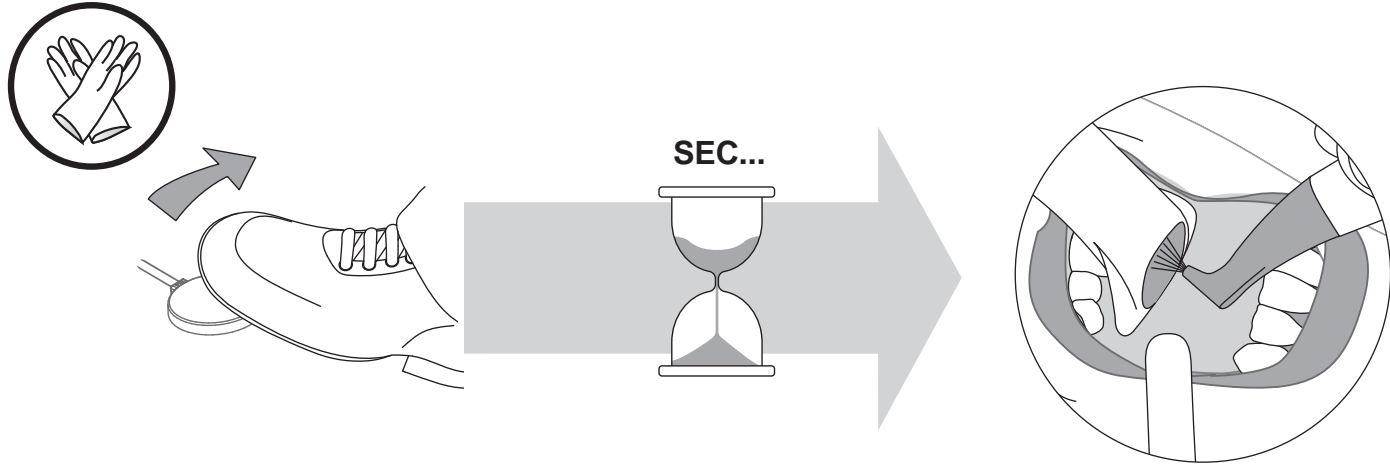
Precautions to be taken at the end of the treatment

 When the foot is lifted off of the foot pedal, the air/powder jet continues for a few more seconds. End the treatment taking this time frame into account.

 The nozzle can be introduced into the high-speed evacuator still in the patient mouth. This allows time for the unit to decompress with no risk of hurting soft tissues.

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Fluoride Application

After completion of treatment, the patient can do a final rinse.

 After the treatment, the teeth are practically free from mucin. It is thus advised to carry out a topical application of fluorine. It is then important to use a colourless fluoride.

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
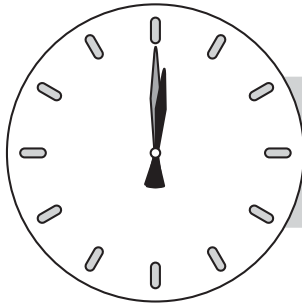
Patient Information

 After the treatment, the teeth are clean and the dental cuticle is completely eliminated. Its reconstitution by salivary proteins requires between 2 and 3 hours. During this time, the teeth do not have any natural protection with respect to staining.

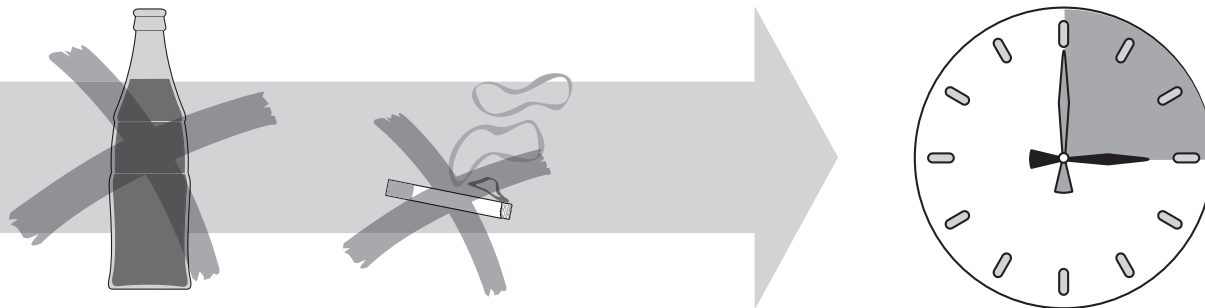 Tell your patient that during 2 to 3 hours following treatment, he should neither smoke, not consume food and drinks containing substances that could strongly colour the teeth (tea, coffee, ...).

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
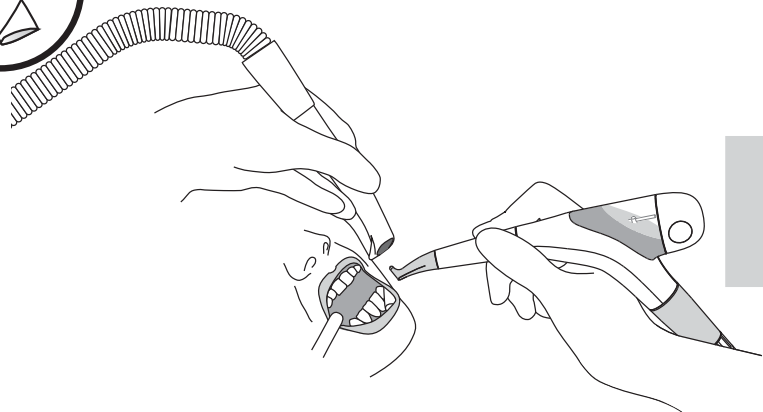
Start the cleaning


MAX.

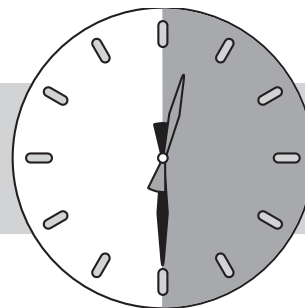 Cleaning must be carried out immediately, no later than 30 minutes after the end of the treatment.

 To easily disconnect the handpiece from the body, rotate the handpiece slightly while pulling in the axis.

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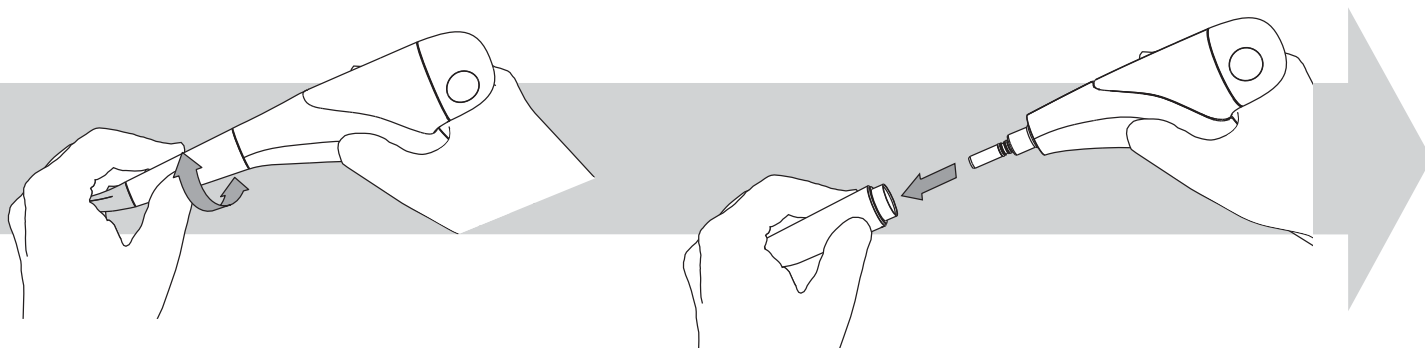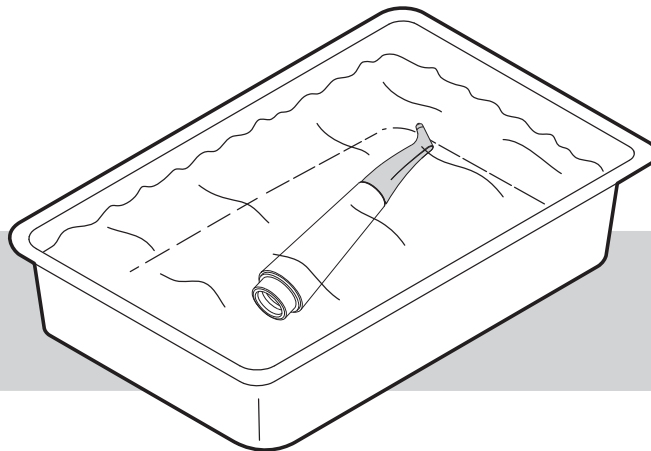
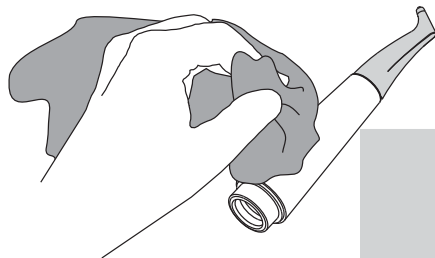
Cleaning the handpiece


Cleaning steps:

- Remove any externally attached soiling by brushing carefully with a soft surface brush or cloth.
- Soak all parts in the cleaning solution for, as a minimum, the time and the concentration specified by the manufacturer of the cleaning agent (ENZOL, 2 %, 10 min).
- Brush all lumens and surfaces with a suitable lumen / surface brush until all visible soiling is removed.
- Rinse through all product lumina (e.g. irrigation and aspiration connection) with the cleaning solution (ENZOL, 2 %), at least 5 times in the flow direction (no back rinsing), using a disposable syringe (min. volume 50 ml).
- Submerge all components completely in deionized water.
- Rinse through all product lumina (e.g. irrigation and aspiration connection) with deionized water at least 3 times in the flow direction (no back rinsing), using a disposable syringe (min. volume 50 ml), using the Easy Clean (if available).
- Repeat the cleaning process if the last rinse does not run clear, or if stains are still visible on the product.
- Dry all components at room temperature.
- Place the handpiece in a sterilization bag.

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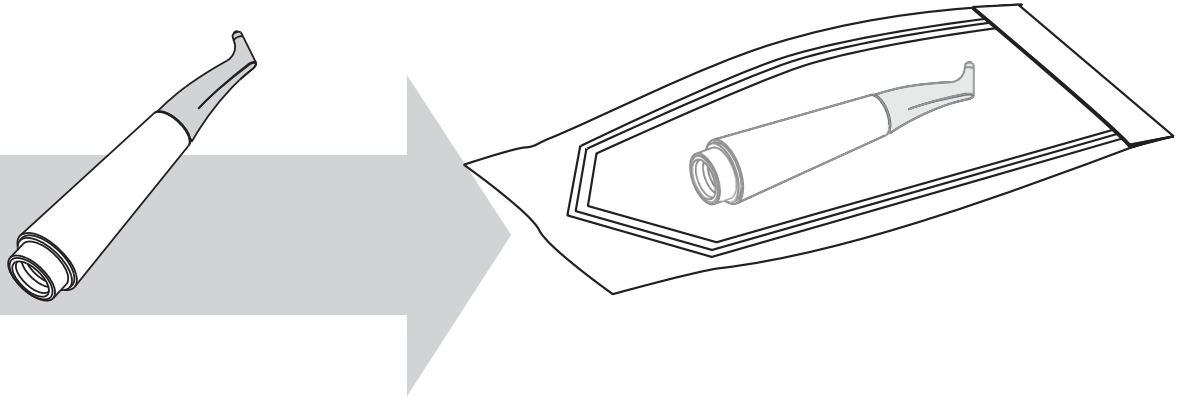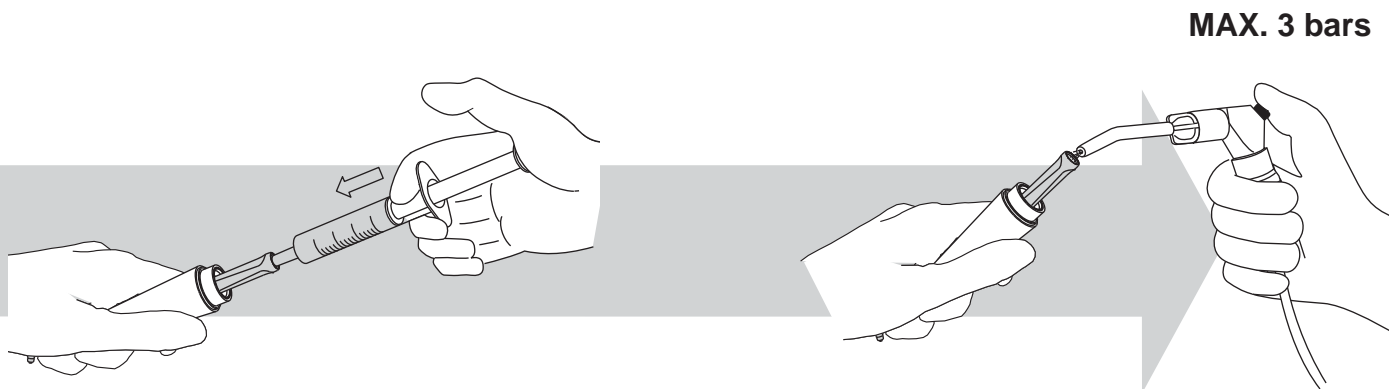
Unclogging / Cleaning the handpiece using the Easy Clean

Clean the handpiece with the Easy Clean supplied with the unit before reprocessing.

-  Rinse through central lumina in the normal direction of flow (no back rinsing) using a 2 ml disposable syringe filled with drink water applied to the purge adaptor. Ideally use distilled or deionized water is used for all rinsing steps.
-  Blow compressed air inside the handpiece to dry it.
-  Clean the handpiece before sterilizing (see the "Cleaning the handpiece" section below).
-  If the handpiece is clogged a light pressure shall be applied. The water passing through the handpiece will dissolve completely the remaining powder inside the lumina and ensure an optimal function.

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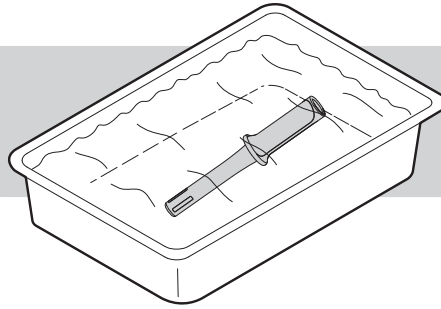
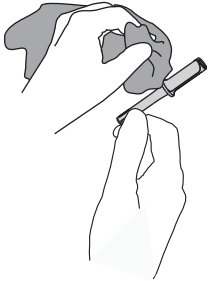
Cleaning the Easy Clean


Cleaning steps:

- Remove any externally attached soiling by brushing carefully with a soft surface brush or cloth.
- Soak the Easy Clean in the cleaning solution for, as a minimum, the time and the concentration specified by the manufacturer of the cleaning agent (ENZOL, 2 %, 10 min).
- Brush the lumen and surface with a suitable lumen / surface brush until all visible soiling is removed.
- Rinse through the lumina with the cleaning solution (ENZOL, 2 %), at least 5 times using a disposable syringe (min. volume 50 ml).
- Submerge the Easy Clean completely in deionized water.
- Rinse through the lumina with deionized water at least 3 times using a disposable syringe (min. volume 50 ml).
- Repeat the cleaning process if the last rinse does not run clear, or if stains are still visible on the product.
- Dry the Easy Clean at room temperature.
- Place the Easy Clean in a sterilization bag.

 The Easy Clean can be cleaned and sterilized in a fractional pre vacuum cycle at 132°C for 4 minutes.

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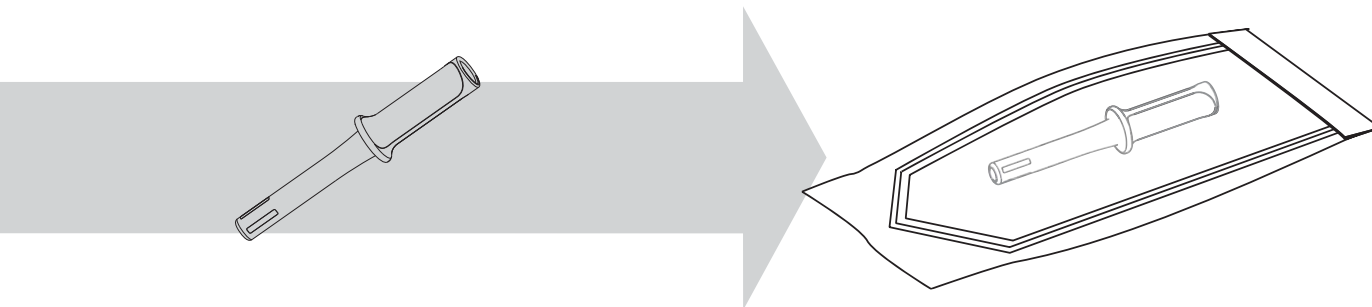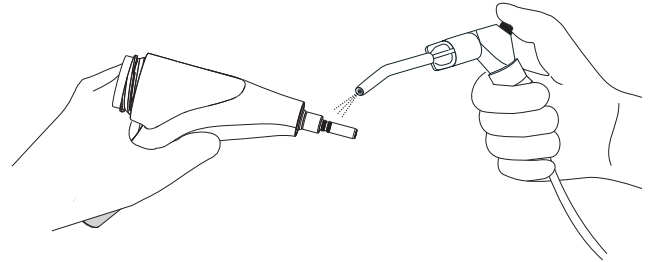
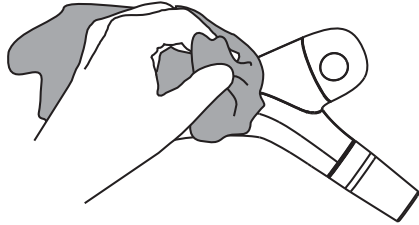
Cleaning and disinfecting the body and body cord




Clean the surface of the body and body cord using a cleaning wipe with proven efficacy (Enzol 2%).

Disinfect the surface of the body and body cord using 70% isopropyl alcohol or other EPA recognized surface disinfectant, following carefully the instructions provided by the disinfection solution manufacturer.

- ⚠ The use of scouring powder or an abrasive sponge will damage its surface.
- ⚠ Do not submerge the body in a cleaning solution bath. It would be likely to be damaged.
- ⚠ Make sure the handpiece connection of the cord is dry.

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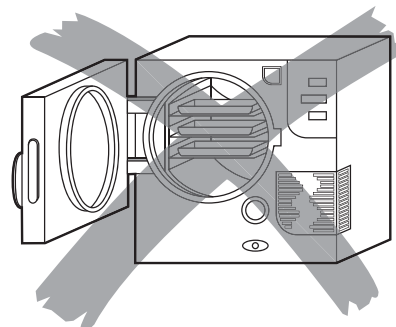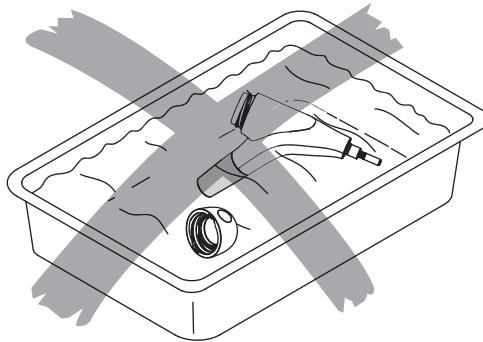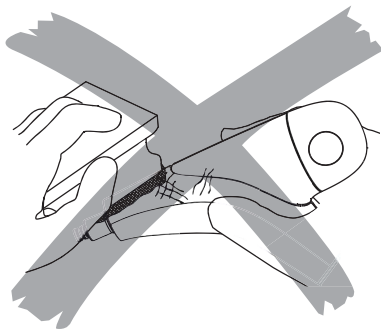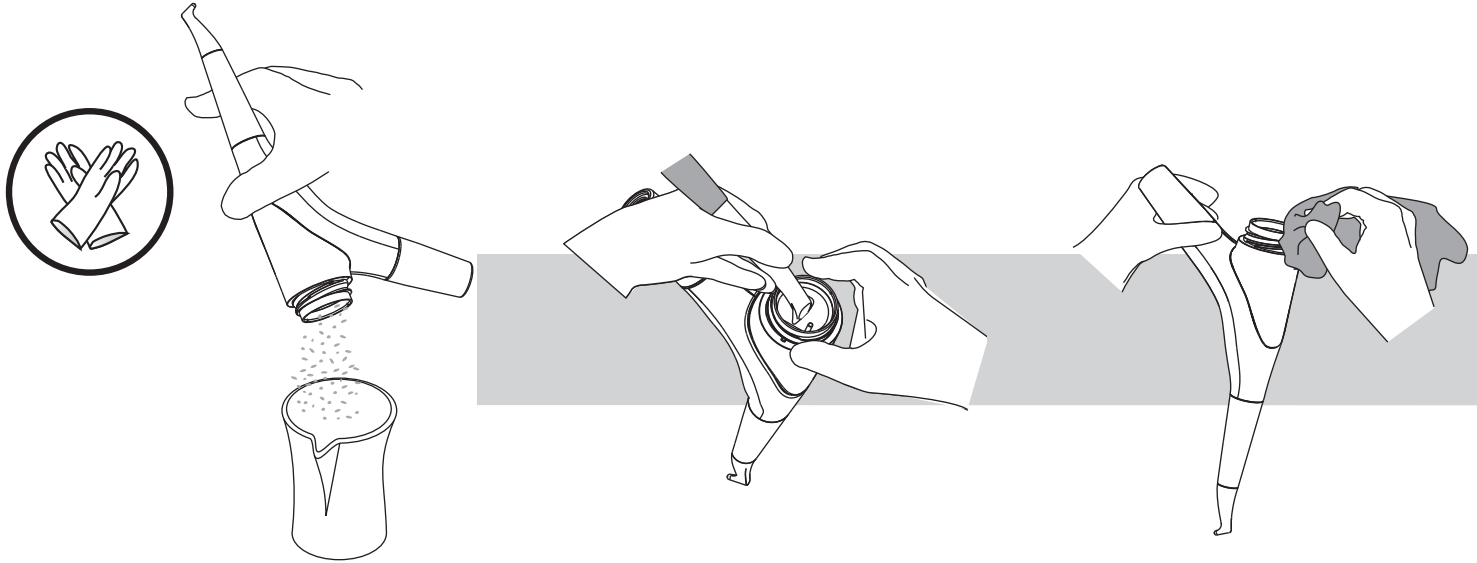
Daily cleaning of the powder chamber


Clean at the end of the work day the powder chamber.

 Empty the powder chamber. Suck out the powder residues with the high-speed evacuator of the dental unit.

Clean the threads of the powder chamber with a cleaning solution (ENZOL, 2 %).

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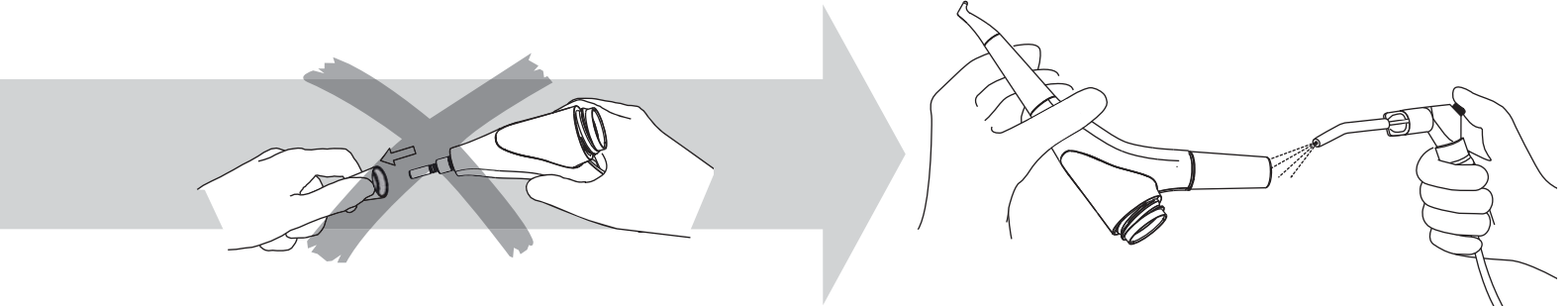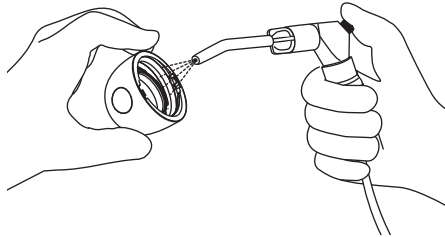
Daily cleaning of the powder chamber cap

Clean the cap at the end of the work day.

The cap must be absolutely dry before closing.

To ensure that the cap fits tightly, the cap ring must be replaced at least once a year.

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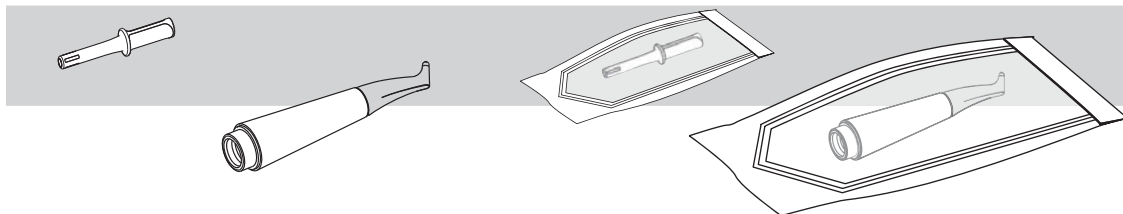
Sterilization of the handpiece and the Easy Clean

Packaging for Sterilization

Refer to standards and validated cycles

 The accessories should be bagged in a sterilization pouch that is validated for use in a 132°C pre-vacuum steam sterilization cycle.

 The sterilization pouch should be sealed according to the manufacturer's instructions.

Sterilization

 Hu-Friedy & EMS recommend sterilization in a fractional pre vacuum cycle at 132°C for 4 min and a drying time of minimum 20 min. Hu-Friedy & EMS recommend use of sterilization pouches, biological indicators and chemical indicators that are legally marketed and have been validated for use in a 132°C 4 minute pre-vacuum steam sterilization cycle.

 Do not exceed the maximum number of sterilization cycles (more information in the technical data section of this manual).

 The instrument may never be heated up >138°C.

 Reprocessing procedures have only limited implications to a medical instrument. The limitation of the numbers of reprocessing procedures is therefore determined by the function / wear of the device.

 The reprocessing must be carried out immediately, no later than 30 minutes after the completion of the operation. Additional information, where necessary, is provided in the respective product-specific usage instructions.

 In case of damage the device should be reprocessed before sending back to the manufacturer for repair.

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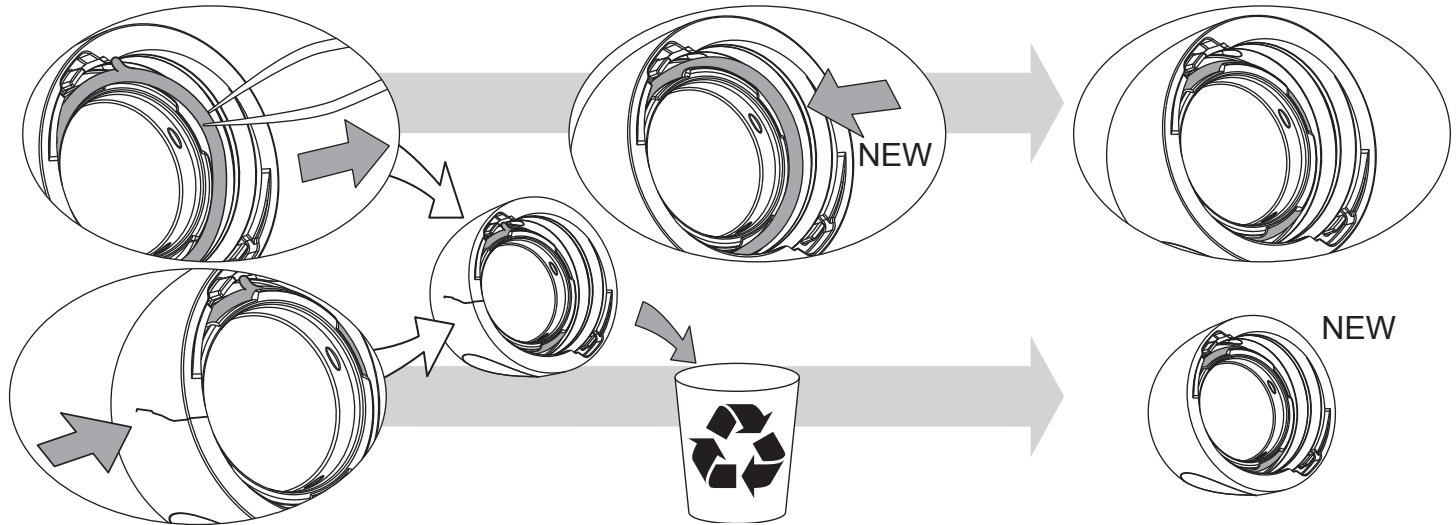
MAINTENANCE

Replacing the cap and the O-rings

 Check the state of the threads of the powder chamber and cap. The powder chamber is pressurized during use. The condition of the powder chamber and cap is an important safety factor.

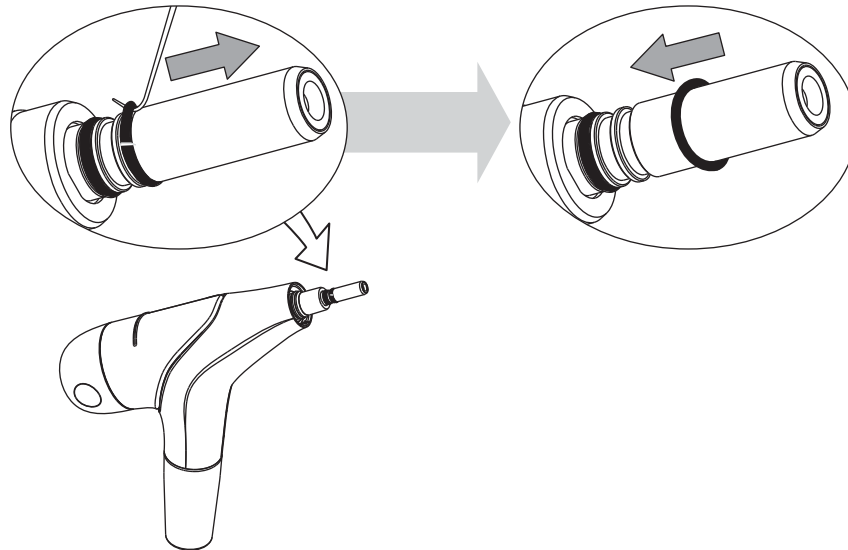
 Replace faulty parts immediately.

 If an O-ring is damaged or worn out, it must be replaced immediately.

Use a non-cutting tool to remove the cap ring.

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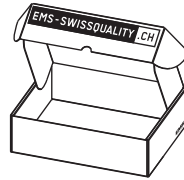
Annual maintenance

To keep your device in a perfect operating condition and to extend its life, send your device each year for an annual maintenance to Hu-Friedy & EMS or an approved Hu-Friedy & EMS repair center.

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SAFETY PRECAUTIONS

Hu-Friedy & EMS and the dealer of this product accept no liability for direct or consequential injury or damage resulting from improper use, arising in particular through non-observance of the operation instructions, or improper preparation and maintenance.

 Use for the intended purpose only. Before using the product, make sure that the manual has been read. This also applies to any equipment used with this product. Failure to observe the operation instructions may result in the patient or user suffering serious injury or the product being damaged, possibly beyond repair.

 Risk of explosion: Do not use this product in the presence of inflammable anaesthetics or gases.

 This product must be used only by trained and qualified personnel (dentist or dental hygienist).

 Before each use, clean and sterilize the different pieces and accessories of this product (nozzles excepted). Please refer to the information provided in the operation instructions. Non-sterile pieces and accessories may cause bacterial or viral infections.

 In order to avoid the use of any inappropriate powders, it is highly recommended to always empty the powder chamber after use.

 The powder containing sodium bicarbonate provided by Hu-Friedy & EMS can be used only for supra-gingival application. For sub-gingival application, please use the specific powders provided by Hu-Friedy & EMS and refer to their operation instructions.

 Do not direct the jet of powder toward fillings, or crown and bridgework as this could damage the restorations.

 Never use "Hu-Friedy & EMS Abrasive Powder" for the treatments carried out with this product.

 Do not use the AIR-FLOW method on severely inflamed tissues.

 The powders provided by Hu-Friedy & EMS are especially designed for use with the unit. Do not use powders from other manufacturers as this could damage the unit or could adversely affect its efficiency.

 Never use "Hu-Friedy & EMS Abrasive Powder" in the unit as this would damage it.

 Always examine the product for damage before starting the treatment. Damaged accessories or a damaged device must not be used and must be replaced. Use original Hu-Friedy & EMS spare parts and accessories only.

 Have this product exclusively repaired by an approved Hu-Friedy & EMS repair center.

 Never use an abrasive powder i.e. silica or alumina based. This would damage the unit.

 The powders provided by Hu-Friedy & EMS are especially designed for use with the unit. Do not use powders from other manufacturers as this could damage the unit or could adversely affect its efficiency.

 For more details about the Hu-Friedy & EMS powder, please read the relative operation instructions.

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STORING THE PRODUCT

 Keep the original packaging until the product is to be disposed of permanently. You can use it for shipping or storing at any time.

Should you wish to store the product for an extended period of time:

- Proceed as described in the chapter "Cleaning and sterilizing".
- Pack the product and all accessories in the original packaging.

Please refer to the storage and transport conditions in the "Technical data".

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PRODUCT DISPOSAL

The product shall not be discarded in domestic household waste.

Should you wish to definitively dispose of the product, please comply with the regulations which apply in your country.

WARRANTY

Damages due to non-adherence to the operating instructions or wear out of parts are excluded from warranty.

The warranty of your product will be cancelled if you try to open it.

ACCESSORIES

Accessories are available from Hu-Friedy & EMS or any authorized dealers. Please contact your customer service directly.

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HU-FRIEDY & EMS-SERVICE

Should your product need additional servicing or repairs, please send it to your dealer or our approved Hu-Friedy & EMS repair center.

In the case of non-authorized repairs or damaged due to non-adherence to the operating instructions, Hu-Friedy & EMS accept no liability whatsoever. This will also void the warranty.

It is best to ship your product in the original packaging. It protects your product against damage during shipment.

 Before dispatching your product including all accessories, please clean and sterilize as described in the operating instructions.

 The connector must be dismantled or changed only by an approved Hu-Friedy & EMS repair center.

 You can ask for a modification of the type of adapter of the apparatus, in the range available near an approved Hu-Friedy & EMS repair center only.

When sending your product directly to the approved Hu-Friedy & EMS repair center, please include the name of your dealer. This simplifies processing for us.

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SYMBOLS

	Manufacturer's logo
	Distributor's logo
	Caution!
	The handpiece should not be exposed to temperatures >135°C
	Serial number
	Catalogue number
	Lot number
	Manufacturer
	Manufacture date

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	Thermal cleaning
	Refer to instruction manual
	CE marking: Refers to directive 93/42 EEC, including EN 60601-1 and EN 60601-1-2

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TECHNICAL DATA

Description

Manufacturer	EMS SA, CH-1260 Nyon, Switzerland
Model	AIR-FLOW handy 3.0
Classification 93 / 42 EEC	Class IIa
Mode	Continuous operation
Service Pressure to the turbine connection : Water	1 to 2.2 bar (1000-2200 hPa) with a service flow of 50-80 ml/min.
Service Pressure to the turbine connection : Air	Static pressure 2.7 to 3.5 bar (2700-3500 hPa)
Weight	Approx. 0.150 kg
Operating conditions	+10°C to +40°C 30% to 75% relative humidity 700 hPa to 1060 hPa air pressure Altitude max. 3000 m
Validated life time of the handpiece	800 sterilization cycles
Storage conditions	+10°C to +40°C 5% to 85% relative humidity 500 hPa to 1060 hPa air pressure
Transport conditions	-10°C to +40°C (ventilated) or 60°C (non ventilated) 75% relative humidity min. 700 hPa air pressure

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TROUBLESHOOTING

Type of problem	Solutions
Water comes into the powder chamber or leaks from the cap	<ul style="list-style-type: none"> • Check the dental unit connection • Check the state of the o-rings of the turbine connection • Connect the unit to the dental unit (be careful, the powder chamber must be empty) • Dry and fill the powder chamber • Send the complete unit to an approved Hu-Friedy & EMS repair center
No powder/air jet coming from the unit	<ul style="list-style-type: none"> • Cut immediately the air input by releasing the foot pedal • Wait for 1-2 minutes for the system to decompress • Disconnect the unit from the dental unit • Press the foot control pedal of the dental unit • If no air comes out of the turbine connector, it is your dental unit that causes the problem • If air comes out of the turbine connector, it is the device that causes the problem • If air comes through the handpiece connector, the handpiece is blocked. • Clean the handpiece with the EasyClean • Unscrew the cap above the wash-hand basin, because the powder can be ejected. • A sealed apparatus, even disconnected, can indeed remain under pressure • Empty the powder chamber and screw the cap back on • Connect the unit to the dental unit (be careful, the powder chamber must be empty) • Press the foot control pedal of the dental unit • If no air comes out of the handpiece connector, the unit is blocked • Send the complete unit to an approved Hu-Friedy & EMS repair center
Air and/or powder is leaking from the threads of the cap	<ul style="list-style-type: none"> • Check the seal and the cleanliness of the thread on the powder chamber and on the cap • Replace the seal or the cap if necessary • Send the device to a certified Hu-Friedy & EMS Repair Center
The unit efficiency diminishes	<ul style="list-style-type: none"> • A powder refill is perhaps necessary • Clean the handpiece • Send the device to a certified Hu-Friedy & EMS Repair Center
Water is leaking between the handpiece and the unit	<ul style="list-style-type: none"> • Replace the seal of the handpiece connector • Send the complete unit to an approved Hu-Friedy & EMS repair center
Air is leaking between the handpiece and the unit	<ul style="list-style-type: none"> • Clean the handpiece • Replace the seal of the handpiece connector • Send the complete unit to an approved Hu-Friedy & EMS repair center
Air and/or water is leaking from the connector of the cord	<ul style="list-style-type: none"> • Check the connector of the cord, if it is necessary, carry out maintenance following the information of the manufacturer • Send the complete unit to an approved Hu-Friedy & EMS repair center

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Hu-Friedy Mfg. Co., LLC
EMS Electro Medical Systems S.A.