


# COLOR CHANGE GUIDE


Below is a color change guide to aid in the interpretation of the color change that occurs when the pouches are processed in an autoclave, or ethylene oxide sterilization chamber.

Ideally, the indicator should change color from the first box to the last box, but due to the large variation in sterilization chambers, this is not always the case. Therefore, there is a suggested acceptable range of color change. See diagrams below for details.

## STEAM STERILIZATION

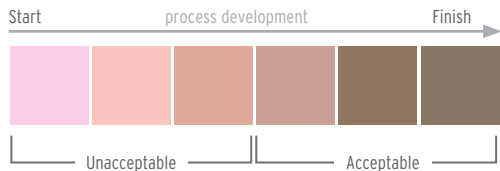

## EO STERILIZATION

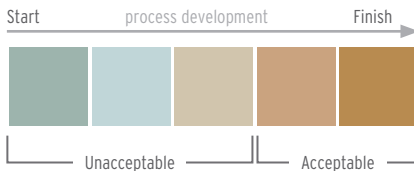

Caution: the chemical indicators are to be used as a sterilization process indicator only. The indicators should be used in conjunction with biological indicators (spore strips) to ensure the devices within the pouches are sterile. For more information on Hu-Friedy SporeCheck 24 hour in-office biological monitor system please visit our website.

How the best perform

