



HU-FRIEDY SURGICAL KIT

LATERAL SINUS LIFT KIT, BLACK LINE



PART CODE	DESCRIPTION
IMN4162	16 Instrument Infinity Series™ Cassette, White
IMP6578SX	Kramer-Nevins Sinus Lift Instr., 47-Degree, Black Line
IMP6577SCX	Kramer-Nevins Sinus Lift Instr., Strong Curve, Black Line
BPACK	Bone Packer DE 6-8mm
SYBG	Syringe Bone Grafting 7mm
BONT	Bone Tamper 7mm
NHCW	Crile-Wood Needle Holder
S16SCX	16 Goldman-Fox Scissors, Black Line
CRURBAN	Urban Cheek Retractor
IM1002	6-Pack Signature Series® Instrument Clip
IMNLATKITX	Complete Lateral Sinus Lift Kit, Black Line

LATERAL SINUS LIFT KIT, BLACK LINE

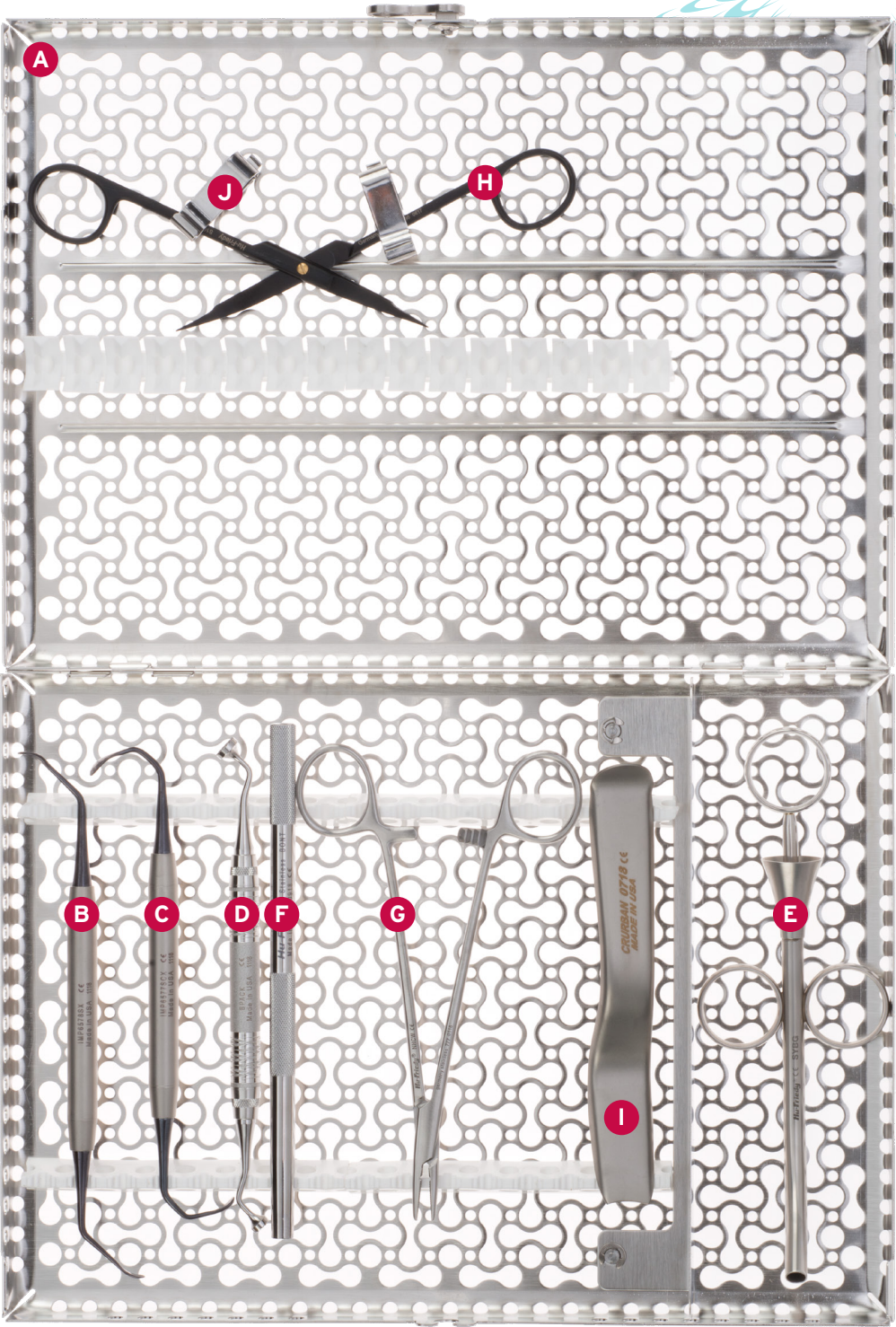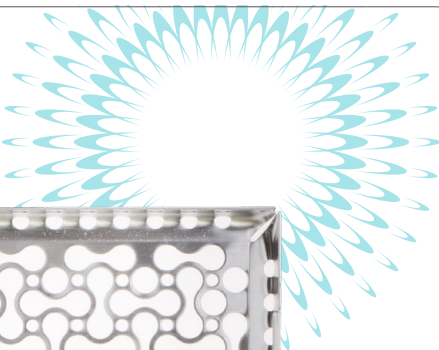

LATERAL SINUS LIFT KIT, BLACK LINE



A IM4162
16 Instrument Infinity Series™ Cassette, White

IMS® cassettes are made of long-lasting, low-maintenance stainless steel. Soft, colored silicone rails securely hold instruments throughout the processing cycle. Available in over 100 different sizes, configurations and color options, and can be specialized to meet your needs.



B IMP6578SX
Kramer-Nevins Sinus Lift Instr., 47-Degree, Black Line

Used during a sinus lift procedure to separate/reflect the Schneiderian membrane from the maxillary bone and to elevate the membrane. With obtuse angles 4.1mm/7.6mm.



C IMP6577SCX
Kramer-Nevins Sinus Lift Instr., Strong Curve, Black Line

Used during a sinus lift procedure to separate/reflect the Schneiderian membrane from the maxillary bone and to elevate the membrane. With acute angles 4.1mm/7.6mm.



D BPACK
Bone Packer DE 6-8mm

Used to condense bone during grafting procedures with working ends 6mm and 8mm wide.



E SYBG
Syringe Bone Grafting 7mm

Used during grafting procedures. Inner Diameter 5.8 mm. Load from tip to prevent clogging.



F BONT
Bone Tamper 7mm

Used to separate the Schneiderian membrane from surrounding bone during lateral sinus lift procedures.



G NHCW
Crile-Wood Needle Holder

Used to grasp and guide the needle when suturing. Recommended suture sizes: 3-0, 4-0, 5-0, 6-0



H S16SCX
16 Goldman-Fox Scissors, Black Line

Used for trimming tissue or cutting sutures. Featuring a non-reflective surface, improved contrast between tissue/scissor. 13.5cm.



I CRURBAN
Urban Cheek Retractor

Used to hold mucoperiosteal flaps, cheeks, lips and tongue away from the surgical area. 20.6mm



J IM1002
6-Pack Signature Series® Instrument Clip

Secure and maintain open position of scissors, hemostats, and other hinged instruments in lid of cassette.