

INSERT WEAR GUIDE

Worn tips can diminish your scaling efficiency. Check your inserts **every week** to maintain optimal performance.

How the best perform

Hu-Friedy

■ 25% Efficiency Loss (Reorder)

■ 50% Efficiency Loss (Discard)

#3 Beavertail

Original Proply Design

After Five™ Right Low - Medium

After Five™ Straight Low - Medium

After Five™ Left Low - Medium

#10 Universal Low - High

#100 Thin Low - Medium

#1000 Triple Bend Low - High

XT 25kHz Low
30kHz Low - Medium

XT Triple Bend Low - Medium

Power Plus™ Standard

Power Plus™ Thin

Power Plus™ Thin Beveled

Power Plus™ Standard Beveled

HOW TO USE YOUR WEAR GUIDE

1. Locate the diagram of the insert to be measured (use color as guide).
2. Position point **A** to point **B** as shown and/or align the curvature of the tip (circled) to match the guide.
3. Rotate insert until tip is flat against the card.
4. Evaluate tip condition, comparing length to red/blue bars.

Reorder tips when 25% efficiency loss (blue bar)


Discard tip when 50% efficiency loss (red bar)



Learn more at [Hu-Friedy.com/Inserts](https://www.hu-friedy.com/inserts)

To reorder inserts call **1-800-Hu-Friedy** or your authorized Hu-Friedy dealer.

©2020 Hu-Friedy Mfg. Co., LLC. All rights reserved. All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC its affiliates or related companies, unless otherwise noted. HF-466/0620

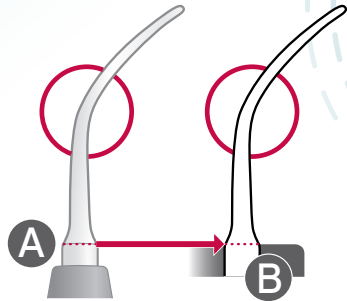

Hu-Friedy