## Instructions for Use

### Giles de Quincey Bone Mill

All Giles de Quincey Bone Mill components are to be cleaned and sterilized prior to each use.

#### General Information:

All assembled instruments must be disassembled prior to reprocessing. The Giles de Quincey Bone Mill has six components when disassembled: the body, two screw caps, one static grinder, one rotating grinder and one handle. No component of the bone mill should be substituted by a product from another manufacturer, doing so may damage the device.

#### Assembly of the GDQ Bone Mill:

Install the rotating grinder (5) by inserting its shaft into the concavity (6) of the body. Once the grinder shaft is placed, assemble the handle (3) to the rotating grinder shaft followed by a screw cap (2). Rotate the screw cap (2) clockwise to tighten.

Next install the static grinder (4) by inserting it into the hole on the body (1). Tighten with the screw cap (2) by rotating it in a clockwise position.

#### Usage:

Insert the bone into the concave end (6) of the bone mill. Close the bone mill by applying pressure to the body handle (7). Once closed, turn the handle (3) in either direction until the bone is ground to the desired particle size. Remove the bone by reducing the force applied to the body handle (7), opening the grinder allowing the chips to fall into the designated cup/bowl.

#### Clean-Up:

Remove any possible leftover bone particles with a fine soft bristled brush, example: a toothbrush or cleaning brush. To reduce the risk of damaging the grinding plates, no stainless steel or other hard metal brushes should be used. Upon removal of all bone particles, disassemble and sterilize.

#### Disassembly of the GDQ Bone Mill:

##### 1. Static Grinder:

- Remove the static grinder (4) by loosening the screw cap (2) in a counterclockwise direction.
- Once loosened, remove the screw (2) from the static grinder (4) and disassemble from the body (1).

##### 2. Rotating Grinder:

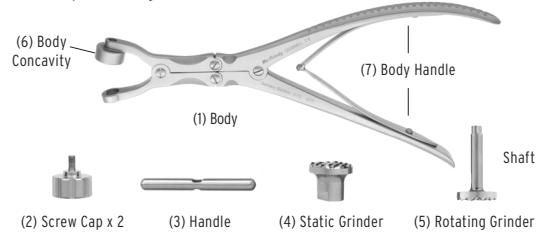
- Loosen the screw cap (2) on the handle (3) shaft by turning it in a counterclockwise direction.
- Once loosened, completely remove, pull apart the handle (3) from the rotating grinder (5) and slide the rotating grinder (5) out of the body concavity (6).

#### Sterilization:


For intended use only by dental professionals. Inspect, clean and sterilize before each use. Steam sterilize for at least 4 minutes at 270°F/132°C or

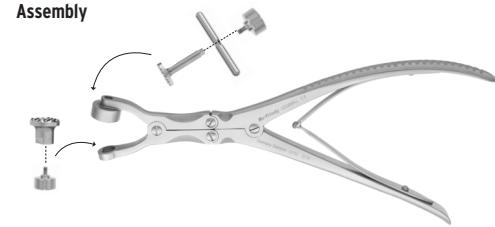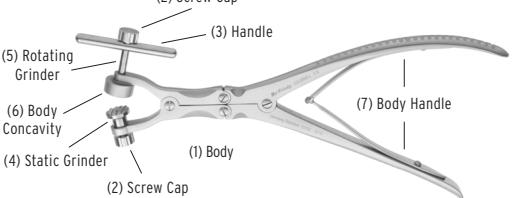
30 minutes at 250°F/121°C. Recommend 30-minute dry time after sterilization cycle. Do not heat above 350°F/177°C. Use recommended chemicals only. Do not disinfect with phenols.

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Hu-Friedy Mfg. Co., LLC:
Hu-Friedy Mfg. Co. LLC