
VistaPure

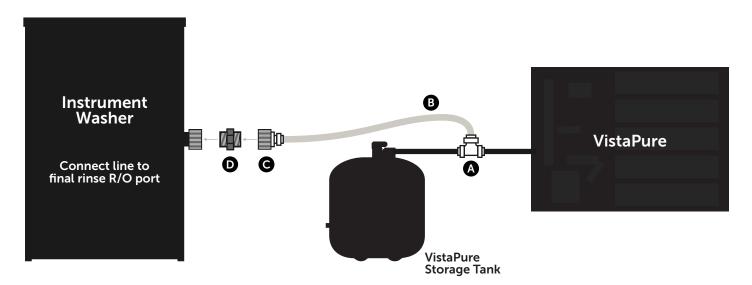

- A 1/4" x 1/4" x 3/8" Tee
- C Female Garden Hose x 3/8" Tube Adapter
- B 3/8" OD LLDPE Tubing (6')
- Male x Male Garden Hose Adapter
- 1 Turn the main city water feed valve to the VistaPure system board off
- 2 Close the elbow valve on the water storage tank
- Relieve water pressure from the system tubing by pressing the button on the autoclave wand or lifting up on the black handle located on the chrome faucet
- Install the 1/4" x 1/4" x 3/8" "tee" anywhere in the 1/4" natural line that runs from the VistaPure system manifold to the water storage tank (see illustration). Simply cut the line with a tubing cutter or sharp blade and install the "tee" by firmly pushing the 1/4" tubing into the 1/4"quick-connect fittings.
- Push one end of the provided 3/8" natural tubing into the 3/8" quick-connect fitting on the "tee." Push firmly.
- Push the other end of the 3/8" natural tubing into the provided black 3/8" quick-connect x female garden hose adapter. Push firmly.

- 7 Thread one end of the provided male x male garden hose adapter into the female end of the black adapter.
- 8 Thread the other end of the male x male garden hose adapter into the female fitting on the final rinse port of the instrument washer.
- 9 Double check all connections and consult the provided installation illustration to verify everything is installed correctly.
- Open the tank valve. Open the main city water feed valve to the VistaPure and return the black handle on the chrome faucet to its normal closed position. Check all connections for leaks.
- Consult the operator's manual for the instrument washer to initialize or reprogram the system to engage the final rinse port.

Vista Research Group

VistaResearchGroup.com

VistaPure[™] S3010 Adapter Kit