

Sparkle™ Air/Water Syringe Tips

REF BCSAWSCL / BCSAWSWH / BCSAWS / BCSAWSB

INSTRUCTIONS FOR USE

No Adapter or converter needed with original equipment

1. On most nut style syringes, loosen the syringe tip locking nut and insert the Sparkle™ Air/Water Syringe Tip the same as you would a metal tip.
2. On the quick-change syringe, press in the outside ring and insert the Sparkle Air/Water Syringe Tip the same as you would a metal tip.
3. Before using the syringe, make sure the Sparkle Air/Water Syringe Tip is firmly seated in the syringe. The Sparkle Air/Water Syringe Tip is properly installed when no lines or grooves are visible beyond the syringe tip locking nut.




Sparkle Air/Water Syringe Tips are packaged and sealed in a clean environment (not sterile).

CROSSTEX
A CANTREL MEDICAL COMPANY



Crosstex International, Inc., a Cantel Medical Company
6789 W. Henrietta Road, Rush, NY 14543 USA
585.359.0130 | 631.582.6777 | crosstex.com

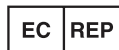

MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany



crosstex.com

Made in China

BCSAWSB-IFU-EN / REV B - 05/2021

All product names are trademarks of Crosstex International, Inc., a Cantel Medical Company, its affiliates or related companies, unless otherwise noted.
©2021 Crosstex International, Inc. All rights reserved.

Not made with natural rubber latex.