Autoclave Monitor Tape

.75 in x 60 yd / 19 mm x 54 m

EN

INSTRUCTIONS FOR USE

Product Description

The HuFriedyGroup Autoclave Monitor Tape is designed for use with steam sterilization as a method of securing packs and identifying processed from unprocessed items.

Intended Use

The HuFriedyGroup Autoclave Monitor Tape is designed to demonstrate that the pack has been exposed to a sterilization process and to distinguish between processed and unprocessed packs. After steam sterilization, the chemical indicator lines will show a visual color change from off yellow to dark brown/black.

Use the HuFriedyGroup Autoclave Monitor Tape with the following steam sterilization cycles.

1. 121°C, 30 Minutes; Gravity Steam
2. 132°C, 4 Minutes; Dynamic Air Removal Steam
3. 135°C, 3 Minutes; Dynamic Air Removal Steam

Instructions for Use

When using the HuFriedyGroup Autoclave Monitor Tape to secure sterilization wraps it is recommended to:

1. Remove the product label and plastic packaging and install the tape roll in a compatible tape dispenser (HuFriedyGroup Tape Dispensers IMS-1270 or IMS-1270H).
2. Unwind a section of Autoclave Monitor Tape long enough to secure the pack.
3. Place one end of the Autoclave Monitor Tape on the package and apply finger pressure across the length of the tape to secure the pack. Do not stretch the tape.
4. Repeat steps 2 and 3 if the method of pack wrapping / size requires two or more strips of indicator tape for secure closure. Do not allow any openings in the folds.
5. When the steam sterilization cycle is complete, the chemical indicator lines will show a visual color change from yellow to dark brown/black.
6. When the sterile pack is ready for use, open by using aseptic technique to avoid contaminating the internal contents.

Shelf-Life:

Unused: 24 Months from date of manufacture.

Available Options:

Part Code	Procedure Color
IMS-1253	Cement
IMS-1254	Composite
IMS-1255	Endo
IMS-1256	Exam
IMS-1257	Extraction
IMS-1258	Perio
IMS-1259	Prophy
IMS-1260	Pedo
IMS-1262	Gray
IMS-1263	Orange
IMS-1264	Purple


Part Code	Procedure Color
IMS-1265	Yellow
IMS-1266	Red
IMS-1267	Green
IMS-1268	Blue
IMS-1269	Black
IMS-1243	Orthodontic
IMS-1247	Restorative
IMS-1248	Surgery
IMS-1251	Amalgam
IMS-1252	C and B

ISO 11140-1 Type 1

hu-friedy.com/compliance-sheets

Storage Conditions

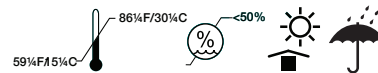Do not freeze.

Keep away from sterilants and excessive heat.

HuFriedy

Hu-Friedy Mfg. Co., LLC
3232 N. Rockwell Street
Chicago, IL 60618 | USA

Hu-Friedy Europe LLC & Co. KG
Kleines Öschle 8
78532 Tuttlingen
Germany

In case of any serious incident that has occurred and involves our devices within the European Union, please report it to Hu-Friedy Mfg. Co., LLC and your national competent authority of the member state.

All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC its affiliates or related companies, unless otherwise noted.

©2021 Hu-Friedy Mfg. Co., LLC. All rights reserved.

Made in China
N4838/0821

HuFriedyGroup
The Best In Practice

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©2021 Hu-Friedy Mfg. Co., LLC. Todos los derechos reservados.