


# Crosstex™ Isolator™ Duckbill Surgical N95 Respirator

REF GDBRESP

## INSTRUCTIONS FOR USE

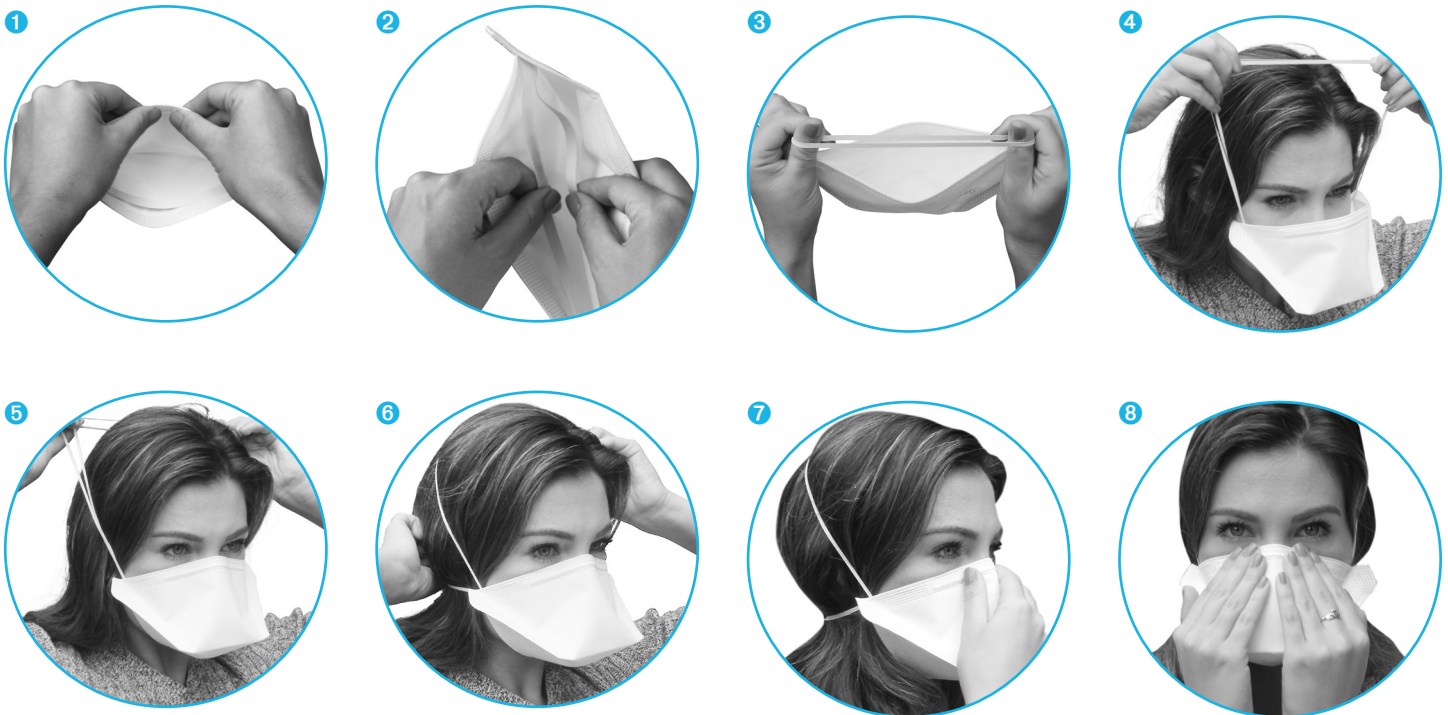

1. Unwrap respirator and bend nosepiece into a curve to open the respirator.
2. Separate the pre-scored elastic headband into two individual bands.
3. Hold the headbands with your index fingers and thumbs.
4. Cup the open respirator **securely** under your chin.
5. Pull both headbands over your head.
6. Rest the top headband on the crown of your head and bring the bottom one down under your ears.
7. Press the nosepiece around the contour of your nose and cheekbones. Adjust the edges of the respirator to make sure there are no gaps.
8. Perform a seal check by gently cupping both hands around the respirator, then exhale sharply to see if you sense or detect any leaks. If leaks are detected, reposition the respirator and/or adjust the straps. If leaks are still detected, then do not enter contaminated area and seek supervisor for further instructions.

**WARNING:** A respirator does not eliminate the risk of contracting any disease or infection. Occupational use of this respirator requires that you be fit tested by a qualitative or quantitative fit test and that your employer comply with OSHA 29 CFR 1910.134, which includes medical evaluation, training and fit testing.

**CROSSTEX**

Distributed by: Crosstex International, Inc.  
6789 W. Henrietta Road, Rush, NY 14543 USA  
585.359.0130 | 631.582.6777 | crosstex.com

Made in USA

GDBRESP-IFU-EN / REV A - 03/2022

All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC, its affiliates or related companies, unless otherwise noted.

©2022 Hu-Friedy Mfg. Co., LLC. All rights reserved.



Not tested for reprocessing.  
Not made with natural rubber latex.  
100% polypropylene