

Slip-Ease™ X-Ray Comfort Cushions

Large | Yellow **REF** BWSEL / Small | Blue **REF** BWSES


INSTRUCTIONS FOR USE

- **Large | Yellow: Fits size 2 film and sensors**
- **Small | Blue: Fits size 0-1 film and sensors**


Photos used in demonstration are Slip-Ease™ X-Ray Comfort Cushions Large, Yellow.



1 Place wireless sensor or plate in barrier sleeve (not included). Insert sensor into Slip-Ease™ Comfort Cushion.

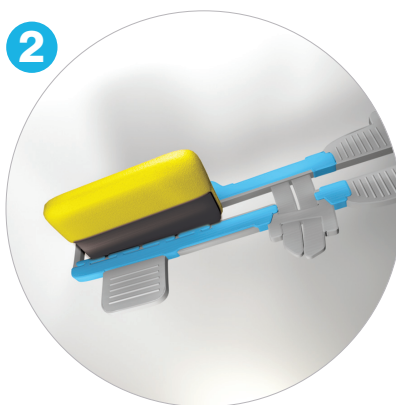

2 Remove "Peel & Stick" liner from the safety strip. Press to stick for safety.

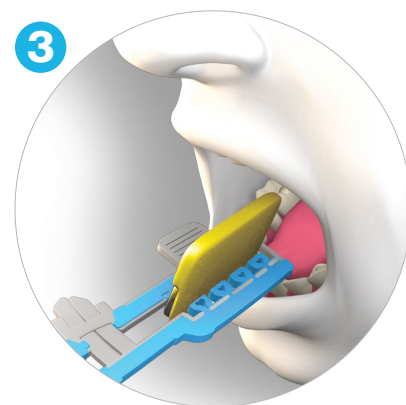

3 Take images and discard Comfort Cushion after use.



Crosstex International, Inc., a Cantel Medical Company
6789 W. Henrietta Road, Rush, NY 14543 USA
585.359.0130 | 631.582.6777 | crosstex.com



MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany



Not made with natural rubber latex.

Made in China

BWSE-IFU-EN / REV B - 04/2021

All product names are trademarks of Crosstex International, Inc., a Cantel Medical Company, its affiliates or related companies, unless otherwise noted.

©2021 Crosstex International, Inc. All rights reserved.