

Sparkle™ Prophy Angle, EZ Contra Prophy Angle

REF TPASSC1 / TPASFC1 / TPASSC5 / TPASFC5 / TPATBC1 / TPASEZ / TPAFEZ

INSTRUCTIONS FOR USE

1. Connect this polishing angle to a slow speed hand piece/attachment. (Optimal speed is 3500 RPM (58 Hz) at 30-35 psi (207-241 kpa). Do not run above 4000 (67 Hz) RPM).
2. Seat angle and secure with tightening mechanism.
3. Discard in accordance with local and national regulations.



Crosstex International, Inc., a Cantel Medical Company
 6789 W. Henrietta Road, Rush, NY 14543 USA
 585.359.0130 | 631.582.6777 | crosstex.com



MDSS GmbH
 Schiffgraben 41
 30175 Hannover, Germany



Warning: This single-use device should only be used with a sterilized handpiece between patients. The device contains small components that can pose a choking hazard



NON STERILE



Not made with natural rubber latex.

Made in China
 TPA-IFU-EN / REV C - 05/2021

All product names are trademarks of Crosstex International, Inc., a Cantel Medical Company, its affiliates or related companies, unless otherwise noted.
 ©2021 Crosstex International, Inc. All rights reserved.