ProGuard™ Fluoride-containing Desensitizing Varnish

Carefully read instructions prior to use

For use in Canada

Product Description:

ProGuard™ Fluoride-containing Desensitizing Varnish is a fluoride-containing varnish for tooth desensitisation used in the treatment of enamel and dentine surfaces. ProGuard Fluoride-containing Desensitizing Varnish also adheres to moist surfaces and tolerates residual moisture and saliva.

ProGuard Fluoride-containing Desensitizing Varnish is an ethanolic suspension on the basis of colophony with artificial flavouring and xylitol as a sweetener.

1 mL of ProGuard Fluoride-containing Desensitizing Varnish contains 50 mg sodium fluoride, which corresponds to 22.6 mg of fluoride.

Indications:

ProGuard™ Fluoride-containing Desensitizing Varnish is indicated for:

- The treatment of hypersensitive teeth.
- The treatment of sensitive root surfaces.

Contraindications:

- ProGuard™ Fluoride-containing Desensitizing Varnish contains colophony and artificial flavouring.
- ProGuard Fluoride-containing Desensitizing Varnish should not be used in patients with a known hypersensitivity (allergy) to these constituents.
- ProGuard Fluoride-containing Desensitizing Varnish should not be used in patients suffering from ulcerative gingivitis or ulcerative oral mucositis.
- ProGuard Fluoride-containing Desensitizing Varnish is not indicated for systemic treatment.

Patient Target Group:

ProGuard™ Fluoride-containing Desensitizing Varnish is suitable for use in all patients without any age or gender restrictions.

Product Performance Features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

Use:**User:**

ProGuard™ Fluoride-containing Desensitizing Varnish should only be applied by a professionally trained dental practitioner.

Recommended dose:

- Treatment of patients with deciduous teeth: Use approximately 0.25 mL of ProGuard™ Fluoride-containing Desensitizing Varnish.
- Treatment of patients with mixed or permanent teeth: Use approx. 0.40 mL of ProGuard Fluoride-containing Desensitizing Varnish.

Apply a thin film of ProGuard Fluoride-containing Desensitizing Varnish to the dental hard tissue. A thick film detaches more easily and offers no therapeutic advantages. Based on experience, the required quantity for an entire set of teeth is between 0.25 mL and 0.40 mL.

Preparation:

Clean the treatment surface thoroughly to remove all calculus. In order to attain an optimal result, excess liquid/saliva should also be removed from the surface to be treated.

Application:

Apply ProGuard™ Fluoride-containing Desensitizing Varnish evenly until the complete surface is covered with a thin film. Moisten the application area (either with a gentle rinse or by utilising the natural production of saliva) in order to ensure the ProGuard Fluoride-containing Desensitizing Varnish sets.

Warnings/Precautionary Measures:

- Patients must be instructed to avoid eating food, drinking alcohol, cleaning their teeth and flossing for the first 4 hours after the treatment.
- Colophony can affect the polymerisation and bonding adhesion of composites.
- ProGuard™ Fluoride-containing Desensitizing Varnish has an adhesive consistency corresponding to its intended purpose.
- Oedematous swelling has been reported in rare cases, particularly after use on larger surfaces.
- Breathing difficulties have been reported in children with asthma in isolated cases where comparable products have been used.
- Patients with sensitive digestive systems may experience nausea after extensive treatment.
- In the case of hypersensitivity: remove the varnish with a soft toothbrush and dental floss before rinsing with warm water.
- No other products containing fluoride should be applied on the same day as the treatment.
- If the patient is taking fluoride tablets, they should skip their daily dose for a few days after treatment with ProGuard Fluoride-containing Desensitizing Varnish.
- Do not swallow the varnish.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Storage:

Store at 4°C - 23°C. Do not use after the expiry date.

Disposal:

Dispose of the product according to local regulations.

Reporting Obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's condition and a serious risk to public health that arise or could have arisen in association with the use of ProGuard™ Fluoride-containing Desensitizing Varnish must be reported to the manufacturer and the responsible authority.

CROSSTEX

Manufactured for: Crosstex International, Inc.
6789 W. Henrietta Road, Rush, NY 14543 USA
585.359.0130 | 631.582.6777 | crosstex.com

Made in Germany

UFVP-IFU-CA-EN / REV F - 05/2022

All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC,

its affiliates or related companies, unless otherwise noted.

©2022 Hu-Friedy Mfg. Co., LLC. All rights reserved.

CR 6000 1267 CA 0522 99

•HuFriedyGroup
The Best In Practice

ProGuard™ Fluoride-containing Desensitizing Varnish

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Fabriqué pour : Crosstex International, Inc.