ProGuard™ Fluoride-containing Desensitizing Varnish

Carefully read instructions prior to use

For use in USA

Product Description:

ProGuard™ Fluoride-containing Desensitizing Varnish is a fluoride containing dental varnish for surface application to enamel and dentin. ProGuard Fluoride-containing Desensitizing Varnish will adhere to wet surfaces and is tolerant to moisture and saliva.

ProGuard Fluoride-containing Desensitizing Varnish is an ethanolic suspension of colophony with artificial flavors and sweetened with xylitol.

1 mL of ProGuard Fluoride-containing Desensitizing Varnish contains 50 mg of sodium fluoride equivalent to 22.6 mg of fluoride.

Indications:

ProGuard™ Fluoride-containing Desensitizing Varnish is intended for use as

- Treatment of hypersensitive teeth
- Treatment of sensitive root surfaces

Contraindications:

ProGuard™ Fluoride-containing Desensitizing Varnish is contraindicated in patients with ulcerative gingivitis or stomatitis or known allergies to colophony.

ProGuard Fluoride-containing Desensitizing Varnish is not intended for systemic treatment.

Methods of Use:

Recommended dosage:

- Treatment of patients with primary dentition:
Use approximately 0.25 mL of ProGuard™ Fluoride containing Desensitizing Varnish.
- Treatment of patients with adolescent or permanent dentition:
Use approx. 0.40 mL of ProGuard Fluoride-containing Desensitizing Varnish.

ProGuard Fluoride-containing Desensitizing Varnish should be applied on a complete set of teeth in a very thin film thickness. A thick film thickness is more likely to peel off and does not provide better therapeutic results. A complete set of teeth can be treated between the ranges of 0.25ml and 0.40mL.

Application:

1. Thoroughly clean tartar from treatment areas.
2. For best results, eliminate any excessive moisture saliva from area to be treated.
3. Use brush applicator or foam pellet to uniformly apply ProGuard™ Fluoride-containing Desensitizing Varnish, covering the entire surface to be treated as a thin film.
4. Allow area to become wet (either by gentle rinsing or natural salivary flow) to ensure setting of ProGuard

Fluoride-containing Desensitizing Varnish.

5. Instruct patient to avoid hard food, alcohol, brushing or flossing for the next 4 hours after application.

Precautionary Measures:

Edematous swellings have been reported on rare occasions especially following application to extensive surfaces. Direct ingestion should be avoided. Dyspnea associated with asthmatic children, has been reported for similar preparations in extremely rare cases. Patients having sensitive digestive systems may experience nausea, usually after extensive treatment. In case of adverse reactions, remove varnish with a soft toothbrush and floss, and then rinse with warm water.

No treatment with other fluoride preparations on day of application. Suspend treatment with fluoride tablets for several days after treatment. Do not swallow.

According to its purpose, ProGuard™ Fluoride-containing Desensitizing Varnish has a sticky consistency.

Storage:

Store at temperatures between 39°F - 73°F (4°C - 23°C). Do not use after expiration date.

This material has been developed solely for use in dentistry. Processing should be done strictly according to the instructions for use. Crosstex recognizes its responsibility to replace products if proven to be defective. Crosstex does not accept liability for any damage or loss, directly or indirectly, stemming from the use of or inability to use the products described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

Descriptions and data constitute no warranty of attributes and are not binding.

CAUTION: U.S. Federal Laws restrict this device to sale by or on the order of a dentist.

No person is authorized to provide any information which deviates from the information provided in the instructions for use.

**Keep this material out of reach of children.
For dental use only.**

CROSSTEX

Manufactured for: Crosstex International, Inc.
6789 W. Henrietta Road, Rush, NY 14543 USA
585.359.0130 | 631.582.6777 | crosstex.com

Made in Germany

UFVP-IFU-USA-EN / REV F - 05/2022

All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC,

its affiliates or related companies, unless otherwise noted.

©2022 Hu-Friedy Mfg. Co., LLC. All rights reserved.

CR 6000 1267 US 0522 99

•HuFriedyGroup
The Best In Practice

ProGuard™ Fluoride-containing Desensitizing Varnish

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Fabricado para: Crosstex International, Inc.